



Wear & Care Instructions

Clavicle Splint

Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

Care & Cleaning

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

Applying the Orthosis

Application of the clavicle strap requires two people.

1. Place the felt/foam pad in the center of back.
2. Extend straps-one over the left shoulder and the other over right shoulder.
3. Grasp straps and slide through buckles and pull gently.
4. Secure hook to loop to secure to desired tension.

IMPORTANT

The advice and information contained in this document is not intended as a substitute for medical counselling. If you have any questions about treatment for a specific condition, please check with a health care professional.



Unit 3 Stourton Link,
Intermezzo Drive,
Leeds LS10 1DF
Tel: +44 (0) 113 207 0435
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com

 **steeper**
Creating life's turning points, together