

Wear & Care Instructions

Universal Deep Abdominal Binder

Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

Care & Cleaning

Product should be air-dried, do not use high temperatures to wash or dry. Close all contact closures and hand wash in warm water with mild soap.

Applying the Orthosis

1. Open the abdominal binder.
2. Grasp elastic and foam sides and position over the abdominal area.
3. Gently stretch the abdominal binder and attach hook section along any part of the foam liner.

Note: Abdominal binder should not be worn for extended periods of time. Please follow the guidance from your healthcare professional.

Steeper Group
Unit 3 Stourton Link
Intermezzo Drive
Leeds LS10 1DF
Tel: 0113 270 4841
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com

 **steeper**
Creating life's turning points, together