

Wear & Care Instructions



Heel Wedge Healing Shoe

Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

Care & Cleaning

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

Applying the Orthosis

1. Place affected foot inside the boot ensuring the heel is in position.
2. Wrap the straps over the product onto opposite side, and attach straps onto hook portion of the product.
3. Straps should be pulled gently for comfortable fit.

Steeper Group
Unit 3 Stourton Link
Intermezzo Drive
Leeds LS10 1DF
Tel: 0113 270 4841
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com

 **steeper**
Creating life's turning points, together