

Wear & Care Instructions

Paediatric Hip Abduction Splint Standard



Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

Features and benefits

Standard Model

Maintains concentric reduction of hips in patients with hip dysplasia. Replaces spica casts and harnesses for certain developmental dysplasia patients.

One piece, non-adjustable polypropylene orthosis holds infant in 90 degree hip flexion and 60 degree abduction. Open design facilitates nappy changes. Closed-cell foam lining does not retain moisture.

Care & Cleaning

Plastic sections can be wiped with a cloth using soap and water. Liners may be removed and washed at 40°C in a washing machine and tumble dried.

IMPORTANT

The advice and information contained in this document is not intended as a substitute for medical counselling. If you have any questions about treatment for a specific condition, please check with a health care professional.



Unit 3 Stourton Link,
Intermezzo Drive,
Leeds LS10 1DF
Tel: +44 (0) 113 207 0435
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com

 **steeper**
Creating life's turning points, together