

# Wear & Care Instructions



## Post-Op Shoe

Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

### Care & Cleaning

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### Applying the Orthosis

1. Place affected foot inside the boot ensuring the heel is in position.
2. Wrap the straps over the shoe onto opposite side and attach straps onto hook portion of the shoe.
3. Straps should be pulled gently for comfortable fit.

Steeper Group  
Unit 3 Stourton Link  
Intermezzo Drive  
Leeds LS10 1DF  
Tel: 0113 270 4841  
Email: [orthoticenquiries@steepergroup.com](mailto:orthoticenquiries@steepergroup.com)

[www.steepergroup.com](http://www.steepergroup.com)

 **steeper**  
Creating life's turning points, together