

Wear & Care Instructions

Full Length Silicone Insoles

Steeper manufactures a quality range of orthotic products and to help you care for the product, this leaflet provides information on how and when to use your prescribed orthosis, how to care for it and how to contact us if you have a problem or require a replacement.

Features and Benefits

Silicone Insoles absorb and dissipate shock forces that may cause pain or discomfort. Designed for relief of heel pain and light rear foot trauma, for use during sporting activities, or simply to improve comfort. Manufactured from medical grade silicone, each insole incorporates calcaneal support.

Indications for Use

- Heel pain
- Calcaneal spur
- Plantar fasciitis

Care and Cleaning

- Hand wash in warm water with mild soap.
- Product should be air-dried, do not use high temperatures to wash or dry.

Steeper Group
Unit 3 Stourton Link
Intermezzo Drive
Leeds LS10 1DF
Tel: 0113 270 4841
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com