



CLICK  
MEDICAL

## QuickFit™ Strap Fitting Instructions

1" - OK1400-130-05  
1.5" - OK1410-130-05  
2" - OK1420-130-05

### Kit Contents:


#### QuickFit Strap




#### Adhesive Velcro Tab



#### 1 Install Velcro at edges of devise.

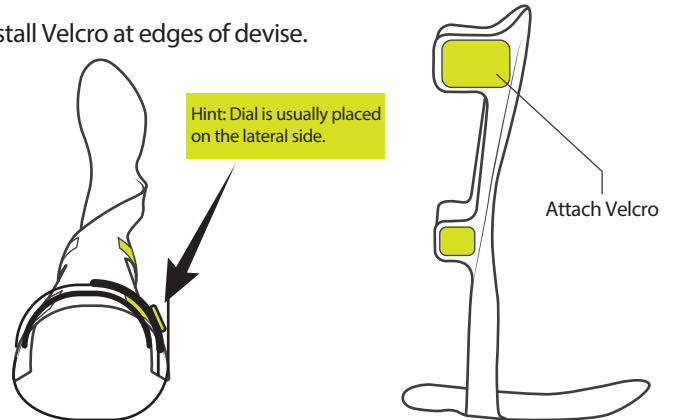


#### 2 Check guide position.

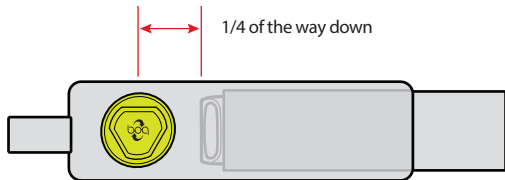


Important: Add optional QuickFit Buckle if desired before continuing to step 3:


#### 3 Position tunnel opening at edge of desired closure, insure strap path is clear of interference.

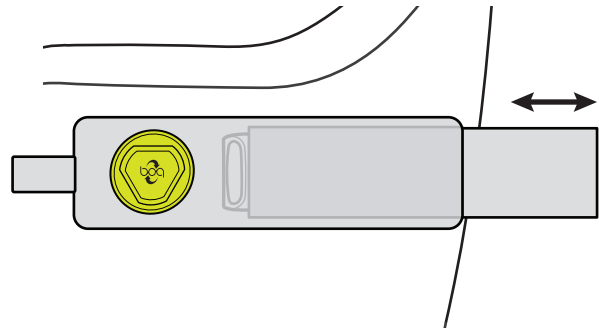


#### 4 Wrap strap from dial around body to determine strap length.

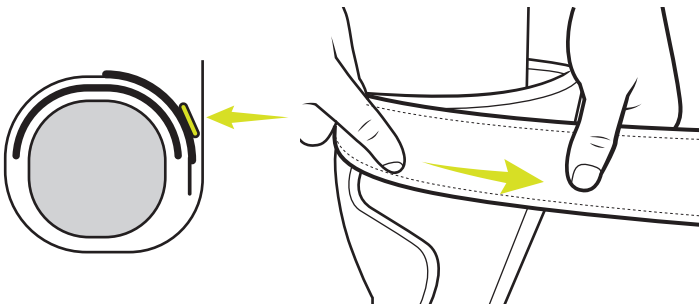


#### 5 Cut strap to size at dial.

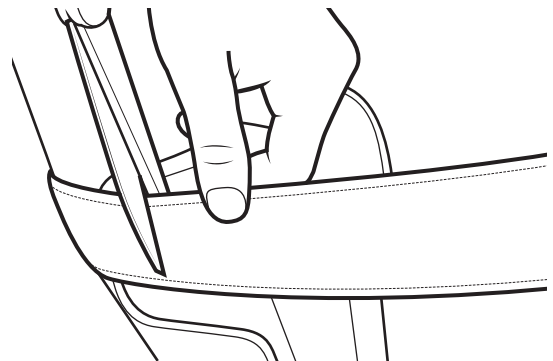

#### 6 Push to engage dial: Twist clockwise to tighten.

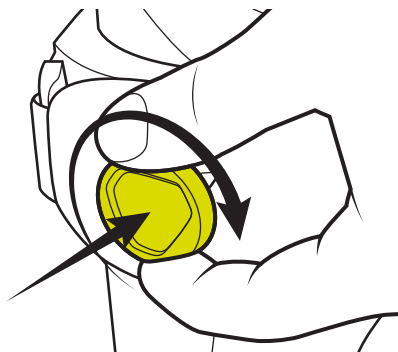

#### 7 Pull up on dial to release tension: Lift Velcro tab to release.

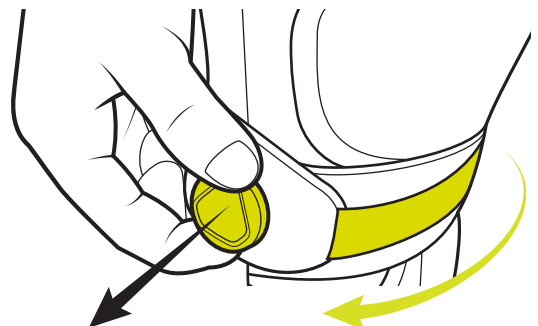




CLICK  
MEDICAL


## QuickFit™ Buckle Add-On Instructions

1" - OK1405-000-05  
1.5" - OK1415-000-05  
2" - OK1425-000-05


- 1 Install QuickFit buckle on device with included Velcro tabs.

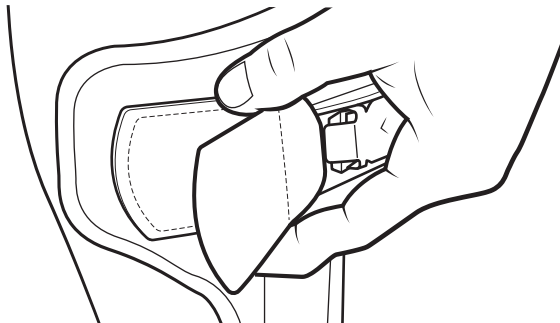


- 2 Attach QuickFit strap:  
A. Insert Velcro

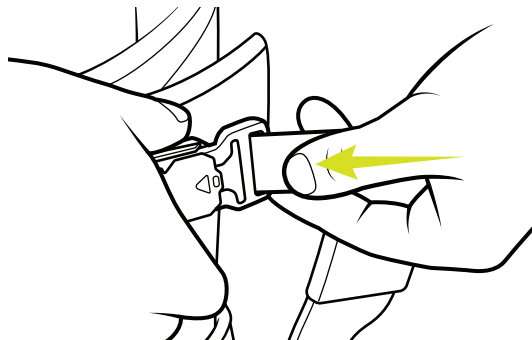

- B. Loop Velcro tab through buckle.

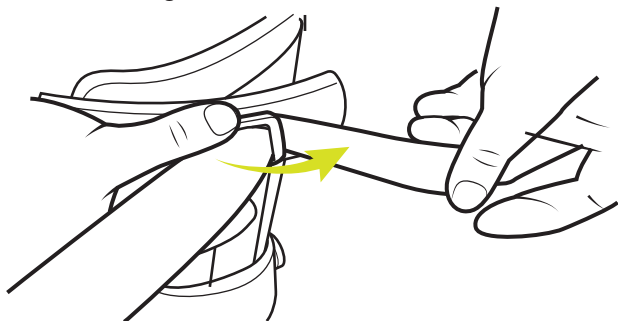


- C. Adhere Velcro to back of strap.

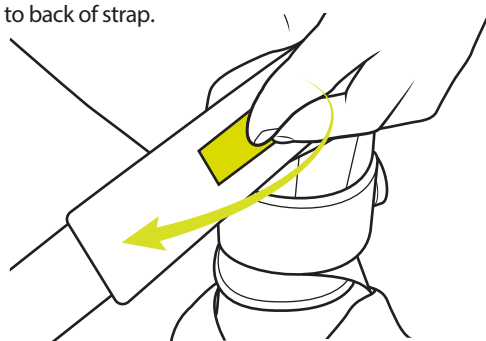


See QuickFit Strap for sizing instructions

- 3 Cut strap to size and attach to Velcro end of buckle.

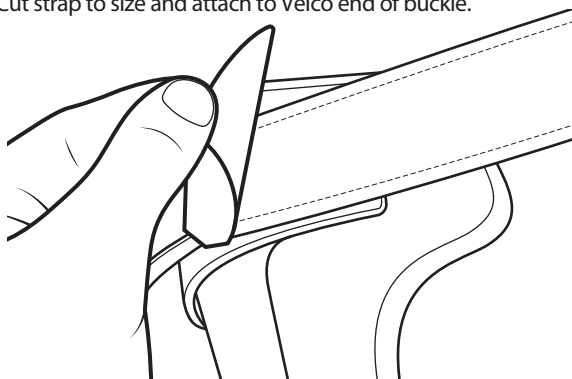


- 4 A. Close strap at self-securing buckle.  
B. Push to engage dial, twist clockwise to tighten strap.

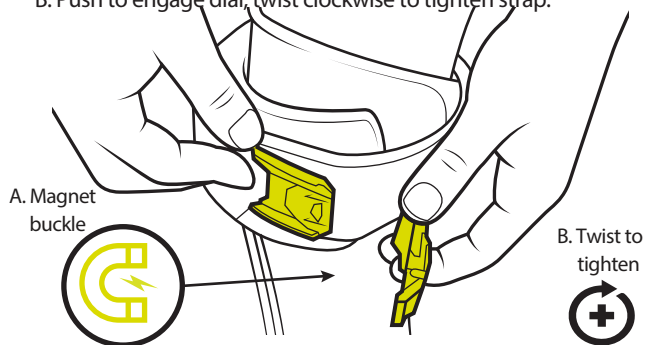

To release strap:

- 1 Pull up dial for quick release.

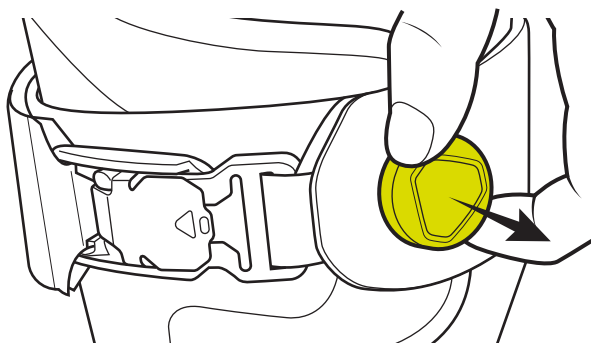

- 2 Pull tab to open.

