



Therapeutic Shoes



Therapeutic Shoes



The Streifeneder therapeutic shoe range covers a variety of applications, helping to ensure a suitable orthosis for every patient. At Steeper, we inherently understand the challenges and pressures that clinicians face, which is why we constantly seek out and evaluate new products that will help create turning points and transform lives. By partnering with world-renowned manufacturers of lower limb orthoses, we can offer clinical independence to clinicians and customers.

Designed to accommodate sometimes large bandages or wound dressings; the Streifeneder therapeutic shoe range also protects the injured feet from dirt, splashing water and the cold. All Streifeneder therapeutic shoes offer a variety of features and benefits that help improve healing, reduce infection and aid patient recovery.

For more information on the Streifeneder collection or to place an order please call our friendly team on 0113 207 0435 or e-mail: orthotickenquiries@steepergroup.com.

Easy Selection Guide

4	Venus	11	Uranus
5	Mars	12	Pluto S
6	Mars S	13	Apollo
7	Saturn	14	Luna
8	Jupiter	15	Komet
9	Merkur	16	Neptun
10	Merkur A	17	Neptun B

Venus

Partial foot decompression shoe, short version



Features and Benefits

- A rigid, ascending out-sole towards the front and a shortened sole construction (negative heel)
- Pressure relief and pressure distribution during the walking cycle
- Can be worn on either the left or right foot
- Improves the healing process of wounds in the forefoot area
- The forefoot relief shoe embraces the entire heel and tarsal area with a softly padded shoe upper
- Heel cap, hook and loop fasteners prevent slipping within the shoe
- Antibacterial shoe lining and fabric cover of the insole

Indication for Use

- Post-surgical after corrections and forefoot injuries

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. flordid charcot foot)
- Use only with walking supports



200R10




200R14

Venus

Accessories and Pressure Distribution Map

Adjustable Safety Guard		Universal Toe Bumper	
Shoe Size	Part Number	Shoe Size	Part Number
3-5	200R10/S	3-5	200R14/S
6-8	200R10/M	6-8	200R14/M
9-11	200R10/L	9-11	200R14/L
12-15	200R10/XL	12-15	200R14/XL

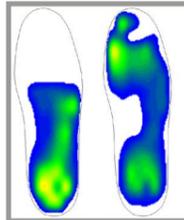

Mars

Heel relief shoe



203R11



203R10

Features and Benefits

- A padded metatarsus roll and a shortened sole construction towards the heel area
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the heel area
- The heel relief shoe embraces ankle and tarsus with a softly padded shoe upper
- Hook and loop fasteners hold the foot in predefined position and prevent slipping within the shoe
- Can be worn on either the left or right foot
- The padded insole prevents pressure peaks in the forefoot area
- Antibacterial shoe lining and fabric cover of the insole

Indication for Use

- Wounds in the heel area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds in the heel area
- Post-surgical and traumas

Contraindications


- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. flordid charcot foot)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-4	22.5cm	8.7cm	203R1/S
5-6.5	23.5cm	9.5cm	203R1/M
7-9	25.5cm	9.5cm	203R1/L
9.5-11	27.5cm	10cm	203R1/XL

Mars

Accessories and Pressure Distribution Map

Heel Protection Cap, with hook and loop fastener on the outside		Forefoot Protection Cap, with hook and loop fastener on the outside	
Shoe Size	Part Number	Shoe Size	Part Number
2-4	203R11/S	2-4	203R10/S
5-6.5	203R11/M	5-6.5	203R10/M
7-9	203R11/L	7-9	203R10/L
9.5-11	203R11/XL	9.5-11	203R10/XL

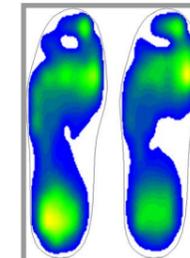

Mars S

Heel relief shoe



Features and Benefits

- New improved sole construction affects targeted shifting of the body centre of gravity onto the forefoot with at the same time comfortable foot roll-over
- The shoe can be worn on the left or right side, due to a hook and loop pad
- The pad can be customised depending on the anatomy of the medial arch, this provides improved medial support and relief of the heel
- An especially effective and long padded insole prevents plantar pressure peaks
- The shaft is individually adjustable to the ankle, back of the foot and forefoot area
- Antibacterial shoe lining and fabric cover of the insole

Indication for Use

- Wounds in the heel area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds in the heel area
- Post-surgical and traumas

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports


UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
1-3	24cm	8.7cm	213R1/S
4-6	26cm	9.1cm	213R1/M
6.5-8	28cm	10cm	213R1/L
9-10	30cm	10.5cm	213R1/XL
11-13	32cm	10.9cm	213R1/XXL



Mars S

Accessories and Pressure Distribution Map

Forefoot Cap, with hook and loop fastener on the outside		Heel Protection Cap, with hook and loop fastener on the outside		Cushioning Sole, 10mm, soft foam with velour coating	
Shoe Size	Part Number	Shoe Size	Part Number	Shoe Size	Part Number
1-3	213R10/S	1-3	213R11/S	1-3	213R12/S
4-6	213R10/M	4-6	213R11/M	4-6	213R12/M
6.5-8	213R10/L	6.5-8	213R11/L	6.5-8	213R12/L
9-10	213R10/XL	9-10	213R11/XL	9-10	213R12/XL
11-13	213R10/XXL	11-13	213R11/XXL	11-13	213R12/XXL

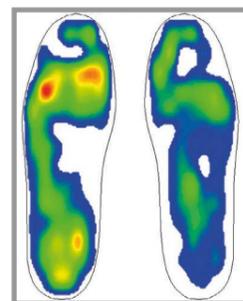

Saturn

Forefoot relief shoe



Features and Benefits

- An ascending out-sole towards the front
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- The shoe can be worn on the left or right side
- The forefoot relief shoe embraces heel and tarsus with a softly padded shoe upper
- Heel cap, hook and loop fasteners prevent slipping within the shoe
- Antibacterial shoe lining and fabric cover of the insole

Indication for Use

- Wounds in the forefoot area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds in the forefoot area
- Post-surgical after corrections and forefoot injuries

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports


UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
3-4	24.5cm	9.6cm	207R1/XS
5-6	27.1cm	9.8cm	207R1/S
6.5-7	27.1cm	10cm	207R1/M
8-9	28.4cm	10.2cm	207R1/L
9.5-10	29.7cm	10.4cm	207R1/XL



Saturn

Accessories and Pressure Distribution Map

Forefoot Protective Cap, with hook and loop fastener on the outside	
Shoe Size	Part Number
3-4	207R10/XS
5-6	207R10/S
6.5-7	207R10/M
8-9	207R10/L
9.5-10	207R10/XL

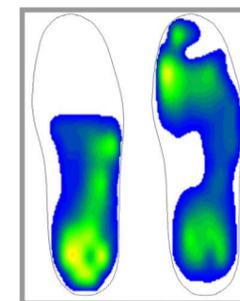

Jupiter

Forefoot relief shoe



Features and Benefits

- A rigid, ascending out-sole towards the front
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- The shoe can be worn on left or right foot
- The forefoot relief shoe embraces the entire heel and tarsal area with a softly padded shoe upper
- Heel cap, hook and loop fasteners prevent slipping within the shoe
- Hook and loop insole for individual customisation
- Improved slip protection by integrated anti-slip nubs
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Wounds in the forefoot area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical after corrections and forefoot injuries

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. flord charcot foot)
- Use only with walking supports



Walking sole with sole spots



205R10




205R12

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
11.5-12.5	22cm	8.7cm	205R1/XS
1-3	24cm	9.1cm	205R1/S
4-6	26cm	9.6cm	205R1/M
6.5-8	28cm	10.5cm	205R1/L
9-10	30cm	10.5cm	205R1/XL
11-13	32cm	10.9cm	205R1/XXL
14-14.5	34cm	11.5cm	205R1/XXXL

Jupiter

Accessories and Pressure Distribution Map

Forefoot Protective Cap, with hook and loop fastener on the outside		Cushioning Sole, 10mm, soft foam with velour coating	
Shoe Size	Part Number	Shoe Size	Part Number
11.5-12.5	205R10/XS	11.5-12.5	205R12/XS
1-3	205R10/S	1-3	205R12/S
4-6	205R10/M	4-6	205R12/M
6.5-8	205R10/L	6.5-8	205R12/L
9-10	205R10/XL	9-10	205R12/XL
11-13	205R10/XXL	11-13	205R12/XXL
14-14.5	205R10/XXXL	14-14.5	205R12/XXXL

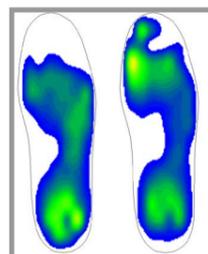

Merkur

Forefoot relief shoe



Features and Benefits

- A rigid, ascending out-sole towards the front (negative heel)
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- The forefoot relief shoe embraces the entire heel and tarsal area with a softly padded shoe upper
- The shoe can be worn on left or right foot
- Heel cap, hook and loop fasteners prevent slipping within the shoe
- Hook and loop insole for individual customisation
- Improved slip protection by integrated anti-slip nubs
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Wounds in the forefoot area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical after corrections and forefoot injuries

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. flord charcot foot)
- Use only with walking supports



Walking sole with sole spots



201R10




201R12

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
11.5-12.5	22cm	8.7cm	201R1/XS
1-3	24cm	9.1cm	201R1/S
4-6	26cm	9.6cm	201R1/M
6.5-8	28cm	10.5cm	201R1/L
9-10	30cm	10.5cm	201R1/XL
11-13	32cm	10.9cm	201R1/XXL

Merkur

Accessories and Pressure Distribution Map

Forefoot Protective Cap, with hook and loop fastener on the outside		Cushioning Sole, 10mm, soft foam with velour coating	
Shoe Size	Part Number	Shoe Size	Part Number
11.5-12.5	201R10/XS	11.5-12.5	201R12/XS
1-3	201R10/S	1-3	201R12/S
4-6	201R10/M	4-6	201R12/M
6.5-8	201R10/L	6.5-8	201R12/L
9-10	201R10/XL	9-10	201R12/XL
11-13	201R10/XXL	11-13	201R12/XXL

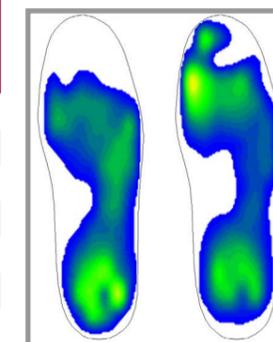

Merkur A

Partial foot decompression shoe



Features and Benefits

- A rigid, slightly ascending out-sole towards the front (negative heel)
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- The post-surgical shoe embraces the entire foot and ankle area with a softly padded shoe upper
- Hook and loop fasteners at the wide flap prevent slipping within the shoe
- The removable padded insole prevents pressure peaks in the foot sole area
- The flap can be opened widely and together with the individually adjustable hook and loop fasteners, providing sufficient space for bandaged feet
- Improved slip protection by integrated anti-slip nubs
- This shoe can be worn on right or left side and also provides height compensation when using Jupiter or Merkur to avoid pelvic obliquity
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical and traumas

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-3	23.8cm	9.1cm	202R1/36
4-5	25.1cm	9.2cm	202R1/38
6-6.5	26.4cm	9.5cm	202R1/40
7-8	27.7cm	9.7cm	202R1/42
9-9.5	29cm	10cm	202R1/44
10-11	30.3cm	10.2cm	202R1/46
12-13	31.6cm	10.5cm	202R1/48
14-14.5	33cm	11.2cm	202R1/50



Walking sole with sole spots




202R12

Merkur A

Accessories and Pressure Distribution Map

Cushioning Sole, 8mm, soft foam with velour coating	
Shoe Size	Part Number
2-3	202R12/36
4-5	202R12/38
6-6.5	202R12/40
7-8	202R12/42
9-9.5	202R12/44
10-11	202R12/46
12-13	202R12/48
14-14.5	202R12/50

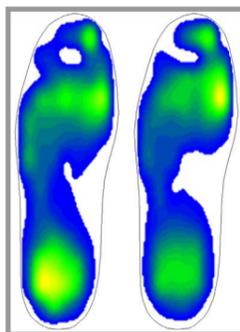

Uranus

Partial foot decompression shoe



Features and Benefits

- A rigid, slightly ascending out-sole towards the front (negative heel)
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- The post-surgical shoe embraces the entire foot and ankle area with a softly padded shoe upper
- Hook and loop fasteners at the wide flap prevent slipping within the shoe
- The removable padded insole prevents pressure peaks in the foot sole area
- The flap can be opened widely and together with the individually adjustable hook and loop fasteners, providing sufficient space for bandaged feet
- Improved slip protection by integrated anti-slip nubs
- This shoe can be worn on the right or left side and also provides height compensation to avoid pelvic obliquity
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds not subject to indication in the foot- and ankle area
- Post-surgical and traumas

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports



Walking sole with sole spots




201R12

Uranus

Accessories and Pressure Distribution Map

Cushioning Sole, 8mm, soft foam with velour coating	
Shoe Size	Part Number
2-3	204R12/36
4-5	204R12/38
6-6.5	204R12/40
7-8	204R12/42
9-9.5	204R12/44
10-11	204R12/46
12-13	204R12/48
14-14.5	204R12/50

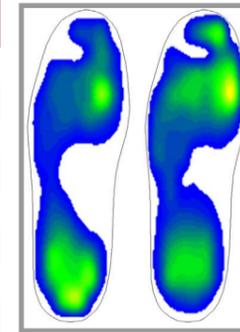

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-3	23.8cm	9.1cm	204R1/36
4-5	25.1cm	9.2cm	204R1/38
6-6.5	26.4cm	9.5cm	204R1/40
7-8	27.7cm	9.7cm	204R1/42
9-9.5	29cm	10cm	204R1/44
10-11	30.3cm	10.2cm	204R1/46
12-13	31.6cm	10.5cm	204R1/48
14-14.5	33cm	11.2cm	204R1/50

Pluto S

Partial foot decompression shoe



Walking sole with sole spots



216R11

216R12

Features and Benefits

- A rigid, slightly ascending out-sole towards the front (negative heel)
- Pressure relief and pressure distribution during the walking cycle
- Improves the healing process of wounds in the forefoot area
- Hook and loop fasteners at the wide flap prevent slipping within the shoe
- The front flap prevents lateralisation of the forefoot; the patient does not slip to the outer side providing an increased feeling of safety (if it is not needed, the front flap can be cut off)
- This shoe can be worn on the right or left side and also provides height compensation to avoid pelvic obliquity.
- The removable padded insole prevents pressure peaks in the foot sole area
- The flap can be opened widely and together with the individually adjustable hook and loop fasteners, providing sufficient space for bandaged feet
- Improved slip protection by integrated anti-slip nubs
- Antibacterial shoe lining and fabric cover of the insole


Indication for use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical and traumas

Contraindications

- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for instable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-3	23.8cm	9.1cm	216R1/36
4-5	25.1cm	9.2cm	216R1/38
6-6.5	26.4cm	9.5cm	216R1/40
7-8	27.7cm	9.7cm	216R1/42
9-9.5	29cm	10cm	216R1/44
10-11	30.3cm	10.2cm	216R1/46
12-13	31.6cm	10.5cm	216R1/48
14-14.5	33cm	11.2cm	216R1/50

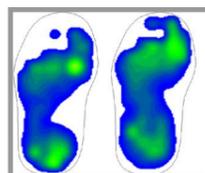

Pluto S

Accessories and Pressure Distribution Map

Cushioning Sole, 10mm, soft foam with velour coating		Forefoot Protective Cap, with hook and loop fastener on the outside	
Shoe Size	Part Number	Shoe Size	Part Number
2-3	216R11/36	2-3	216R12/36
4-5	216R11/38	4-5	216R12/38
6-6.5	216R11/40	6-6.5	216R12/40
7-8	216R11/42	7-8	216R12/42
9-9.5	216R11/44	9-9.5	216R12/44
10-11	216R11/46	10-11	216R12/46
12-13	216R11/48	12-13	216R12/48
14-14.5	216R11/50	14-14.5	216R12/50

Apollo

Partial foot decompression shoe with zero-heel technology



209R11



209R12

Features and Benefits

- Post-surgical shoe with distinctive rocker sole follows the no-heel-technology thereby relieving the whole foot during stance and swing phase
- Improves the healing process of wounds in the forefoot area
- This shoe can be worn on the left or right side, but cannot be worn in pairs
- The removable padded insole prevents pressure peaks in the foot sole area
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Post-surgical to help heal surgical operations and wounds of the forefoot, such as hallux valgus corrections
- Due to the shape of the rocker bottom sole, the user experiences quick mobilisation. For certain surgical techniques such as close-to-base metatarsal alignment, the mobilisation must be discussed and if necessary, a different therapy shoe type must be selected.

Contraindications


- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-3	23.8cm	8.2cm	209R1/36
4-5	25.1cm	8.5cm	209R1/38
6-6.5	26.4cm	8.8cm	209R1/40
7-8	27.7cm	9.1cm	209R1/42
9-9.5	29cm	9.4cm	209R1/44
10-11	30.3cm	9.7cm	209R1/46
12-13	31.6cm	10cm	209R1/48
14-14.5	33cm	10.3cm	209R1/50

Apollo

Accessories and Pressure Distribution Map

Cushioning Sole, 10mm, soft foam with velour coating		Forefoot Protective Cap, with hook and loop fastener on the outside	
Shoe Size	Part Number	Shoe Size	Part Number
2-3	209R11/36	2-3	209R12/36
4-5	209R11/38	4-5	209R12/38
6-6.5	209R11/40	6-6.5	209R12/40
7-8	209R11/42	7-8	209R12/42
9-9.5	209R11/44	9-9.5	209R12/44
10-11	209R11/46	10-11	209R12/46
12-13	209R11/48	12-13	209R12/48
14-14.5	209R11/50	14-14.5	209R12/50

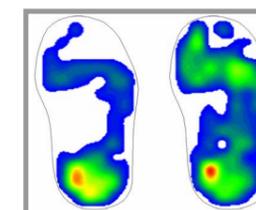

Luna

Long-term surgical shoe with three removable cushioning soles and reinforced out-sole



Features and Benefits

- The sandal-like, wide surgical shoe with hoisted toe rim is made to encompass large volume dressings and protect them from shifting
- Three removable and adjustable cushioning soles additionally relieve delicate pressure peaks from the plantar foot sole
- The reinforced out-sole relieves the injured foot
- The removable cushioning soles can be individually adjusted depending on the indication
- The higher shaft and the stable reinforced heel area offers the foot a secure grip
- The anti slip sole significantly improves both the mid-stance phase and lateral stability
- Antibacterial shoe lining and fabric cover of the insole

Indication for use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical and traumas

Contraindications

- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code Left	Product Code Right
2-3	23.8cm	9.9cm	210R1/36L	210R1/36R
4-5	25.1cm	10.2cm	210R1/38L	210R1/38R
6-6.5	26.4cm	10.5cm	210R1/40L	210R1/40R
7-8	27.7cm	10.8cm	210R1/42L	210R1/42R
9-9.5	29cm	11.1cm	210R1/44L	210R1/44R
10-11	30.3cm	11.4cm	210R1/46L	210R1/46R
12-13	31.6cm	11.7cm	210R1/48L	210R1/48R
14-14.5	33cm	12cm	210R1/50L	210R1/50R



Anti slip sole



210R11




210R12

Luna

Accessories and Pressure Distribution Map

Cushioning Sole Set with textile lamination 5mm EVA, approx. 10 Shore, 5mm EVA, approx. 20 Shore, 5mm EVA, approx. 30 Shore		
Shoe Size	Part Number Left	Part Number Right
2-3	210R11/36L	210R11/36R
4-5	210R11/38L	210R11/38R
6-6.5	210R11/40L	210R11/40R
7-8	210R11/42L	210R11/42R
9-9.5	210R11/44L	210R11/44R
10-11	210R11/46L	210R11/46R
12-13	210R11/48L	210R11/48R
14-14.5	210R11/50L	210R11/50R

Forefoot Protective Cap, with hook and loop fastener on the outside		
Shoe Size	Part Number Left	Part Number Right
2-3	210R12/36L	210R12/36R
4-5	210R12/38L	210R12/38R
6-6.5	210R12/40L	210R12/40R
7-8	210R12/42L	210R12/42R
9-9.5	210R12/44L	210R12/44R
10-11	210R12/46L	210R12/46R
12-13	210R12/48L	210R12/48R
14-14.5	210R12/50L	210R12/50R

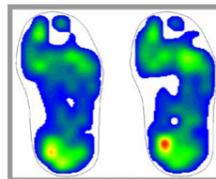

Komet

Especially low partial foot decompression shoe in outdoor-design



Features and Benefits

- Pressure distribution and relief in the forefoot area by rigid/stiff rocker sole
- Benefits the healing process of wounds in the forefoot area
- Removable padded insole absorbs pressure peaks
- Reinforced heel area and flaps in the forefoot stabilise the foot
- Adjustable hook and loop closures of the flaps for large volume dressings
- Wide, cornered sole shape in the forefoot area for surgery techniques with Kirschner-wire
- Highly slip-proof by profiled rocker sole
- Anti-bacterial fabric top cover of the insole is based on ammonium-silane-compound; inhibiting the colonisation of certain bacterial strains (e.g. staphylococcus aureus, klebsiella pneumoniae)

Indication for use

- Wounds in the forefoot area caused by diabetes mellitus
- Peripheral artery occlusions
- Post-surgical and traumas

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Do not use for unstable or fracture prone diabetic neuro-osteoarthropathic foot syndrome, DNOAP (e.g. florid charcot foot)
- Use only with walking supports



217R11



217R12

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code
2-3	24.2cm	10.3cm	217R1/36
4-5	25.5cm	10.6cm	217R1/38
6-6.5	26.8cm	10.9cm	217R1/40
7-8	28.2cm	11.2cm	217R1/42
9-9.5	29.5cm	11.5cm	217R1/44
10-11	30.8cm	11.8cm	217R1/46
12-13	32.2cm	12.1cm	217R1/48
14-14.5	33.5cm	12.4cm	217R1/50

Komet

Accessories

Cushioning Sole, 10mm, soft foam with velour coating		Forefoot Protective Cap, with hook and loop fastener on the outside	
Shoe Size	Part Number	Shoe Size	Part Number
2-3	217R11/36	2-3	217R12/36
4-5	217R11/38	4-5	217R12/38
6-6.5	217R11/40	6-6.5	217R12/40
7-8	217R11/42	7-8	217R12/42
9-9.5	217R11/44	9-9.5	217R12/44
10-11	217R11/46	10-11	217R12/46
12-13	217R11/48	12-13	217R12/48
14-14.5	217R11/50	14-14.5	217R12/50

Neptun

Long-term surgical shoe



Features and Benefits

- Surgical shoe designed to hold feet with voluminous bandages and protect them from slipping within the shoe as well as from cold and wet weather
- Embraces the entire foot and ankle area with a softly padded shoe upper
- Hook and loop fasteners at the wide flap prevent slipping within the shoe
- The removable padded insole prevents pressure peaks in the foot sole area
- The flap can be opened widely and together with the individually adjustable hook and loop fasteners, providing sufficient space for bandaged feet
- Antibacterial shoe lining and fabric cover of the insole



Indication of use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds not subject to indication in the foot and ankle area
- Post-surgical and traumas

Contraindications

- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Use only with walking supports




214R12

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code	
			Shoe	
			Left	Right
3	24.3cm	9.3cm	214R1/36L	214R1/36R
4	24.9cm	9.4cm	214R1/37L	214R1/37R
5	25.6cm	9.6cm	214R1/38L	214R1/38R
6	26.3cm	9.7cm	214R1/39L	214R1/39R
6.5	26.9cm	9.9cm	214R1/40L	214R1/40R
7	27.6cm	10cm	214R1/41L	214R1/41R
8	28.3cm	10.2cm	214R1/42L	214R1/42R
9	29cm	10.3cm	214R1/43L	214R1/43R
9.5	30cm	10.5cm	214R1/44L	214R1/44R
10	30.3cm	10.6cm	214R1/45L	214R1/45R
11	30.9cm	10.8cm	214R1/46L	214R1/46R
12	31cm	10.9cm	214R1/47L	214R1/47R
13	32cm	11cm	214R1/48L	214R1/48R

Neptun

Accessories and Pressure Distribution Map

Cushioning Sole, 3mm, soft foam with velour coating		
Shoe Size	Part Number Left	Part Number Right
3	214R12/36L	214R12/36R
4	214R12/37L	214R12/37R
5	214R12/38L	214R12/38R
6	214R12/39L	214R12/39R
6.5	214R12/40L	214R12/40R
7	214R12/41L	214R12/41R
8	214R12/42L	214R12/42R
9	214R12/43L	214R12/43R
9.5	214R12/44L	214R12/44R
10	214R12/45L	214R12/45R
11	214R12/46L	214R12/46R
12	214R12/47L	214R12/47R
13	214R12/48L	214R12/48R

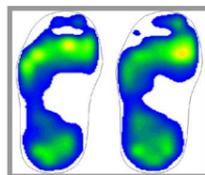

Neptun B

Long-term surgical shoe with wide out-sole & high volume



Features and Benefits

- This surgical shoe is designed to hold feet with voluminous bandages and protect them from slipping within the shoe as well as from cold and wet weather
- Embraces the entire foot and ankle area with a softly padded shoe upper
- Hook and loop fasteners at the wide flap prevent slipping within the shoe
- The removable padded insole prevents pressure peaks in the foot sole area
- The anti slip sole significantly improves both the mid-stance phase and lateral stability
- The flap can be opened widely and together with the individually adjustable hook and loop fasteners, providing sufficient space for bandaged feet
- The high shaft offers sufficient stability
- Antibacterial shoe lining and fabric cover of the insole



Anti slip sole



212R11

Indication for use

- Wounds in the foot and ankle area caused by diabetes mellitus
- Peripheral artery occlusions
- Other wounds not subject to indication in the foot- and ankle area
- Post-surgical and traumas
- Rehabilitation of lymphoedema, ulcus cruris venosum, elephantiasis, adipositas

Contraindications


- Do not use for the above indications, if a sufficient pressure relief cannot be achieved
- Do not wear directly on injured, irritated or damaged skin (before application, please cover the skin with wound dressing or a sock)
- Use only with walking supports

UK Shoe Size	Inner Shoe Length	Inner Shoe Width	Product Code	
			Shoe	
			Left	Right
2-3	23.8cm	9.9cm	215R1/36L	215R1/36R
4-5	25.1cm	10.2cm	215R1/38L	215R1/38R
6-6.5	26.4cm	10.5cm	215R1/40L	215R1/40R
7-8	27.7cm	10.8cm	215R1/42L	215R1/42R
9-9.5	29cm	11.1cm	215R1/44L	215R1/44R
10-11	30.3cm	11.4cm	215R1/46L	215R1/46R
12-13	31.6cm	11.7cm	215R1/48L	215R1/48R
14-14.5	33cm	12cm	215R1/50L	215R1/50R

Neptun B

Accessories and Pressure Distribution Map

Cushioning Sole, 8mm, soft foam with velour coating		
Shoe Size	Part Number Left	Part Number Right
2-3	212R11/36L	212R11/36R
4-5	212R11/38L	212R11/38R
6-6.5	212R11/40L	212R11/40R
7-8	212R11/42L	212R11/42R
9-9.5	212R11/44L	212R11/44R
10-11	212R11/46L	212R11/46R
12-13	212R11/48L	212R11/48R
14-14.5	212R11/50L	212R11/50R

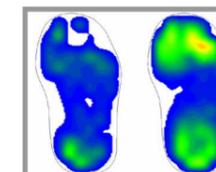


Diabetic Specification



Over the last decade, the number of patients with diabetes has risen by 150% and with the condition growing at a particularly high rate, diabetes is now one of the world's most common long-term health conditions. Diabetes, like any health condition, should be taken seriously and, as well as medical care, technical orthopaedic care also requires intensive preventative measures on the part of the patient. Diabetes can be chronic in nature and thus delay wound healing, for this reason, attention is paid to making durable therapeutic shoes of particularly high quality. Our therapeutic shoe collection is suitable for patients with both Type 1 and Type 2 Diabetes.

Wounds on the lower extremities are most frequently treated as part of modern wound management. Therapeutic shoes adjusted to the wearer are part of complete wound care and protection of the debridement (wound cleaning). These do not only have the task of encompassing often voluminous dressings, but they also protect the injured feet from dirt, spray and the cold. Different shoes can be used depending on the wound formation in the area of the foot and ankle. Please see the adjacent chart for further details on the full range of therapeutic shoes.

Therapeutic Shoes Diabetic Foot Offloading Chart

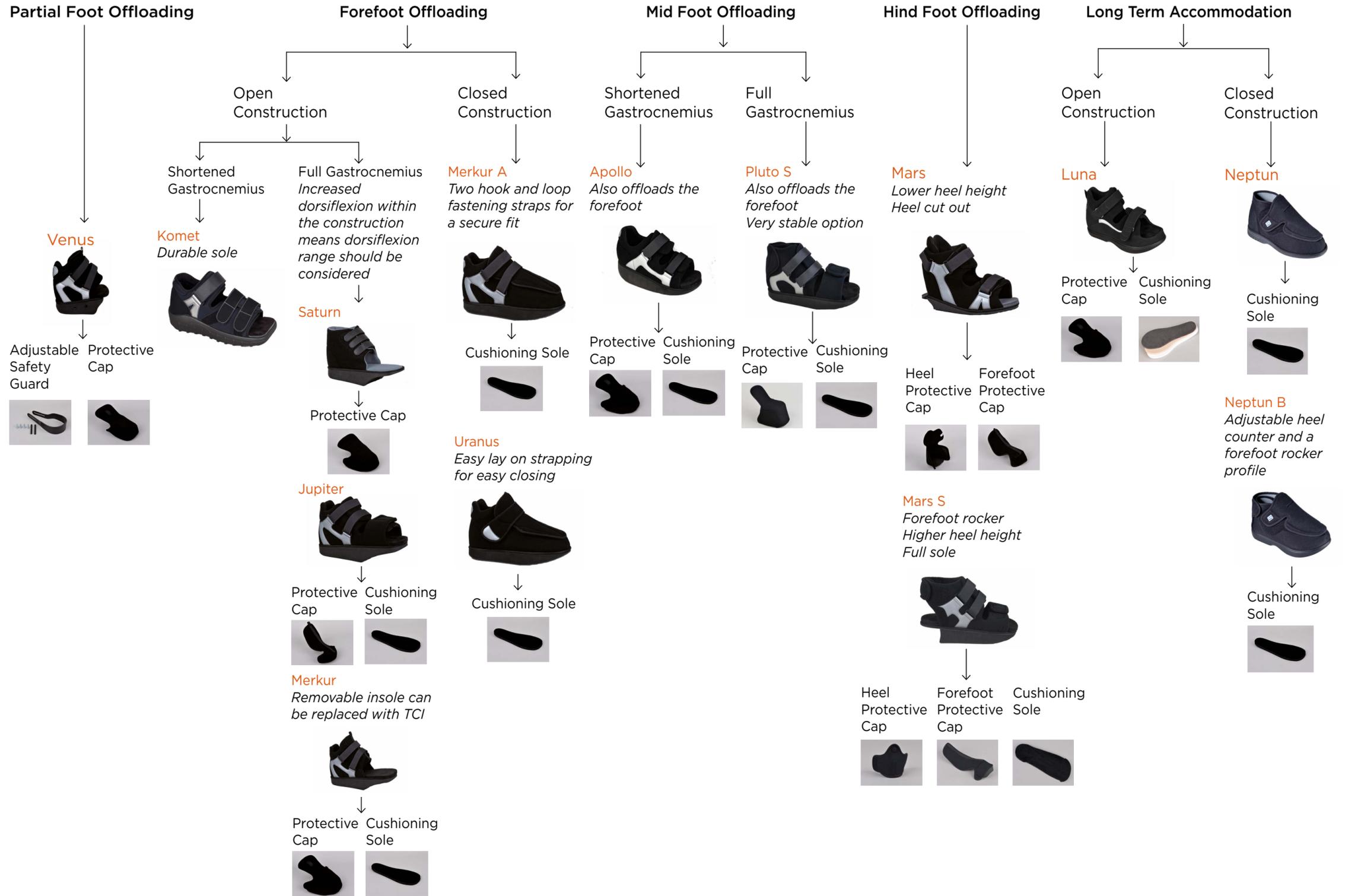

Education



With its high-quality selection, our range of comprehensive therapeutic shoes provides assistance to orthopaedic professionals and orthopaedic shoe technicians in seeking out and adjusting the right shoe once the need has been indicated. Ultimately, our common goal is to prevent the risk of a wound arising and impending toe and foot amputation.

The adjacent chart highlights the therapeutic shoes and accessories that can be used to provide offloading to different areas of the foot.

For more information on the Streifender collection, for demonstrations and product advice or to place an order please call our friendly team on 0113 207 0435 or e-mail: orthotickenquiries@steepergroup.com.



Steeper Group
Unit 3 Stourton Link
Intemezzo Drive
Leeds
LS10 1DF

Tel: +44 (0) 113 207 0435
Email: orthoticenquiries@steepergroup.com

www.steepergroup.com

STPOR030 Issue 2