
RevoFit2™ Diagnostic Kit

PK2150-320-05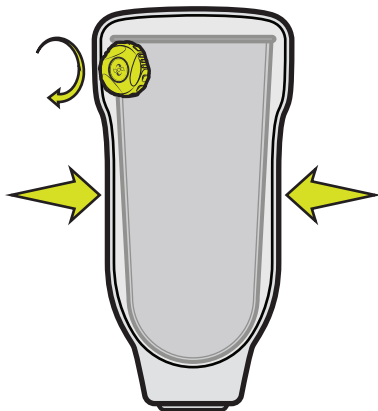

Advisory:

Do not use on patients with neuropathy, neuromas, or other nerve issues.
Fabrication only by a certified professional, trained in the RevoFit™ system.
Filament will wear, inspect and replace regularly.

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
WELCOME TO
RevoFitTM

Thank you for
being a champion
of **adjustability!**


Your expertise gives
RevoFit technology
the **power to**
change lives.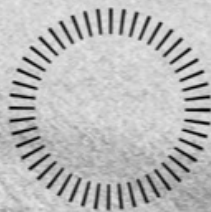

CLICK

RevoFit2™ Diagnostic Kit Contents:

6ft Teflon Tubing
7ft of Lace + Lace Feeder

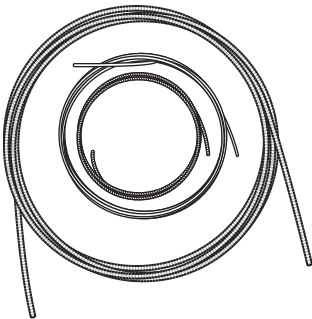RevoFit2
Dial Base

High Power
Boa Dial

Recommended Designs:

Below Knee:
3 Panel Design
or Symes Door

Above Knee:
Adjustable Medial
/Lateral Panels
+
Adjustable Strap
in Definitive
(additional kits required)


+Additional Kit Required


Recommended
Suspension System:
RevoLock™ Lanyard Kit
PN:3000-220-05

Design Overview:

- 1 Determine location of panels or areas of adjustment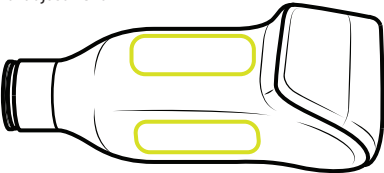

- 2 Determine location of reel

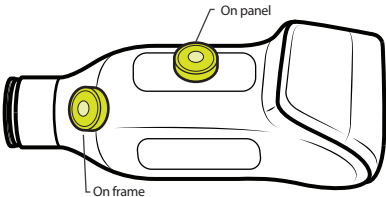- 3 Determine lace routing

For Inline Routing - Use port II and III

For Remote Routing - Use port I and II


Inline - mounted on panel

Inline - mounted on socket frame


Remote - mounted on socket frame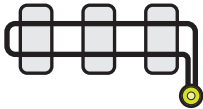

Fabrication Overview:


Glue on RevoFit Components

1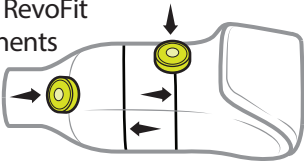


Wrap Casting Tape

2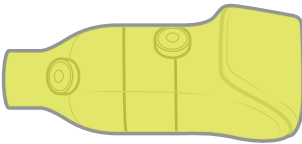


Cut Out Windows

3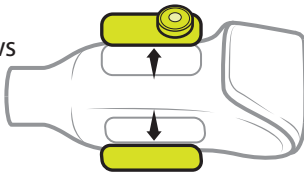


Install + Shape Pads

4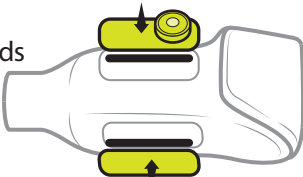


Lace Socket

5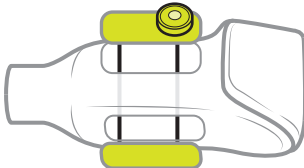


Install Reel + Tune Pads

6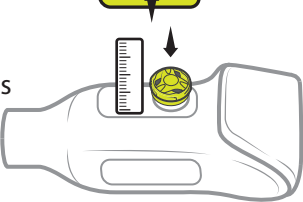


Diagnostic Instructions:

1 Pull Plastic.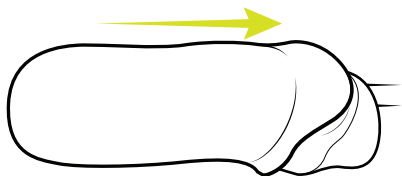



- 2 a. Roughly sand socket.
- b. Draw in trim lines.
- c. Determine reel, panel and lace path location.
- d. Roughly draw in lace path.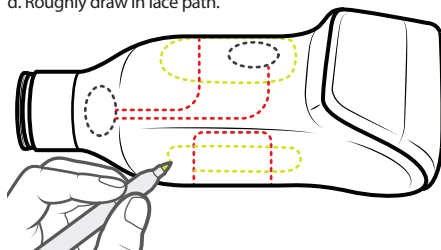


3 Draw tube guide lines:

- a. Ensure that lace path crosses all panels with the $\frac{1}{4}$ spacing rule.
- b. Ensure tubing crosses panels perpendicularly.
- c. Don't turn tubing until 1 cm past any panel edge.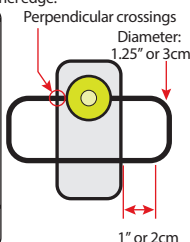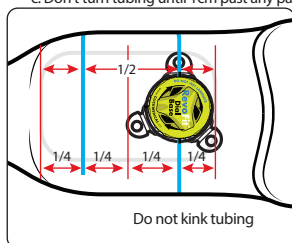


4 Reel mounting:

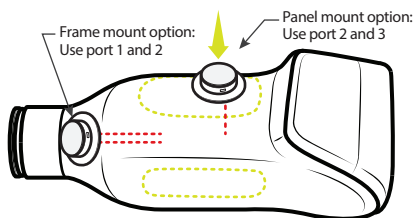
Note: Indicate position of lace path

5 Peel backing paper and stick to check socket.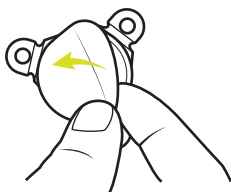


6 Glue on tubing:

- a. Glue on tubing according to lace pattern.
- b. Cut second end of tubing to length, plug and insert second end into dial base.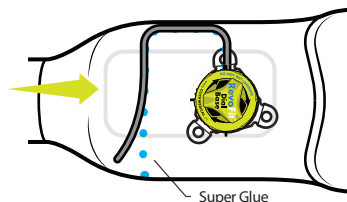


NOTE: Take picture of socket for reference use in step 8.

7 Wrap all parts of system in casting tape.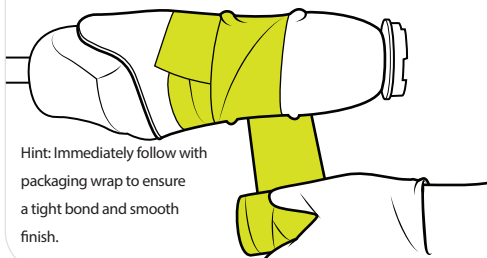

Hint: Immediately follow with packaging wrap to ensure a tight bond and smooth finish.


8 Mark with a pen the location of the panels to cut out: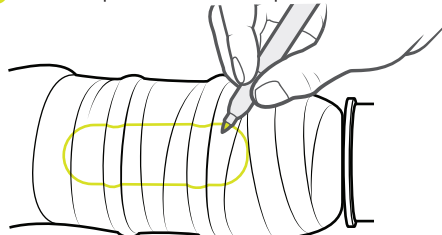


HINT: Use photo from step 6 above.

Diagnostic Instructions Continued:


9 Cut out the panels: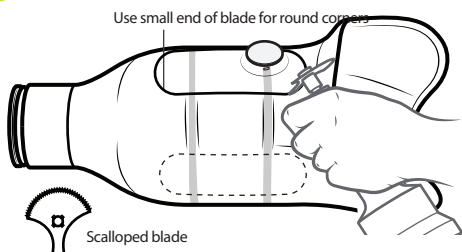


Sand edges

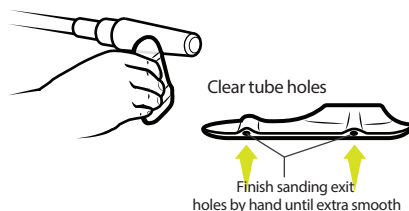10 Prepare housing:

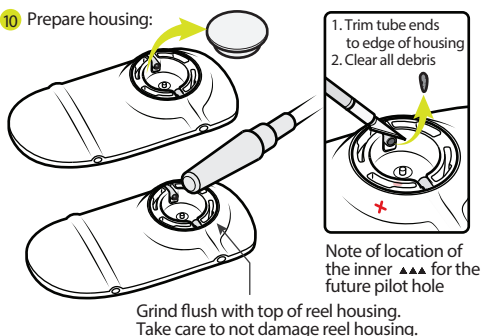11 Drill pilot hole to access tab for reel removal: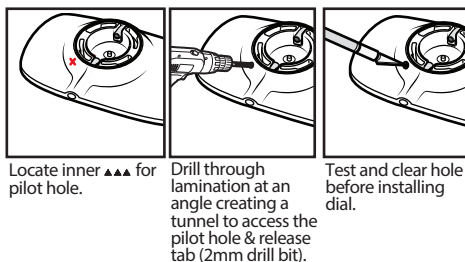


12 Fabricate and install pads:


Pad thickness, material and shape are used to refine pressure and fit.

Pad Material: Pads should be dense enough to create pressure and not compress under tension.


Pad Thickness: Average 3/16" (4.75mm) Push shore: 35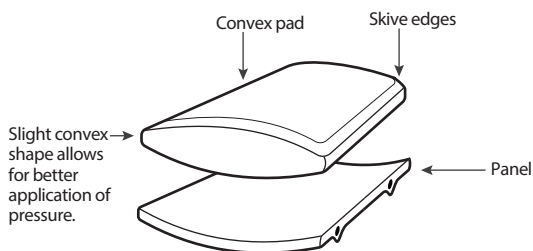


INCORRECT - Pad too thin when panel is flush with socket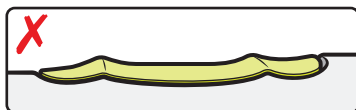


Weak Closure Force

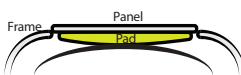CORRECT - Pad is the correct thickness when panel sits above socket when fit is snug to patient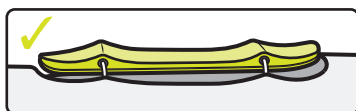


Good Closure Force

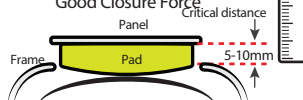
Diagnostic Instructions Continued:

13 Feed lace: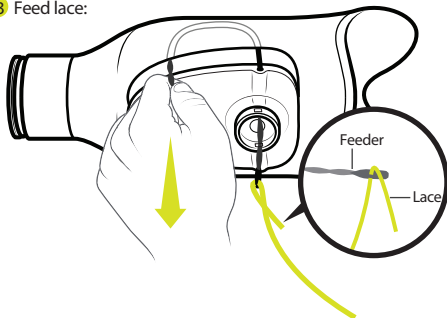


14 Attach lace to reel:

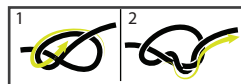
Insert laces into smallest end of pass through on spool. Tie double overhand knots on each end.

Leave 8cm of lace on both ends

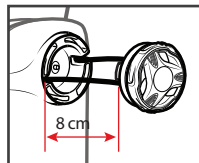
15 Insert reel into socket - turn counter-clockwise to lock: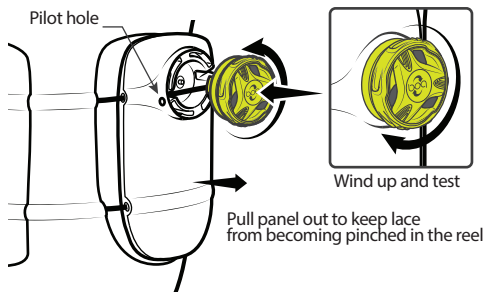


✓ Finished!

- Fit patient
- Tune in fit, by refining pad shape... if necessary.
- Provide IFU's and replacement lace pack to patient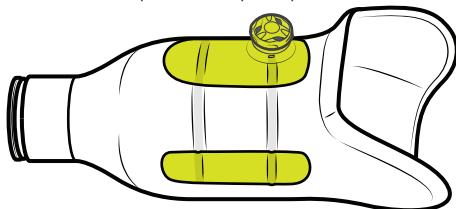


How to remove dial:

- Open dial
- Use tool to press inward on tab
- Simultaneously turn housing clockwise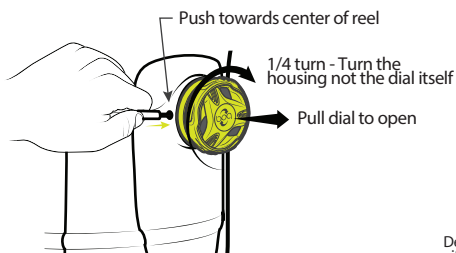


Tab location:

If you inserted the reel PRIOR to drilling an access hole use vise grip instructions below. Once dial is removed, drill pilot hole.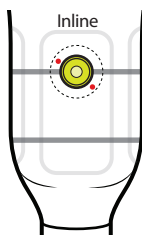


Depending on the location, either at 4 o'clock or 11 o'clock.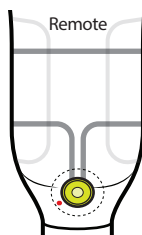


Always at 8:30.

Locate tab (see above):

Lock on to housing of dial with needle nose vise grips at tab location.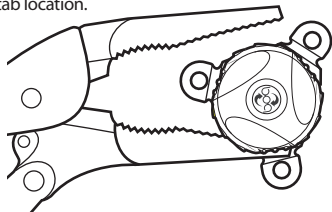

Rotate clockwise to remove.

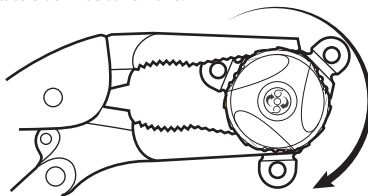CLICK

1205 Hilltop Parkway
Warehouse 101
Steamboat Springs CO, 80487

970-670-7012 • ClickMedical.co