
RevoFit2™ Lamination Kit

PK2000-320-05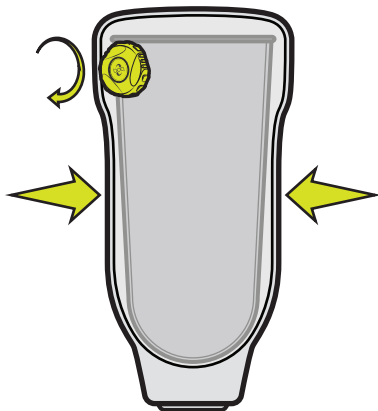

Advisory:

Do not use on patients with neuropathy, neuromas, or other nerve issues.
Fabrication only by a certified professional, trained in the RevoFit System.
Filament will wear, inspect and replace regularly.

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
WELCOME TO
RevoFitTM

Thank you for
being a champion
of **adjustability!**


Your expertise gives
RevoFit technology
the **power to**
change lives.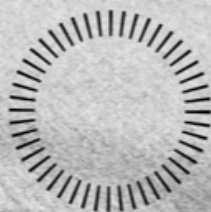


CLICK

RevoFit2™ Lamination Kit Contents:

6ft Teflon Tubing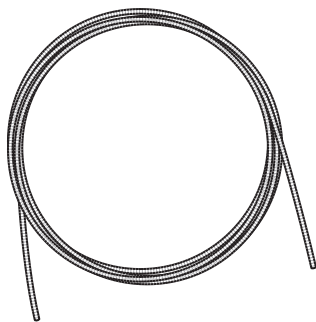

7ft Lace

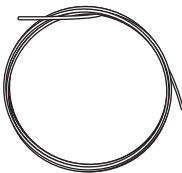
Lace Tool

RevoFit2 Lamination Dummy

High Power Boa Dial

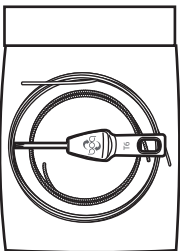Patient Replacement Lace Pack

Recommended Designs:

Below Knee:
3 Panel Design
or Symes Door

Above Knee:
2 System with
Medial/Lateral Panels +
Adjustable Strap


+Additional Kit Required


Recommended
Suspension System:
RevoLock™ Lanyard Kit
PN:3000-220-05

Design Overview:

- 1 Determine location of panels or areas of adjustment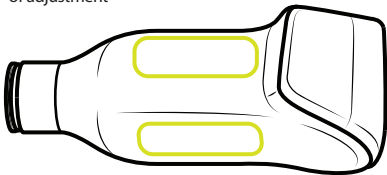


- 2 Determine location of reel

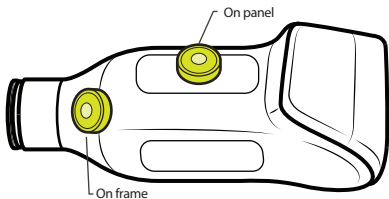
- 3 Determine lace routing

For Inline Routing - Use port II and III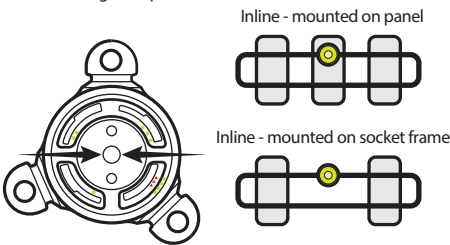


For Inline Routing - Use port I and II

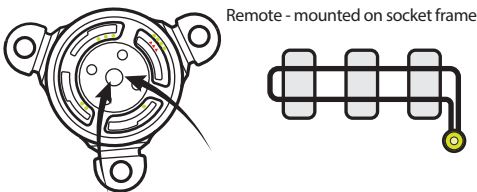Fabrication Overview:

Glue on RevoFit Components

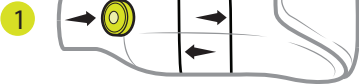Laminate

Cut Out Windows

Install + Shape Pads

Lace Socket


Install Reel + Tune Pads


Lamination Instructions:



- 1 Prep the mold with a thin first lamination layer:
Reinforce areas under the tubing and under the panel cutout areas.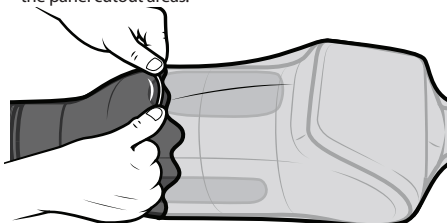


- 2 a. Roughly sand socket.
b. Draw in trim lines.
c. Determine reel, panel and lace path location.
d. Roughly draw in lace path.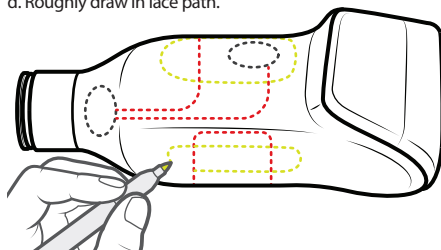

- 3 Draw tube guide lines:
a. Ensure that lace path crosses all panels with the 1/4 spacing rule.
b. Ensure tubing crosses panels perpendicularly.
c. Don't turn tubing until 1cm past any panel edge.

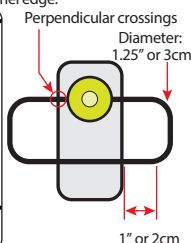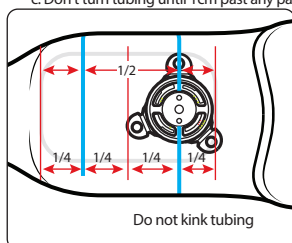
- 4 Reel mounting:
If mounting reel on frame, glue it into place now.
If mounting reel on panel refer to Step 3 before gluing.
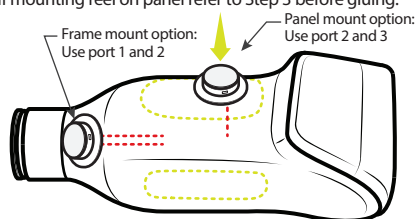


Note: Indicate position of lace path

- 5 Trim one tube, pack with clay, insert into bayonet:
c. Insert Tube
Make sure tubing is inserted 2cm into silicone.

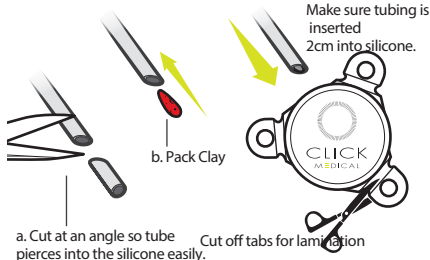
- 6 Glue on tubing:
a. Glue on tubing according to lace pattern.
b. Cut second end of tubing to length, plug and insert second end into bayonet.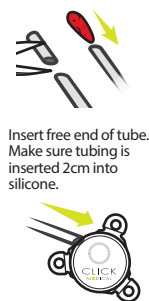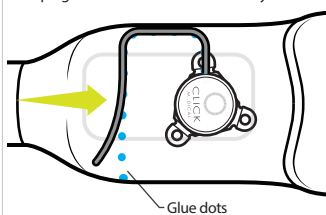


NOTE: Take picture of socket for reference after second lamination.

- 7 Laminate again:
Do not displace tubing or reel housing while laminating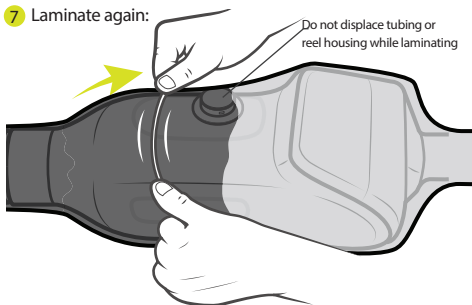


- 8 Mark with a pen the location of the panels to cut out: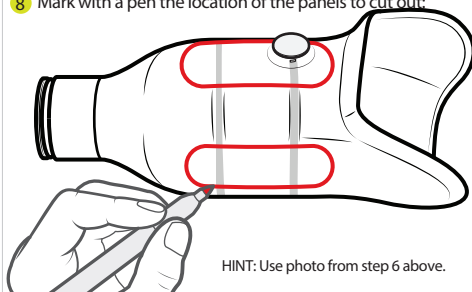


HINT: Use photo from step 6 above.

Lamination Instructions Continued:


9 Cut out the panels: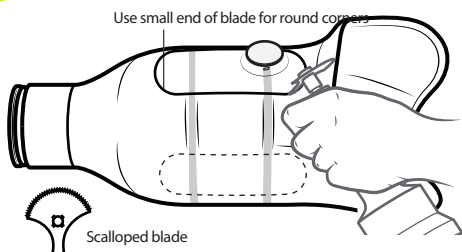


Sand edges

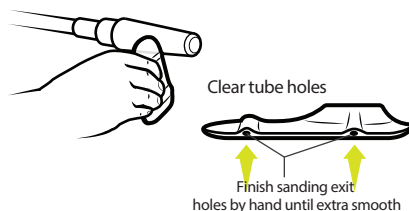10 Prepare housing:

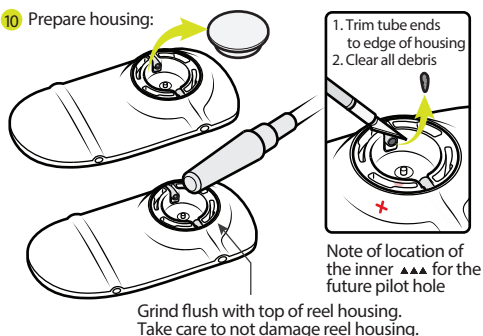11 Drill pilot hole to access tab for reel removal: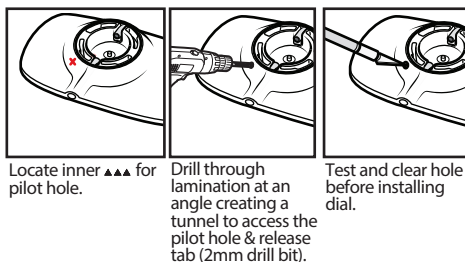


12 Fabricate and install pads:


Pad thickness, material and shape are used to refine pressure and fit.

Pad Material: Pads should be dense enough to create pressure and not compress under tension.


Pad Thickness: Average 3/16" (4.75mm) Push shore: 35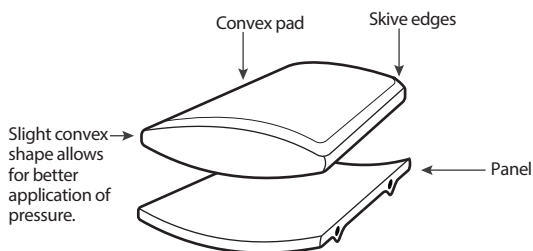


INCORRECT - Pad too thin when panel is flush with socket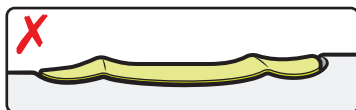


Weak Closure Force

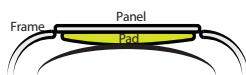CORRECT - Pad is the correct thickness when panel sits above socket when fit is snug to patient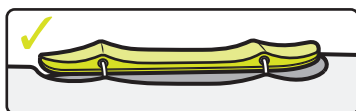


Good Closure Force

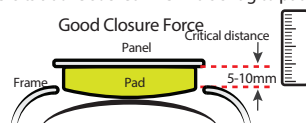
Lamination Instructions Continued:


13 Feed lace: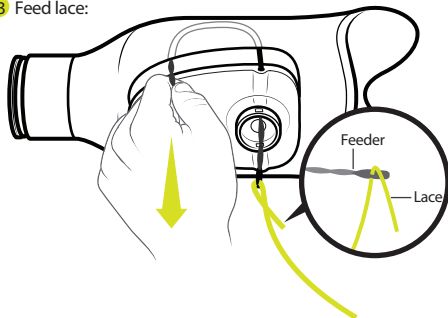

14 Attach lace to reel:

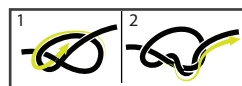
Leave 8cm of lace on both ends

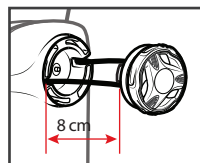
Insert laces into smallest end of pass through on spool. Tie double overhand knots on each end.

15 Insert reel into socket - turn counter-clockwise to lock: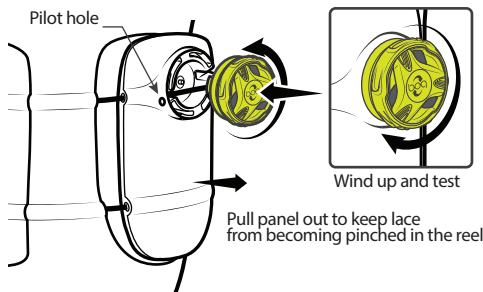


✓ Finished!

- Fit patient
- Tune in fit, by refining pad shape... if necessary.
- Provide IFU's and replacement lace pack to patient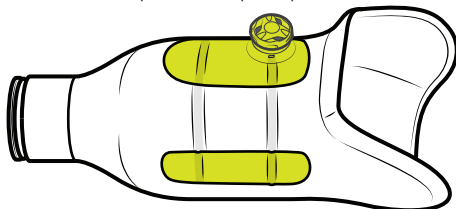


How to remove dial:

- Open dial
- Use tool to press inward on tab
- Simultaneously turn housing clockwise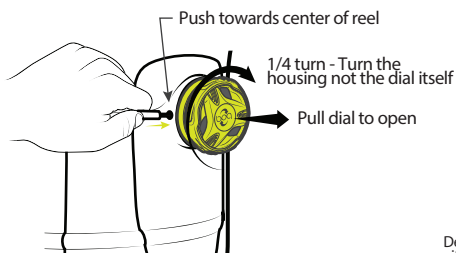


Tab location:

If you inserted the reel PRIOR to drilling an access hole use vise grip instructions below. Once dial is removed, drill pilot hole.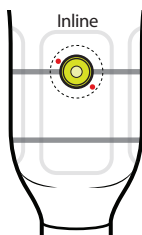


Depending on the location, either at 4 o'clock or 11 o'clock.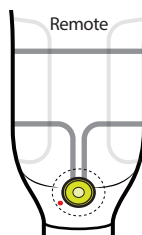


Always at 8:30.

Locate tab (see above)

Lock on to housing of dial with needle nose vise grips.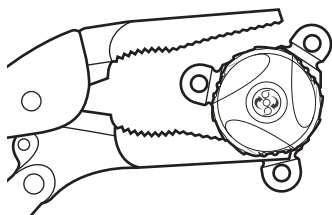

Rotate clockwise to remove

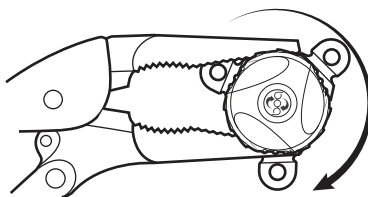CLICK

1205 Hilltop Parkway
Warehouse 101
Steamboat Springs CO, 80487

970-670-7012 • ClickMedical.co