

Freedom Kinterra

The Kinterra Foot / Ankle System combines hydraulics and carbon fibre technology to provide low to moderate impact K3 ambulators a much improved walking gait, even when negotiating slopes and undulating terrain.

The hydraulic ankle has 2° of dorsiflexion against a sprung bumper and 10° of planterflexion and subsequent dorsiflexion. The rate of yield of these can be adjusted individually to achieve the most effective and comfortable gait for the user when descending a slope and the carbon fibre keel provides a smoother transition when ascending a slope. The Dorsi-Assist spring improves ground clearance and eases the transition from a downhill ramp to level ground, with a reduced risk of catching the toe.

Provides the ideal combination when used with the Plié 3 microprocessor knee (see page 98).

CES FRE F05: Use this code to read the Kinterra clinical evaluation summary on the Steeper website.

Features and Benefits

- Articulating ankle delivers reduced socket pressures for healthier residual limbs
- Increased ground contact and a more symmetrical gait
- Improved stability and safety, especially on slopes and varied terrain
- Dorsi-Assist spring aids ground clearance in swing phase
- Individual adjustment of dorsi and plantar flexion
- Safe for occasional contact with water, but not intentional, frequent contact or submersion. After contact with salt or chlorinated water, feet should be rinsed/cleaned.

Cosmetics



Mid profile foot shell Sandal toe: sizes 22-28 only



Ordering Information

Trial Period - This foot has a 30 day trial period - only on sizes 23-30 and categories 2-7.

Impact Level Descriptions

Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

Ordering Example - NROM526L

Part Number	Category	Size	Left / Right
NROM	5	26	L

Sandal Toe Ordering Example - NROM526SL

Part Number	Category	Size	Sandal Toe	Left / Right
NROM	5	26	S	L


Includes foot shell and Spectra® Sock

Category Selection

Weight (kg)	Impact Level	
	LOW	MOD
44-52	1	1
53-59	1	2
60-68	2	3
69-77	3	4
78-88	4	5
89-100	5	6
101-116	6	7
117-125	7	

Bumper Kit	Size (cm)
NKIT-00-11333-00	22-25
NKIT-00-11334-00	26-28
NKIT-00-11335-00	29-31

Technical Information

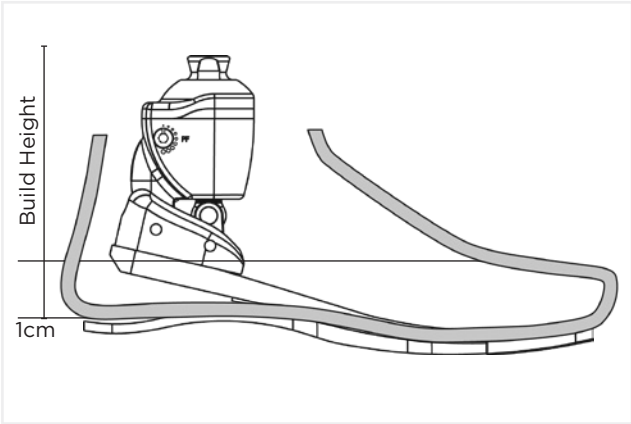

Build Height and Weight

Size (cm)	Build Height (cm)	Foot inc. Shell
22-25	12	795g
26-28	12.1	897g
29-31	12.4	992g

Product Specification

Weight Limit	Impact Level	Sizes (cm)	Warranty
125kg	L M H	22-31	3 years Foot shell - 6 months

Spare Parts Ordering Information

Part Number	Foot Shell Type
NFTC-3M-0▲●4-R■	Standard, with CAP
NFTC-3M-1▲●4-R■	Standard, without CAP
NFTC-2H-1▲●4-R■	High-profile, without CAP
NFTC-2M-1▲●4-S■	Sandal Toe, without CAP

Spare Parts Ordering Guide
 ▲ = Size in cm (22-31 = Regular, 22-28 = Sandal Toe)
 ● = Skin Tone (L = Light, M = Medium, D = Dark)
 ■ = "L" or "R" for Left or Right

Spectra® Sock	Size (cm)	Sock Size (cm)
NSO-NPS-X0030-00	22-25	30
NSO-NPS-X0035-00	26-28	35
NSO-NPS-X0040-00	29-31	40

X denotes colour. Insert 1 for white, or 2 for black. Note: Default colour is black