

Freestyle Swim™

The Freestyle Swim foot enables the user to go from walking to swimming. A single adjustment moves the ankle from a 10mm heel height setting for walking, to 70° plantar flexion for swimming, by means of a push button.

The ankle is used in conjunction with the LP Symes prosthetic foot keel and comes complete with a cosmetic foot shell to protect the system and a protective rubber cap that is snapped over the ankle for further protection for the ankle unit.

Features and Benefits

- Users can walk comfortably to and from the swim area
- Adjust to 70° plantar flexion for swimming with the simple push of a button
- Utilise a flipper to gain swimming speed and decrease energy expenditure
- The extended foot position balances the swimming action and increases efficiency



Waterproof - designed for swimming and frequent use in water

Cosmetics



Foot shell



Ordering Information

Trial Period - There is no trial period on this foot.

Impact Level Descriptions

Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Note: Although the 'high' impact level states 'running' as an appropriate activity, the Freestyle Swim Foot is an activity-specific foot and is not recommended for running.

Ordering Example - NLP2W2322L

Part Number	Category	Size	Left / Right
NLP2W2	3	22	L


Includes foot shell and Spectra® Sock

Category Selection

Weight (kg)	Impact Level		
	LOW	MOD	HIGH
44-52	1	1	2
53-59	1	2	3
60-68	2	3	4
69-77	3	4	5
78-88	4	5	6
89-100	5	6	
101-116	6		

Bumper Kit	Size (cm)
NKIT-00-11100-00	22-25
NKIT-00-11300-00	26-28
NKIT-00-11500-00	29-31

Technical Information

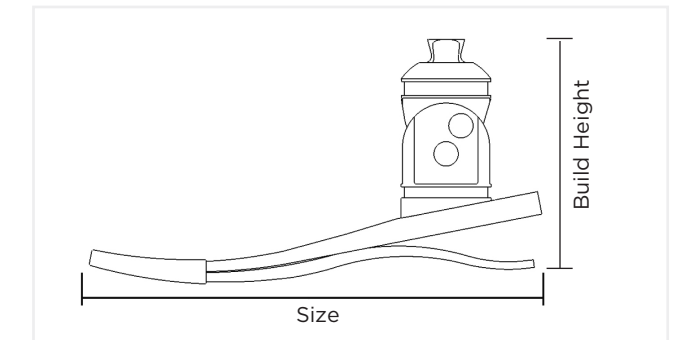

Build Height and Weight		
Size (cm)	Build Height (cm)	Foot inc. Shell
22-25	10.2	680g
26-28	10.7	790g
29-31	11.7	945g

Product Specification

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
116kg	L M H	680g (Size 25)	22-31	10.2 (Size 25)	1 year Foot shell - 6 months

*Water note: Rinse with fresh water and dry following exposure to water, particularly salt or chlorinated.

Spare Parts Ordering Information

Part Number	Foot Shell Type
NFTC-3M-1▲●4-R■	Standard, without CAP

Protective Cover Repair Kit

NKIT-00-20000-00

Spare Parts Ordering Guide

- ▲ = Size in cm (22-31)
- = Skin Tone (L = Light, M = Medium, D = Dark)
- = "L" or "R" for Left or Right

Spectra® Sock	Size (cm)	Sock Size (cm)
NSO-NPS-X0025-00	22-23	25
NSO-NPS-X0030-00	24-27	30
NSO-NPS-X0035-00	28-31	35

X denotes colour. Insert 1 for white, or 2 for black. **Note:** Default colour is black