

KIT-00-11303-00

Highlander® Heel Stiffening Bumper Kit, Sizes 22-25cm

For optional use.



Test bumper placement:

Temporarily attach at recommended location using pre-applied adhesive. Suggested placement will stiffen the heel by approximately one category. If necessary, adjust anterior/posterior placement to achieve desired stiffness.

Permanently bond:

Lightly abrade carbon fiber with sandpaper, then clean with acetone. Remove temporary adhesive on bumpers using a cloth with acetone. Do not soak bumpers in acetone. Bond bumpers in pre-selected position using cyanoacrylate (Super Glue).

Global Headquarters

3 Morgan
Irvine, CA 92618
U.S.A.
toll-free 888.818.6777
tel 949.672.0032
fax 949.672.0084
www.freedom-innovations.com

European Head Office

Jaargetijdenweg 4
7532 SX Enschede
The Netherlands
toll-free 00800-2806 2806
tel +31 (0)53-20 30 300
fax +31 (0)53-20 30 305
www.freedom-innovations.eu



R-720-200 Rev. 10/14



KIT-00-11304-00

Highlander® Heel Stiffening Bumper Kit, Sizes 26-28cm

For optional use.



Test bumper placement:

Temporarily attach at recommended location using pre-applied adhesive. Suggested placement will stiffen the heel by approximately one category. If necessary, adjust anterior/posterior placement to achieve desired stiffness.

Permanently bond:

Lightly abrade carbon fiber with sandpaper, then clean with acetone. Remove temporary adhesive on bumpers using a cloth with acetone. Do not soak bumpers in acetone. Bond bumpers in pre-selected position using cyanoacrylate (Super Glue).

Global Headquarters

3 Morgan
Irvine, CA 92618
U.S.A.
toll-free 888.818.6777
tel 949.672.0032
fax 949.672.0084
www.freedom-innovations.com

European Head Office

Jaargetijdenweg 4
7532 SX Enschede
The Netherlands
toll-free 00800-2806 2806
tel +31 (0)53-20 30 300
fax +31 (0)53-20 30 305
www.freedom-innovations.eu




R-720-201 Rev. 10/14



KIT-00-11305-00

Highlander® Heel Stiffening Bumper Kit, Sizes 29-31cm

For optional use.

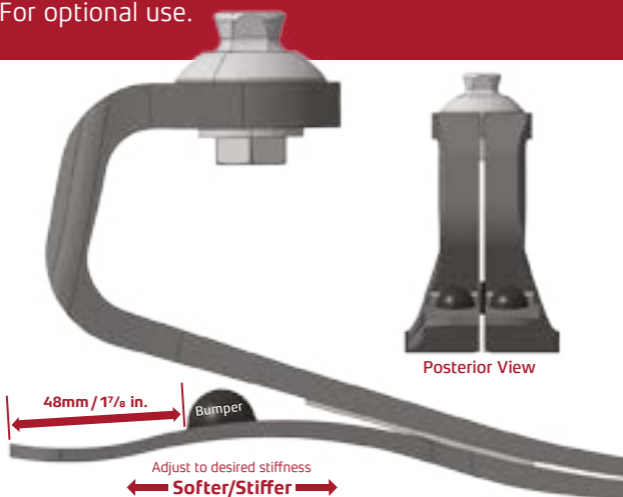

Test bumper placement:

Temporarily attach at recommended location using pre-applied adhesive. Suggested placement will stiffen the heel by approximately one category. If necessary, adjust anterior/posterior placement to achieve desired stiffness.

Permanently bond:

Lightly abrade carbon fiber with sandpaper, then clean with acetone. Remove temporary adhesive on bumpers using a cloth with acetone. Do not soak bumpers in acetone. Bond bumpers in pre-selected position using cyanoacrylate (Super Glue).

Global Headquarters

3 Morgan
Irvine, CA 92618
U.S.A.
toll-free 888.818.6777
tel 949.672.0032
fax 949.672.0084
www.freedom-innovations.com

European Head Office

Jaargetijdenweg 4
7532 SX Enschede
The Netherlands
toll-free 00800-2806 2806
tel +31 (0)53-20 30 300
fax +31 (0)53-20 30 305
www.freedom-innovations.eu



R-720-202 Rev. 10/14

