


# Slalom Ski Foot Instructions for Use

Product Number: SKI

- Assembly** The Slalom Ski Foot module is pre-assembled consisting of a graphite component (keel) and soleplate. After dynamic alignment, secure pyramid adjustment screws with a thread locking adhesive (i.e., Loctite T242). Torque the pyramid adjustment screws to the manufacturer's specifications.
- Dynamic Alignment** The Slalom Ski Foot should be aligned so the user is comfortable in his/her ski stance position. The pyramid of the foot is angled anteriorly 8°. The foot may require plantarflexion or dorsiflexion to accommodate the user's preferred stance position. The position of the contralateral ski boot (if applicable) provides a reference.
- Use** The Slalom Ski Foot is designed to insert directly into the ski binding, eliminating the need for a ski boot. The sole material protects the composite components during limited ambulation.

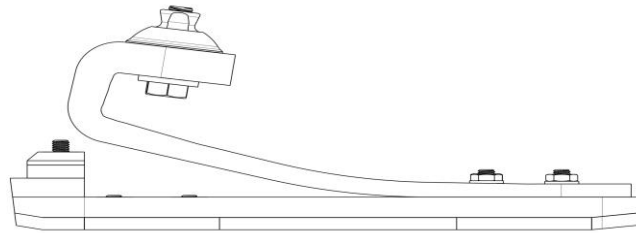

- Slalom Ski Foot** Minimum clearance: ~140mm average  
Maximum user weight: 113 kg (250 lbs)  
Available size: Custom made to the size of the ski binding  
Warranty: Graphite components/pyramid connector (12 months)

- Maintenance** The foot module requires periodic maintenance.
- Inspect the foot module every six months. If the user is more active, more frequent inspection may be necessary. Service as necessary.
  - Replace sole material as necessary to prevent damage to the composite components.
  - The foot module may be cleaned and/or disinfected with soap and warm water.

- Warnings** Failure to adhere to the guidelines of the *Instructions for Use* will void the warranty.
- The Slalom Ski Foot is intended for alpine skiing exclusively. Use for other activities may cause injury and will void the warranty.
  - The Slalom Ski Foot is not intended to be used for jumping on skis of any kind. Jumping may cause injury and will void the warranty.
  - Freedom Innovations foot modules are manufactured to fit industry standard pyramids and receivers. It is the prosthetist's responsibility to select and/or fabricate properly fitting attachment components.
  - Never attempt to loosen the bolt affixing the pyramid connector.
  - The user should discontinue use and consult his/her prosthetist if any part of the prosthesis starts to make noise.
  - The user should inform his/her prosthetist of significantly body weight changes.
  - Freedom Innovations foot products are manufactured and tested for a particular weight and activity impact level. Use by another user for whom it was not originally manufactured may cause injury and shall void any written or implied warranty.



3 Morgan  
Irvine, California 92618  
USA  
Toll Free (888) 818-6777  
FAX (949) 672-0084



Advena Ltd. Pure Offices, Plato  
Close, Warwick, CV34 6WE UK