

Maverick Xtreme  
Maverick Xtreme AT



## Advancing Fibreglass Technology



Freedom Innovations' Maverick fibreglass feet will exceed the highest demands of users that revel in extreme activities over all terrains, taking them wherever they want to go for whatever they want to do.

Made with a lightweight, aerospace graded fibreglass composite combined with an innovative keel design and "Wave" heel plate, Maverick feet offer durability and flexibility without compromising on product weight or energy storage and return.

Supplied in both sandal toe and regular toe, Maverick feet are designed to accommodate a range of lifestyles and activities including those in water.

### Maverick Xtreme

A solid keel version that provides extreme durability and flexibility. The full length, unbolted sole plate ensures seamless stance phase roll-over.



### Maverick Xtreme AT

Benefiting from all the features of the Xtreme but with a  $\pm 16^\circ$  inversion/eversion split keel design providing users with excellent stability on uneven terrains.



## Maverick Xtreme and Maverick Xtreme AT

### Features and Benefits

#### Extreme flexibility

Superb durability and flexibility through inherent characteristics of aerospace grade fibreglass.



#### Designed for active users

Ideal for highly active users who require an alternative to carbon fibre, for use in extreme activities.



#### Waterproof

Designed to be water resistant, safe for contact with water and submersion, enabling users to enjoy water activities.



#### Smooth roll-over

A full length, unbolted sole plate ensure seamless stance phase roll-over.



#### $\pm 16^\circ$ inversion/eversion

Split keel/heel design of the Maverick Xtreme AT provides excellent inversion and eversion for patient stability on uneven terrains.



#### Lightweight

Manufactured in a lightweight fibreglass composite, the Maverick feet are lighter than competitive products, even with the foot shell and sock included.



#### Sandal toe

To accommodate the various lifestyles and activities of users, both Maverick feet are available in sandal toe.



#### "Wave" heel plate

The innovative design allows for more energy to be stored and returned.





## Maverick Xtreme and Maverick Xtreme AT

### Technical Specifications

#### Sizes

23 - 30cm (sandal toe sizes: 23 - 28 cm)

#### Build height

17cm (size 28, category 4)

#### Product weight

821g (Maverick Xtreme, size 28, category 4, including foot shell)

798g (Maverick Xtreme AT, size 28, category 4, including foot shell)

#### User weight limit

Up to 166kg

#### Toe option

Regular and sandal toe

#### Split keel

Maverick Xtreme AT

#### Foot shell skin tones

Light, Medium and Dark

#### Warranty

36 months (foot shell 6 months)

#### Target user

High impact extreme (K3 and K4) level users who require an alternative to carbon fibre for use in extreme activities. Maverick Xtreme and Maverick Xtreme AT is suitable for transtibial, transfemoral, unilateral and bilateral amputees

## Maverick Xtreme and Maverick Xtreme AT

### Ordering Information

For more information on the Maverick Xtreme and to place an order please contact Customer Services on +44 (0) 870 240 4133 or via e-mail on [customerservices@steepergroup.com](mailto:customerservices@steepergroup.com)

Category Selection				Impact Level Descriptions
Weight (kg)	Impact Level			
		LOW	MOD	HIGH
44-52	1	1	2	<b>Low:</b> Daily activities include mostly level ground walking, moving around in the home and in the community.
53-59	1	2	3	
60-68	2	3	4	
69-77	3	4	5	<b>Moderate:</b> Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).
78-88	4	5	6	
89-100	5	6	7	
101-116	6	7	8	<b>High:</b> Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.
117-130	7	8	9	
131-147	8	9		
148-166	9			

Part Number	Description	Available Sizes (cm)
NF11	Maverick Xtreme	23-30 cm
NF21	Maverick Xtreme AT	23-30 cm

#### Ordering Guide

NF11 ◆ ▲ ■ ◆

◆ = Category

▲ = Size in cm (23-30)

■ = Toe (R = Reg, S = Sandal)

◆ = "L" or "R" for Left or Right

#### Maverick Xtreme Sandal Toe Ordering Example - NF11526SL

Part Number	Category	Size	Sandal Toe	Left / Right
NF11	5	26	S	L

Note: this foot comes with a 30 day trial period (sizes 23-28 and categories 2-7 only).

Steeper Group, Unit 3 Stourton Link, Intermezzo Drive, Leeds, LS10 1DF

Tel: +44 (0) 870 240 4133

Email: [customerservices@steepergroup.com](mailto:customerservices@steepergroup.com)

[www.steepergroup.com](http://www.steepergroup.com)

STPPR044