

# Seattle Energy

The full-length, split composite keel in the Seattle Energy provides increased dynamic response to the user, improved medial/lateral compliance, additional stability and a cushioned heel.

With its removable pyramid and spacer, the Seattle Energy is very effective for active patients with Symes amputations.

## Features and Benefits

- Dynamic toe and cushioned heel response
- Full length, split composite keel for increased stability
- Removable, lifelike, sandal toe foot shell
- Pre-assembled with male pyramid for easy installation
- Pyramid and spacer can be removed for Symes patient applications
- Available in both Light and Dark foot shells



Foot construction with foot shell removed

## Cosmetics



Light

Dark



## Ordering Information

Trial Period - There is no trial period on this foot.

### Impact Level Descriptions

**Low:** Low speed walking and low intensity activities - no variable cadence.

**Low-Medium:** Low impact activities and walking on uneven surfaces.

**Medium:** Medium impact activities including light sports.

Part Number	Description	Available Sizes (cm)
NSEF190	9.5mm Heel, Light	22-30 cm
NSEF193	9.5mm Heel, Dark	22-30 cm

### Ordering Example - NSEF190-25L-2

Part Number	Size	Left / Right	Category
NSEF190	25	L	2

Includes foot shell and Spectra® Sock

## Category Selection

To optimise the selection and ensure amputee's safety, follow the steps below to determine the appropriate keel.

1. Locate the column that corresponds with the amputee's impact level.
2. Within the selected column, locate the amputee's weight.
3. Place the relevant category number into the part number for ordering.

**Note:** If the amputee has a long BK, carries heavy loads or will reach a higher impact level within a year, choose the next category higher.

Category	Low	Low - Medium	Medium
4	101-136kg	101-136kg	101-136kg
3	81-100kg	81-100kg	81-100kg
2	61-80kg	61-80kg	61-80kg
1	< 60kg	< 60kg	< 60kg

# Feet - Trulife - Seattle Energy

## Product Specification

Build Height and Heel Rise			
Part Number	Size (cm)	Build Height (cm)	Heel Rise (mm)
NSEF190	22-30	10	9.5
NSEF193	22-30	10	9.5

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
136kg	L M H	330g Size 25 without foot shell	22-30	10	2 years Foot shell - 6 months

## Additional Parts Ordering Information

Symes Nut			
Part Number	Weight Limit	Weight	
NSSY300	136kg	42g	

## Spare Parts Ordering Information

Replacement Foot Shell, Light		
Part Number	Size (cm)	
NSFC190	22-30	

Replacement Foot Shell, Dark		
Part Number	Size (cm)	
NSFC193	22-30	

Foot Shell Ordering Example - NSFC190-22L		
Part Number	Size	Left / Right
NSFC190	22	L