


## Seattle Lightfoot with Pyramid

Supplied complete with a removable pyramid and spacer and ready to install, this natural looking foot has a low profile build height option and 4 keel strengths, continuing to be suitable for a wide range of applications.

With the pyramid and spacer removed, the low profile build height option makes the Seattle Lightfoot an ideal foot for a user with a Symes amputation. Fundamentally, it is the same as the very effective Lightfoot2 that it replaces.

### Features and Benefits

- Low profile selectable keel
- Ideal for the longer residuum
- Appropriate for low, medium and high activity levels
- Natural appearance
- Sandal toe
- Available in both Light and Dark foot shells

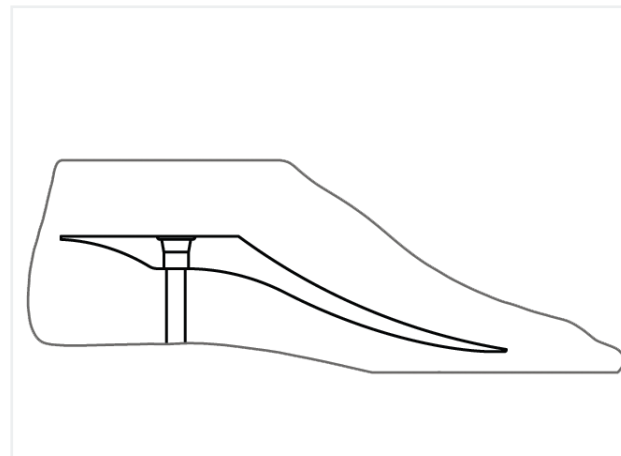

Seattle Lightfoot keel view

### Cosmetics



Light

Dark



### Ordering Information

Trial Period - There is no trial period on this foot.

#### Impact Level Descriptions

**Low:** Low speed walking and low intensity activities - no variable cadence.

**Low-Medium:** Low impact activities and walking on uneven surfaces.

**Medium:** Medium impact activities including light sports.

Part Number	Description	Available Sizes (cm)
NSLF195	9.5mm Heel, Light	22-30
NSLF198	9.5mm Heel, Dark	22-30

#### Ordering Example - NSLF195-25L-H6 or NSLF198-25L-H6

Part Number	Size	Left / Right	Category
NSLF195	25	L	H6
NSLF198	25	L	H6

### Category Selection

To optimise the selection and ensure amputee's safety, follow the steps below to determine the appropriate keel.

1. Locate the column that corresponds with the amputee's impact level.
2. Within the selected column, locate the amputee's weight.
3. Place the relevant H number into the part number for ordering.

**Note:** If the amputee has a long BK, carries heavy loads or will reach a higher impact level within a year, choose the next category higher.

Category Selection			
	Low	Low - Medium	Medium
H9	-	116-136kg	114-136kg
H8	-	87-115kg	91-113kg
H7	123-136kg	75-86kg	69-90kg
H6	82-122kg	59-74kg	45-68kg
H5	< 82kg	< 59kg	< 45kg

### Product Specification

Build Height and Heel Rise			
Part Number	Size (cm)	Build Height (cm)	Heel Rise (mm)
NSLF195	22-30	9	9.5
NSLF198	22-30	9	9.5

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
136kg	L M H	480g Size 25	22-30	9 Size 25	2 years Foot shell - 1 year

### Additional Parts Ordering Information

Symes Nut			
Part Number	Weight Limit	Weight	
NSSY300	136kg	42g	

