

# Seattle Triumph

The foot range that was developed from the S-shaped carbon fibre system, has been brought together into one new option, the Seattle Triumph. It is ideal for medium to high activity users, since the S-shaped design provides the foot with shock absorption, torsional rotation and controlled, but fast plantar flexion, making it particularly good for use on transfemoral prostheses.

The energy stored gives a smooth transition to foot-flat which is stable, thanks to the split keel and, with a good dynamic response and the transition through to toe-off is equally smooth. It is also surprisingly light for a foot that offers such a high level of function.



## Features and Benefits

- S-spring stores energy at heel strike, released into controlled and fast plantar flexion and smooth transition to foot-flat
- Vertical shock absorption
- Torsional rotation of approx 20 degrees
- Split keel for compliance and stability
- Lightweight
- Solid toe

## Cosmetics



Light

Dark

## Ordering Information

Trial Period - This foot has a 60 day trial period.

### Impact Level Descriptions

**Medium:** Medium impact activities including light sports.

**High:** High impact activities such as running and basketball.

Part Number	Description
NSTF190	Seattle Triumph Foot, Light
NSTF193	Seattle Triumph Foot, Dark

### Ordering Example - NSTF190-22L-4

Part Number	Size	Left / Right	Category
NSTF190	22	L	4

Includes foot shell and Spectra® Sock

## Category Selection

To optimise the selection and ensure amputee's safety, follow the steps below to determine the appropriate keel.

1. Locate the column that corresponds with the amputee's impact level.
2. Within the selected column, locate the amputee's weight.
3. Place the relevant number into the part number for ordering.

**Note:** If the amputee has a long BK, carries heavy loads or will reach a higher impact level within a year, choose the next category higher.

Category Selection			
	Medium	Medium-High	High
9	141-160kg	124-145kg	106-125kg
8	121-140kg	106-123kg	91-105kg
7	101-120kg	89-105kg	76-90kg
6	91-100kg	80-88kg	69-75kg
5	81-90kg	71-79kg	61-68kg
4	71-80kg	62-70kg	53-60kg
3	66-70kg	58-61kg	50-53kg
2	60-65kg	53-57kg	46-49kg
1	< 60kg	< 53kg	< 46kg

# Feet - Trulife - Seattle Triumph

## Product Specification

Build Height and Heel Rise			
Part Number	Size (cm)	Build Height (cm)	Heel Rise (mm)
NSTF190	22-30	18	9.5
NSTF193	22-30	18	9.5

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
160kg	M H	493g Size 25 (without foot shell)	22-30	18 Size 25	3 years Foot shell - 6 months

## Spare Parts Ordering Information

Replacement Foot Shell, Light		
Part Number	Size (cm)	
NSFC290	22-30	

Replacement Foot Shell, Dark		
Part Number	Size (cm)	
NSFC293	22-30	

Foot Shell Ordering Example - NSFC290-22L			
Part Number	Size	Left / Right	
NSFC290	22	L	