

Freedom Sierra®



Ideal for active users, the Sierra foot ensures a smoother gait without the energy loss typically associated with other shock absorbing systems. Its unique angle-top design increases the length of the carbon fibre spring, which maximises energy return and flexibility. The split-keel provides excellent inversion/eversion, allowing amputees to walk with confidence on uneven terrain.

CES FRE F02: Use this code to read the Sierra clinical evaluation summary on the Steeper website.




***Water-resistant:** Safe for contact with water and occasional submersion. Rinse immediately with fresh water and dry following exposure to water, particularly salt or chlorinated.

Features and Benefits

- Full length heel ensures exceptionally smooth gait
- Split-keel design
- Integral male pyramid
- Available in 9 stiffness categories

Cosmetics

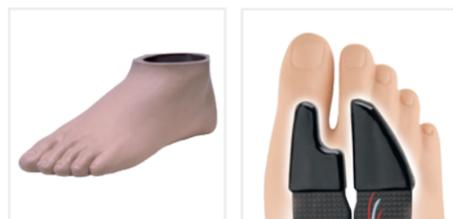

Mid profile foot shell

Sandal toe: sizes 22-28 only



Ordering Information

Trial Period - This foot has a 30 day trial period on sizes 23-30 and categories 2-8 only.

Impact Level Descriptions

Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Ordering Example - NFS1526L

Part Number	Category	Size (22-31)	Left/Right
NFS1	5	26	L

Sandal Toe Ordering Example - NFS1526SL

Part Number	Category	Size	Sandal Toe	Left/Right
NFS1	5	26	S	L

Sandal toe is only available in sizes 22-28.

Includes foot shell and Spectra® Sock

Category Selection

Weight (kg)	Impact Level		
	Low	Mod	High
44-52	1	1	2
53-59	1	2	3
60-68	2	3	4
69-77	3	4	5
78-88	4	5	6
89-100	5	6	7
101-116	6	7	8
117-130	7	8	9
131-147	8	9	
148-166	9		

Product Specification

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
166kg	L M H	515g	22-31	14.6 Size 25cm	3 years Foot shell - 6 months

Feet - PROTEOR - Freedom Sierra

Spare Parts Ordering Information

Part Number	Foot Shell Type	
NFTC-3M-0 ▲●4-R ■	Standard, with CAP	
NFTC-2M-1 ▲●4-S ■	Sandal Toe, without CAP	
Cosmetic Attachment Plate		
NCAP-3M-PO-▲●4-■		


Spare Parts Ordering Guide

▲ = Size in cm (22-31 = Regular, 22-28 = Sandal Toe)
 ● = Skin Tone (L = Light, M = Medium, D = Dark)
 ■ = "L" or "R" for Left or Right

Spectra® Sock	Colour	Size (cm)	Sock Size (cm)
NSO-NPS-20030-00	B	22-25	30
NSO-NPS-20035-00	B	26-28	35
NSO-NPS-20040-00	B	29-31	40

Colours: B = Black

Bumper Kit	Size (cm)
NKIT-00-11353-00	22-25
NKIT-00-11354-00	26-28
NKIT-00-11355-00	29-31

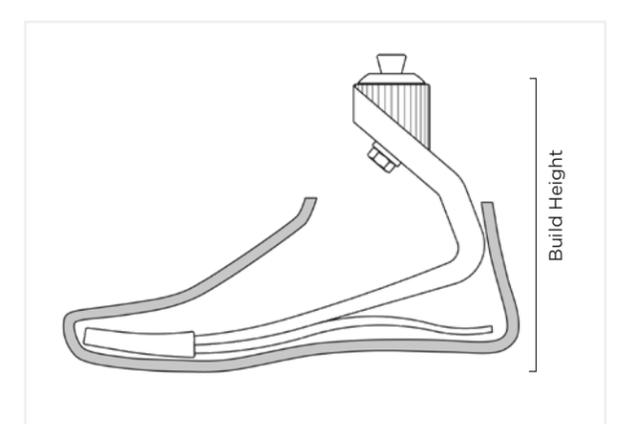

Technical Information

Build Height and Weight		
Size (cm)	Build Height (cm)	Foot inc. Shell
22-25	14.6	515g
26-28	17.2	640g
29-31	18.7	750g