

Sierra®

Ideal for active users, the Sierra foot ensures a smoother gait without the energy loss typically associated with other shock absorbing systems. Its unique angle-top design increases the length of the carbon fibre spring, maximising energy return and flexibility. The split-keel provides excellent inversion-eversion, allowing amputees to walk with confidence on uneven terrain.

CES FRE F02: Use this code to read the Sierra clinical evaluation summary on the Steeper website.

Features and Benefits

- Full length heel ensures exceptionally smooth gait
- Split keel design
- Integral male pyramid
- Available in 9 stiffness categories




Water resistant - safe for contact with water and submersion*

Cosmetics



Mid profile foot shell

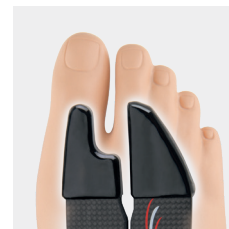

Sandal toe: sizes 22-28 only



Ordering Information

Trial Period - This foot has a 30 day trial period - only on sizes 23-30 and categories 2-8.

Impact Level Descriptions

Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Ordering Example - NFS1526L

Part Number	Category	Size	Left / Right
NFS1	5	26	L

Sandal Toe Ordering Example - NFS1526SL

Part Number	Category	Size	Sandal Toe	Left / Right
NFS1	5	26	S	L

Sandal toe is only available in sizes 22-28.

Includes foot shell and Spectra® Sock

Category Selection


Weight (kg)	Impact Level		
	LOW	MOD	HIGH
44-52	1	1	2
53-59	1	2	3
60-68	2	3	4
69-77	3	4	5
78-88	4	5	6
89-100	5	6	7
101-116	6	7	8
117-130	7	8	9
131-147	8	9	
148-166	9		

*Water note: Rinse with fresh water and dry following exposure to water, particularly salt or chlorinated.

Product Specification

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
166kg	L M H	515g	22-31	14.6 (size 25)	3 years Foot shell - 6 months

Technical Information

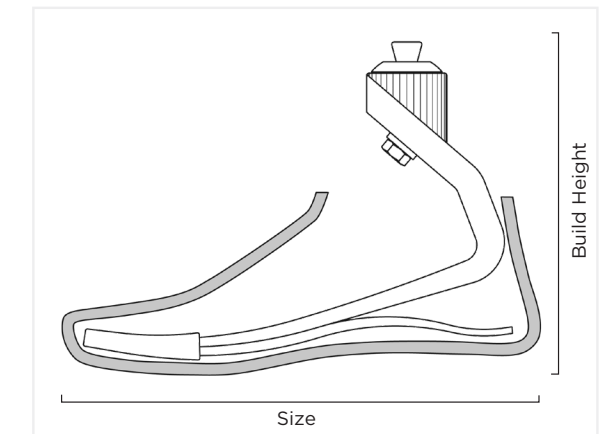

Build Height and Weight

Size (cm)	Build Height (cm)	Foot inc. Shell
22-25	14.6	515g
26-28	17.2	640g
29-31	18.7	750g

Spare Parts Ordering Information



Part Number	Foot Shell Type
NFTC-3M-0 ▲●4-R ■	Standard, with CAP
NFTC-3M-1 ▲●4-R ■	Standard, without CAP
NFTC-2H-1 ▲●4-R ■	High-profile, without CAP
NFTC-2M-1 ▲●4-S ■	Sandal Toe, without CAP



Cosmetic Attachment Plate
NCAP-3M-PO-▲●4-■

Spare Parts Ordering Guide

- ▲ = Size in cm (22-31 = Regular, 22-28 = Sandal Toe)
- = Skin Tone (L = Light, M = Medium, D = Dark)
- = "L" or "R" for Left or Right

Spectra® Sock	Size (cm)	Sock Size (cm)
NSO-NPS-X0030-00	22-25	30
NSO-NPS-X0035-00	26-28	35
NSO-NPS-X0040-00	29-31	40

X denotes colour. Insert 1 for white, or 2 for black. **Note:** Default colour is black

Bumper Kit	Size (cm)
NKIT-00-11353-00	22-25
NKIT-00-11354-00	26-28
NKIT-00-11355-00	29-31