

# Slalom Ski™

The Slalom Ski foot is the first prosthetic foot designed to be quickly and easily inserted directly into a ski binding, allowing the user to just step in and go. The Slalom eliminates the need for a ski boot and can be configured to fit most standard ski bindings. It provides flexibility, shock absorption and fluid motion without the rigid shock normally experienced by amputee skiers.

## Features and Benefits

- Eliminates the need for a ski boot, by clipping directly into a ski binding
- Provides necessary flex and shock absorption for fluid motion
- Better control and range of motion increases endurance, allowing more time on the slopes
- Slip resistant rubber sole provides stability in snow and icy conditions



Water resistant - safe for contact with water and submersion\*



## Ordering Information

Trial Period - There is no trial period on this foot.

### Impact Level Descriptions

**Moderate:** Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

**High:** Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

**Note:** Although a 'high' impact rated foot, the Slalom Ski Foot is an activity-specific foot and is intended for alpine skiing exclusively.

### To Order

Please contact Steeper Customer Services directly, providing the following information:

- Foot size in cms
- Patient's body weight in kgs (no impact level required)

This is a special order item, therefore please allow 10-14 day lead time.

### How to measure for size for a Unilateral Patient

1. Preferred measurement method: Measure the length of the patient's sound limb ski boot in cm, from heel to toe.
2. Alternative method: Measure the length of the ski binding in cm

### How to measure for size for a Bilateral Patient


1. Measure the length of the ski binding in cm

**Note:** The Slalom Ski Foot is made to "DIN specifications" in order to pair with standard ski bindings. If the patient is using something other than standard ski bindings, please contact Steeper to discuss special requirements.

Category Selection	
Weight (kg)	Category
44-72	11
73-90	12
91-114	13

The foot is supplied with the integral pyramid adapter. There is no foot shell supplied with this product.

## Technical Information

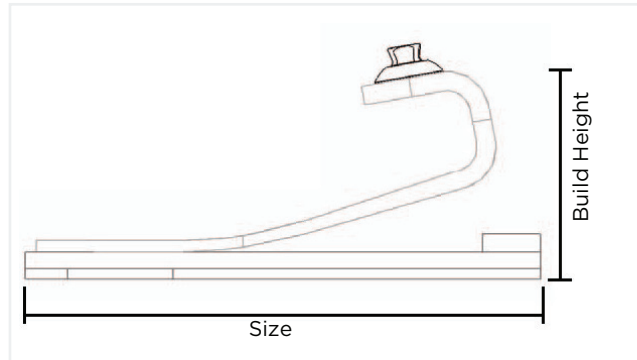

Build Height and Weight		
Size	Build Height (cm)	Foot inc. Shell
Custom	14	1055g

## Product Specification

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
114kg	M H	1055g (Size 25)	Custom made	14 (Size 25)	1 year

**\*Water note:** Rinse with fresh water and dry following exposure to water, particularly salt or chlorinated.