

Venture®

With a full length toe lever, the Venture foot provides a more dynamic response for active users.

Its two-stage response enables the front bumper to provide comfort during walking, whilst the bumper compression is limited to engage the composite keel during high activity. The heel bumper is adjustable to provide the preferred heel strike compression.

Designed for the demanding lifestyle, whether being used in the gym or the office, the precise engineering of the Venture generates a dynamic performance, without sacrificing comfort.

Features and Benefits

- Tri-axial technology to easily adapt to any terrain
- Energy storing to reduce fatigue
- Adjustable StrideControl™ - using 4mm hex key to customise the user's gait timing, even with the foot shell on
- Smooth transition from high to low activities



Stride Control™ adjuster



Energy storing



Transverse rotation
inversion-eversion

Cosmetics



CAP



Foot shell

Ordering Information

Trial Period - This foot has a 30 day trial period.

Impact Level Descriptions


Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Venture includes foot shell, CPI Sock and Cosmetic Attachment Plate. See Order Forms on page 36 for additional ordering information.

Technical Information

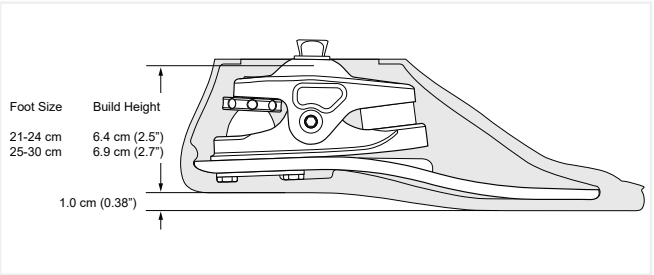

Shelltread - Ordering Information

	Shelltread	Size
	NVT ■●▲	21-30cm
	NVT ■●▲ W	25-30cm

Shelltread Ordering Guide


- = "L" or "R" for Left or Right
- = Colour (N = Neutral, J = Jet Black)
- ▲ = Foot size in cm (21-30)
- W = Wide

Product Specification

Mounting	Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty	Foot Shell
Endo (IP) Exo (ALX)	113kg (Sizes 21-24cm) 125kg (Sizes 25-30cm)		585g	21-30	6.4-6.9 (Endo IP) 12.7-13.2 (Exo ALX)	3 years Foot shell - 6 months	<div>C T B J</div> <div>Caucasian, Tan, Brown or Jet Black</div> <div> narrow & wide width</div>

Feet - College Park - Venture

Spare Parts Ordering Information

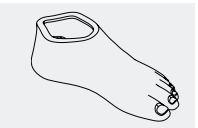

Foot Shell	Size
NV S ■●▲	21-30
NV S ■●▲ W	24-30


W = Wide




CPI Sock	Size
NV SS 2124	21-24
NV SS 2526	25-26
NV SS 2730	27-30




CAP	Size
NV C ■● 2124	21-24
NV C ■● 2526	25-26
NV C ■● 2730	27-30

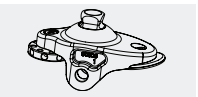

Endo Ankle	Size
NV ENA 2124	21-24
NV ENA 2530	25-30

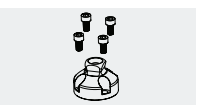


Pyramid Tool Kit	Size
NTS EXPK 30	21-30

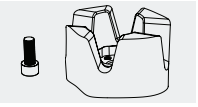

Exo Block Kit	Size
NV XBK 2124	21-24
NV XBK 2526	25-26
NV XBK 2730	27-30

Includes Mounting Bolt

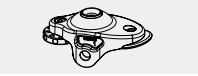

Exo Ankle	Size
NV ALXA 2124	21-24
NV ALXA 2530	25-30

Spare Parts Ordering Guide

- = "L" or "R" for Left or Right
- = Colour (C = Caucasian, T = Tan, B = Brown, J = Jet Black)
- ▲ = Foot size in cm (21-30)
- W = Wide