

# 4Seal TFS and TFC Senso Liners

The TFS/TFC above-knee 4Seal Senso liners ensures a secure fit between the residual limb and the prosthesis. The matrix of the 4Seal Senso is woven with a particularly soft thread, this results in a pleasantly smooth surface inside the liner for the care of very sensitive residual limb skin.


The silicone used is very stable, kind to skin and easy to clean. Due to four comfortable and flexible sealing fins, it is possible to provide a pleasant wearing comfort without constriction.

The liner can be turned inside out and applied without the need for alcohol spray, providing more flexibility and independence.

## Features and Benefits



## Residual Limb Suitability

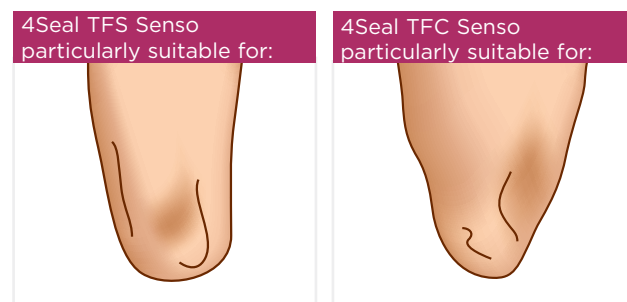

Even residual limb shape


Conical end of the residual limb



# Liners - Össur® - 4Seal TFS and TFC Senso


## Ordering Information

4Seal Senso TFS		4Seal Senso TFC	
Size	Part Number	Size	Part Number
26.5cm	NM896326	25cm	NM896425
28cm	NM896328	26.5cm	NM896426
30cm	NM896330	28cm	NM896428
32cm	NM896332	30cm	NM896430
34cm	NM896334	32cm	NM896432
36cm	NM896336	34cm	NM896434
38cm	NM896338	36cm	NM896436
40cm	NM896340	38cm	NM896438
45cm	NM896345	40cm	NM896440
50cm	NM896350		

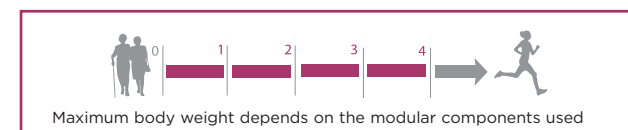

## Technical Information

Material	Silicone
Matrix	Proprioceptive, continuous, internal
Length	Approx 40cm
Amputation Level	Transfemoral
Items Included	Instruction manual, drying stand
Warranty	6 months

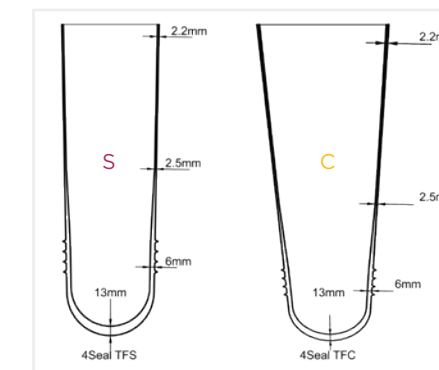


## Size Selection

Distal Circ. D (Height 4cm)	Res.Limb Length L 10 - 15	Res.Limb Length L 15 - 20	Res.Limb Length L 20 - 25	Res.Limb Length L 25 - 30	Res.Limb Length L 30 - 35	Liner Size
Proximal Circumference P*						
24 - 26.5	- 30 >	- 34 >	- 36 >	- 39 >	- 42 >	25
26.5 - 28	< 31 >	< 35 >	< 38 >	< 41 >	< 44 >	26.5
28 - 30	< 33 >	< 37 >	< 40 >	< 43 >	< 46 >	28
30 - 32	< 35 >	< 39 >	< 42 >	< 46 >	< 49 >	30
32 - 34	< 36 >	< 41 >	< 45 >	< 48 >	< 52 >	32
34 - 36	< 38 >	< 43 >	< 47 >	< 51 >	< 54 >	34
36 - 38	< 40 >	< 45 >	< 49 >	< 53 >	< 57 >	36
38 - 40	< 42 >	< 47 >	< 51 >	< 56 >	< 60 >	38
40 - 45	< 44 >	< 49 >	< 54 >	< 58 >	< 62 >	40
45 - 50	S	S	S	S	S	45
50 - 55	S	S	S	S	S	50
	S C	S C	S C	S C	S C	

### Determining the size

- Determine the **circumferential dimension D (distal)** of the residual limb 4cm above the distal end. The circumferential dimension minus 0-1 results in the Össur liner size. **Attention:** Bony structures (such as the edge of the femur) must be included in the circumferential dimension.
- Determine the residual limb length L, from the end (minus 4cm) to the inseam height (perineum).
- Determine the **circumferential dimension P (proximal)** at the inseam height (perineum)\*.
- The residual limb length and circumference are the basis for selecting the correct version of the Össur liner (transfemoral, standard or conical).

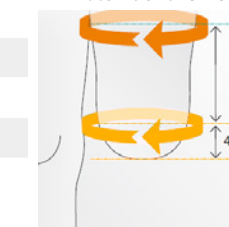

\*Explanation of the table:  
 standard /conical  
 A standard liner (= S) applies if the proximal circumference P is below the reference value (<). A conical liner (= C) applies if the proximal circumference P is above the reference value (>).

## Össur 4Seal Volume Management Sock

The Össur volume compensation stocking with integrated non-slip top band also features innovative EASY CUT technology. The stocking compensates for variations in volume and can easily be shortened to five different lengths without having to separate it, meaning no additional sewing is necessary.



Ordering Information				
Size	Össur 4Seal TFS Size	Össur 4Seal TFC Size	Length	Part Number
Small	26.5 - 32cm	25 - 28cm	24cm	N1880010
Medium	34 - 40cm	30 - 34cm	24cm	N1880011
Large	45 - 50cm	36 - 40cm	24cm	N1880012