

4Seal TFS and TFC Senso Liners

The TFS/TFC above-knee 4Seal Senso liners ensures a secure fit between the residual limb and the prosthesis. The matrix of the 4Seal Senso is woven with a particularly soft thread, this results in a pleasantly smooth surface inside the liner for the care of very sensitive residual limb skin.


The silicone used is very stable, kind to skin and easy to clean. Due to four comfortable and flexible sealing fins, it is possible to provide a pleasant wearing comfort without constriction.

The liner can be turned inside out and applied without the need for alcohol spray, providing more flexibility and independence.

Features and Benefits



Residual Limb Suitability

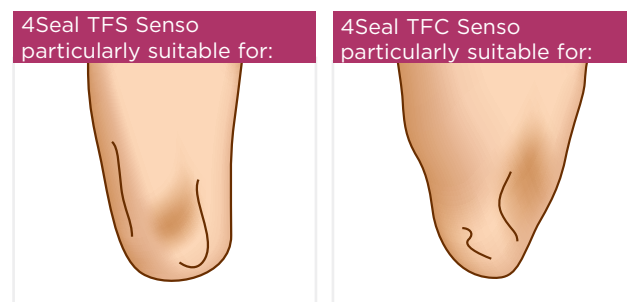

Even residual limb shape


Conical end of the residual limb



Silicone Liners - Össur® - 4Seal TFS and TFC Senso


Ordering Information

4Seal TFS Senso		4Seal TFC Senso	
Size (cm)	Part Number	Size (cm)	Part Number
26.5	NM896326	25	NM896425
28	NM896328	26.5	NM896426
30	NM896330	28	NM896428
32	NM896332	30	NM896430
34	NM896334	32	NM896432
36	NM896336	34	NM896434
38	NM896338	36	NM896436
40	NM896340	38	NM896438
45	NM896345	40	NM896440
50	NM896350	-	-

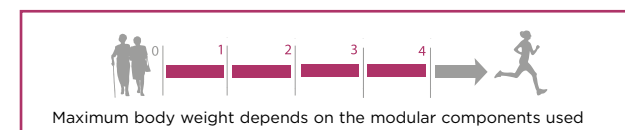

Technical Information

Material	Silicone
Matrix	Proprioceptive, continuous, internal
Length	Approx 40cm
Amputation Level	Transfemoral
Items Included	Instruction manual, drying stand
Warranty	6 months

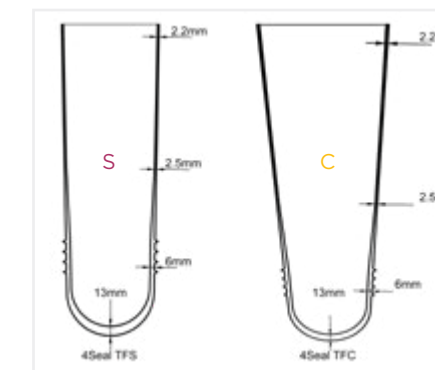


Size Selection

Distal Circ. D (Height 4cm)	Res.Limb Length L					Liner Size
	10 - 15	15 - 20	20 - 25	25 - 30	30 - 35	
Proximal Circumference P*						
24 - 26.5	< 30 >	< 34 >	< 36 >	< 39 >	< 42 >	25
26.5 - 28	< 31 >	< 35 >	< 38 >	< 41 >	< 44 >	26.5
28 - 30	< 33 >	< 37 >	< 40 >	< 43 >	< 46 >	28
30 - 32	< 35 >	< 39 >	< 42 >	< 46 >	< 49 >	30
32 - 34	< 36 >	< 41 >	< 45 >	< 48 >	< 52 >	32
34 - 36	< 38 >	< 43 >	< 47 >	< 51 >	< 54 >	34
36 - 38	< 40 >	< 45 >	< 49 >	< 53 >	< 57 >	36
38 - 40	< 42 >	< 47 >	< 51 >	< 56 >	< 60 >	38
40 - 45	< 44 >	< 49 >	< 54 >	< 58 >	< 62 >	40
45 - 50	S	S	S	S	S	45
50 - 55	S	S	S	S	S	50
	S C	S C	S C	S C	S C	

Determining the size

- Determine the **circumferential dimension D (distal)** of the residual limb 4cm above the distal end. The circumferential dimension minus 0-1 results in the Össur liner size. **Attention:** Bony structures (such as the edge of the femur) must be included in the circumferential dimension.
- Determine the residual limb length L, from the end (minus 4cm) to the inseam height (perineum).
- Determine the **circumferential dimension P (proximal)** at the inseam height (perineum).*
- The residual limb length and circumference are the basis for selecting the correct version of the Össur liner (transfemoral, standard or conical).

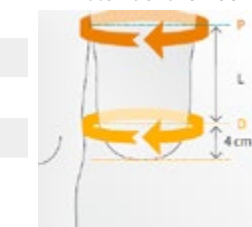

*Explanation of the table:
 standard /conical
 A standard liner (= S) applies if the proximal circumference P is below the reference value (<). A conical liner (= C) applies if the proximal circumference P is above the reference value (>).

Össur 4Seal Volume Management Sock

The Össur volume compensation stocking with integrated non-slip top band also features innovative EASY CUT technology. The stocking compensates for variations in volume and can easily be shortened to five different lengths without having to separate it, meaning no additional sewing is necessary.



Ordering Information				
Size	Össur 4Seal TFS Size (cm)	Össur 4Seal TFC Size (cm)	Length (cm)	Part Number
Small	26.5 - 32	25 - 28	24	N1880010
Medium	34 - 40	30 - 34	24	N1880011
Large	45 - 50	36 - 40	24	N1880012