

Protect 3C Cushion Liner


The Protect 3C Cushion liner is a soft silicone liner that is particularly suitable for below knee (transtibial) amputees with reduced soft tissue coverage and/or poor skin condition up to mobility class 3. It contains a surgical grade oil and vitamin E, which reduces adhesion, making the liner easier to apply and remove and may improve skin condition in as little as 3 days.

Designed to be used in a total contact socket with expulsion valve and suction sleeve, such as the Össur Genu, a high level of suspension can be achieved. The extra wall thickness of this liner provides very good cushioning and high comfort. An outer texture cover allows the liner to be fitted and removed from the socket easily without the need for time-consuming sprays.


Features and Benefits



Residual Limb Suitability

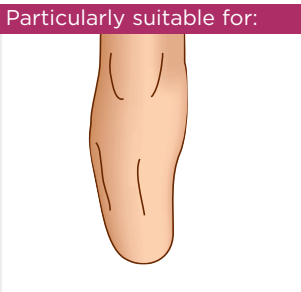

Even residual limb shape, decreasing slightly distally, good soft tissue cover

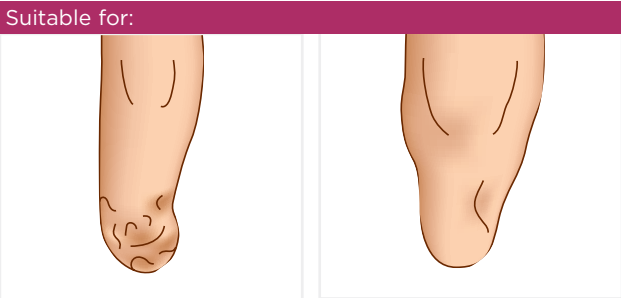

Residual limb bonier, rather poor soft tissue cover and/or with severe scars


More conical residual limb shape, good soft tissue cover

Liners - Össur® - Protect 3C Cushion

Ordering Information

Size	Part Number
18cm	NMC97318
20cm	NMC97320
22cm	NMC97322
23.5cm	NMC97323
25cm	NMC97325
26.5cm	NMC97326
28cm	NMC97328
30cm	NMC97330
32cm	NMC97332
34cm	NMC97334
36cm	NMC97336
38cm	NMC97338
40cm	NMC97340
42cm	NMC97342
45cm	NMC97345

Size Selection


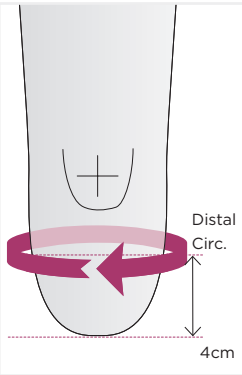


- Determining the size**
1. Measure the circumference of the residual limb 4cm above the distal end of the residual limb.
  2. Attention: Bony structures (e.g. end of the tibia) must be included in the circumference. If the distal tibia is not located within the above measurement, measure higher than 4cm above the distal end of the residual limb.
  3. The circumference minus 0-1 liner size determines the size to be ordered.

- Notes on trimming**
1. The matrix must not be cut.
  2. The trim line for a lower leg prosthesis should lie at least 10cm above the region of the socket edge.
  3. Use the Össur pinking scissors for trimming (Part No. N1950006).

Technical Information

Material	Silicone gel, opaque
Length	Approx 35cm
Amputation Level	Transtibial
Distal Connector	None
Items Included	Silicone glue, Össur washing and drying stand
Warranty	6 months

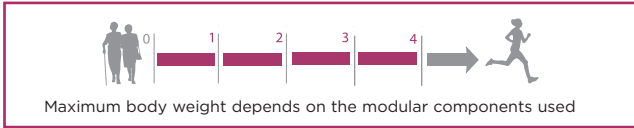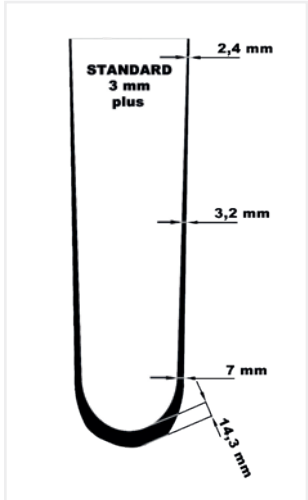

Replaceable With:



Össur Relax 3C Cushion  
Page 150