


Sensitive 3C Liner


The Sensitive 3C liner is a silicone interface liner that is suitable for below knee (transtibial) amputees with reduced soft tissue coverage and/or scar tissue up to mobility class 2 due to the soft silicone cushioning. A durable outer textile cover allows the liner to be fitted and removed easily from the socket without the need for time-consuming sprays.

**CES OSS L03:** Use this code to read the Sensitive 3C liner clinical evaluation summary on the Steeper website.


Features and Benefits

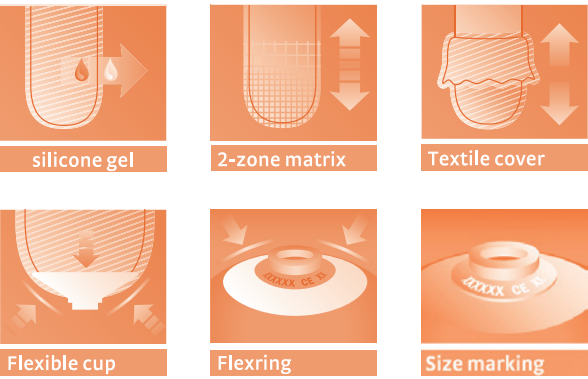

Residual Limb Suitability

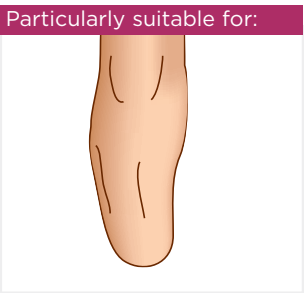

Even residual limb shape, decreasing slightly distally, good soft tissue cover

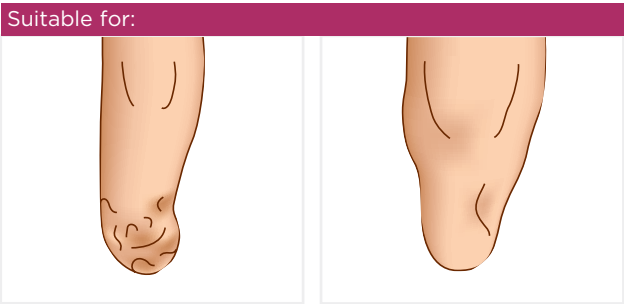


Residual limb bonier, rather poor soft tissue cover and/or with severe scars

More conical residual limb shape, good soft tissue cover

Ordering Information

Size	Part Number
18cm	NM891518
20cm	NM891520
22cm	NM891522
23.5cm	NM891523
25cm	NM891525
26.5cm	NM891526
28cm	NM891528
30cm	NM891530
32cm	NM891532
34cm	NM891534
36cm	NM891536
38cm	NM891538
40cm	NM891540
42cm	NM891542
45cm	NM891545

Size Selection

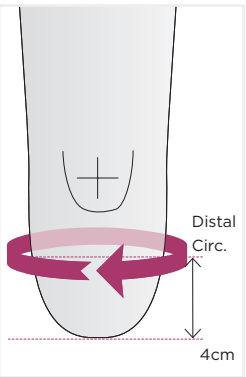

- Determining the size**
1. Measure the circumference of the residual limb 4cm above the distal end of the residual limb.
  2. Attention: Bony structures (e.g. end of the tibia) must be included in the circumference. If the distal tibia is not located within the above measurement, measure higher than 4cm above the distal end of the residual limb.
  3. The circumference minus 0-1 liner size determines the size to be ordered.


- Determining the matrix**
1. Measure the distance from the distal end of the residual limb to the head of the fibula.
  2. Attention: The matrix must not go above the head of the fibula.

- Notes on trimming**
1. The matrix must not be cut.
  2. The trim line for a lower leg prosthesis should lie at least 10cm above the region of the socket edge.
  3. Use the Össur pinking scissors for trimming (Part No. N1950006).

Liners - Össur® - Sensitive 3C

Technical Information

Material	Silicone
Matrix	2-zone matrix, internal 10+3cm
Length	Approx 35cm
Amputation Level	Transtibial
Distal Connector	Pin Adapter
Items Included	Silicone glue, Össur washing and drying stand
Warranty	6 months

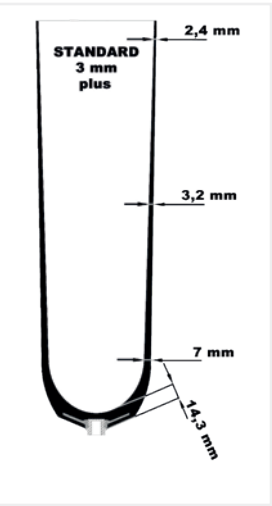

Replaceable With:

	Össur Liner Protect 3C Page 140
	Össur Liner Relax 3C Page 150
	Össur Liner First 3 Page 132