


Soft C Liner

The Soft C liner is a white silicone interface liner suitable for all mobility classes of below knee (transtibial) amputees with good or limited soft tissue coverage. This liner can be easily fitted and removed thanks to the durable outer textile cover that eliminates the need for time-consuming sprays.

CES OSS L02: Read the Soft C liner clinical evaluation summary on the Steeper website.


Features and Benefits

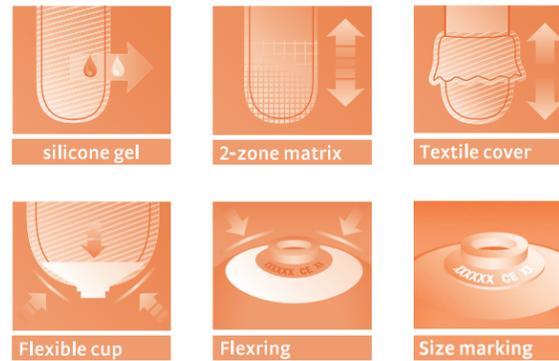

Residual Limb Suitability



Even residual limb shape, decreasing slightly distally, good soft tissue cover

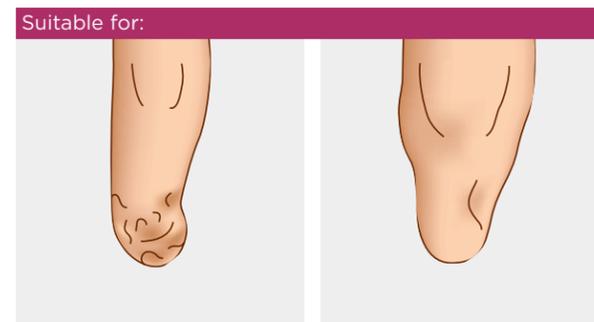

Residual limb bonier, rather poor soft tissue cover and/or with severe scars


More conical residual limb shape, good soft tissue cover



Ordering Information

Size	10+3cm Matrix Part Number
18cm	NM890418
20cm	NM890420
22cm	NM890422
23.5cm	NM890423
25cm	NM890425
26.5cm	NM890426
28cm	NM890428
30cm	NM890430
32cm	NM890432
34cm	NM890434
36cm	NM890436
38cm	NM890438
40cm	NM890440
42cm	NM890442
45cm	NM890445

Size Selection

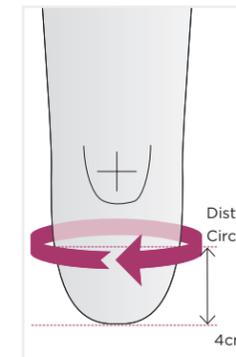

Determining the size

1. Measure the circumference of the residual limb 4cm above the distal end of the residual limb.

2. Attention: Bony structures (e.g. end of the tibia) must be included in the circumference. If the distal tibia is not located within the above measurement, measure higher than 4cm above the distal end of the residual limb.

3. The circumference minus 0-1 liner size determines the size to be ordered.

Determining the matrix

1. Measure the distance from the distal end of the residual limb to the head of the fibula.

2. Attention: The matrix must not go above the head of the fibula.

Notes on trimming



1. The matrix must not be cut.

2. The trim line for a lower leg prosthesis should lie at least 10cm above the region of the socket edge.

3. Use the Össur pinking scissors for trimming (Part No. N1950006).

Technical Information

Material	Silicone, skin colour
Matrix	2-zone matrix, internal 5+3cm and 10+3cm
Length	Approx 35cm
Amputation Level	Transtibial
Distal Connector	Pin Adapter
Items Included	Silicone glue, Össur washing and drying stand
Warranty	6 months

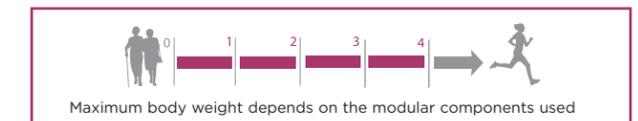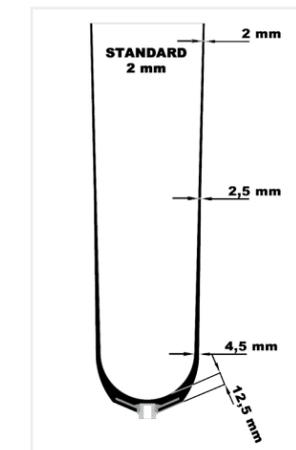

Replaceable With:



Össur First Liner

Page 122