

Please pass this leaflet to the user of this glove:

Care

Take care when fitting and using Silicone cosmetic gloves. Silicone is a soft, flexible material susceptible to punctures, cuts and abrasions. The material used will resist most staining media.

High Definition Nails

Silicone gloves have nails painted by hand following the manufacturing process. Bonding to the base material is good.

If desired it is possible to create a flexible bed that will accept nail varnishes and will allow re-colouring. A special silicone adhesive can be supplied that allows this. Please contact us for details. We do not recommend the use of polystyrene nails attached with Cyano-acrylic glue as attempts to remove nails bonded with this adhesive will permanently damage the glove

Cleaning

General soiling can be removed with soap and water, industrial hand cleaners or 'washing up' liquid. After cleaning remove all traces of the cleaner and wipe the surface dry. Most dyes will be fully removed by this process. A liquid domestic fabric softener can be used in solution to reduce the 'tackiness' of the surface.

Fitting a New Glove

- Consult your prosthetist for advice, particularly with myoelectric arms.
- Check that you have a suitable replacement glove.
- Remove the damaged glove by cutting along its length using blunt nosed scissors. Be careful not to cut cables or internal cosmesis. It may be necessary to cut digits off individually.
- Remove any internal glove fillers for later use.
- Inspect the prosthesis. If water ingress, corrosion or other damage is evident, contact your prosthetist.
- Wipe the prosthesis clean and dry carefully.
- Warming this type of glove will not alter its properties. It should be fitted at room temperature
- Apply a water based lubricant sparingly over the surface of the hand, particularly the fingers and widest sections,
- If fitting a myoelectric hand partially close it and remove the battery.
- Slide the glove in place.

Swatch Information

E24654- Standard Finish Glove Colours.

E24655- True Finish™ Glove Colours.

Note: Steeper can accept no responsibility for damage resulting from the application of inks, varnishes or artificial nails.

The design and manufacture of STEEPER equipment and components are subject to a policy of continuous re-appraisal. The Company therefore reserves the right to introduce changes and withdraw products without notice.

Enquiries:

Steeper Group, Unit 3, Stourton Link,
Intermezzo Drive, Leeds, LS10 1DF

Tel: +44 (0) 870 240 4133

Email: customerservices@steepergroup.com

www.steepergroup.com

3619 Paesanos Parkway, Suite 200
San Antonio, TX 78231

Tel: 210 481 4126

Email: inquiries@steeperusa.com

www.steeperusa.com