

Cosmetic Hands

Ideal for users who place great importance on cosmetic appearance and system weight, the advanced materials and manufacturing techniques used to produce these hands ensure detailed, lightweight and durable solutions. These hands are designed for dress purposes only and are intended for use with Steeper cosmetic gloves and high-definition silicone covers. Features include:

- Made from flexible skinned lightweight foam around a metal skeleton mounted on a plastic base
- Passive fingers but the skeleton allows for individual digits to be positioned manually
- Robust construction for positioning
- Easy connection to oval or round wrists, or M12 threaded stems

Extended Wires Foam Hand

An additional range of cosmetic hands with extended wires and no internal base is also available to construct robust cosmetic prostheses for partial hand or wrist disarticulation patients.

A socket with a tie plate (B05864) is fabricated and the hand is shortened to the appropriate length, then the extended wires are trimmed and passed through the tie plate. Foam may be added to smooth the transition between socket and hand before the cosmetic glove is fitted.



Cosmetic Hands and Gloves Ordering Information

Hand Part Number	Extended Wires Part Number	Left/Right	Glove No.	Hand Size (W, L, Wrist Dia)	Application	Length	Fitted Measures	
							Palm Circumference	Middle Finger
B11081	B11103	Left	19	2" x 3" x 1 1/2"	Unisex Child - 2-5 Yrs	220mm (8 1/2")	126mm (5")	49mm (2")
B11082	B11104	Right	20	2" x 3" x 1 1/2"	Unisex Child - 2-5 Yrs	220mm (8 1/2")	127mm (5")	49mm (2")
B11079	B11105	Left	23	2 1/4" x 4" x 1 3/4"	Unisex Child - 5-8 Yrs	220mm (8 1/2")	141mm (5 1/2")	58mm (2 1/4")
B11080	B11106	Right	24	2 1/4" x 4" x 1 3/4"	Unisex Child - 5-8 Yrs	220mm (8 1/2")	145mm (5 3/4")	61mm (2 3/8")
B11077	B11101	Left	17	2 3/8" x 4" x 1 3/4"	Unisex Child - 6-10 Yrs	270mm (10 1/4")	156mm (6 1/8")	61mm (2 3/8")
B11078	B11102	Right	18	2 3/8" x 4" x 1 3/4"	Unisex Child - 6-10 Yrs	270mm (10 1/4")	152mm (6")	67mm (2 5/8")
B11075	B11099	Left	15	2 1/2" x 4" x 1 3/4"	Unisex Child - 8-12 Yrs	240mm (9 1/2")	160mm (6 1/4")	66mm (2 5/8")
B11076	B11100	Right	16	2 1/2" x 4" x 1 3/4"	Unisex Child - 8-12 Yrs	240mm (9 1/2")	160mm (6 1/4")	69mm (2 3/4")
B11073	B11097	Left	11	2 1/2" x 5" x 2"	Female Child - 8-12 Yrs	290mm (11 1/2")	165mm (6 1/2")	72mm (2 7/8")
B11074	B11098	Right	12	2 1/2" x 5" x 2"	Female Child - 8-12 Yrs	290mm (11 1/2")	172mm (6 3/4")	71mm (2 3/4")
FHL7	FHEXL7	Left	L7	2 3/4" x 5 1/2" x 2 1/8"	Female Youth/Adult	580mm (23")	172mm (6 3/4")	82mm (3 1/4")
FHL8	FHEXL8	Right	L8	2 3/4" x 5 1/2" x 2 1/8"	Female Youth/Adult	580mm (23")	172mm (6 3/4")	83mm (3 1/4")
B11069	B11095	Left	7*	2 3/4" x 5 1/2" x 2 1/8"	Female Youth/Adult	350mm (13 3/4")	169mm (6 5/8")	75mm (3")
B11070	B11096	Right	8*	2 3/4" x 5 1/2" x 2 1/8"	Female Youth/Adult	350mm (13 3/4")	173mm (6 3/4")	79mm (3 1/8")
B11067	B11067EX	Left	107	2 3/4" x 5 1/2" x 2 1/8"	Unisex Youth/Adult - HD	340mm (13 1/2")	168mm (6 5/8")	75mm (3")
B11067	B11067EX	Left	90	2 3/4" x 5 1/2" x 2 1/8"	Unisex Youth/Adult - HD	400mm (15 3/4")	168mm (6 5/8")	75mm (3")
B11068	B11068EX	Right	108	2 3/4" x 5 1/2" x 2 1/8"	Unisex Youth/Adult - HD	340mm (13 1/2")	175mm (6 7/8")	75mm (3")
B11068	B11068EX	Right	91	2 3/4" x 5 1/2" x 2 1/8"	Unisex Youth/Adult - HD	400mm (15 3/4")	175mm (6 7/8")	75mm (3")
FHL5	FHEXL5	Left	L5	3" x 5 1/2" x 2 1/8"	Unisex Adult	640mm (25 1/4")	186mm (7 1/4")	81mm (3 1/16")
FHL6	FHEXL6	Right	L6	3" x 5 1/2" x 2 1/8"	Unisex Adult	640mm (25 1/4")	186mm (7 1/4")	85mm (3 3/8")
B11063	B11091	Left	5*	3" x 5 1/2" x 2 1/8"	Male Adult	400mm (15 3/4")	183mm (7 3/16")	83mm (3 1/4")
B11064	B11092	Right	6*	3" x 5 1/2" x 2 1/8"	Male Adult	400mm (15 3/4")	188mm (7 3/8")	83mm (3 1/4")
B11065	B11093	Left	5*	3" x 5 1/2" x 2 1/8"	Female Adult	360mm (14")	184mm (7 1/4")	84mm (3 5/16")
B11066	B11094	Right	6*	3" x 5 1/2" x 2 1/8"	Female Adult	360mm (14")	187mm (7 3/8")	88mm (3 1/2")
B11061	B11061EX	Left	105L*	3" x 5 1/2" x 2 1/8"	Female Adult - HD	540mm (21 1/4")	191mm (7 1/2")	80mm (3 1/8")
B11062	B11062EX	Right	106L*	3" x 5 1/2" x 2 1/8"	Female Adult - HD	540mm (21 1/4")	189mm (7 1/2")	82mm (3 1/4")
FHL3	FHEXL3	Left	L3	3 1/4" x 5 1/2" x 2 1/8"	Male Adult	630mm (25")	205mm (8")	89mm (3 1/2")
FHL4	FHEXL4	Right	L4	3 1/4" x 5 1/2" x 2 1/8"	Male Adult	630mm (25")	205mm (8")	90mm (3 1/2")
B11059	B11087	Left	3*	3 1/4" x 5 1/2" x 2 1/8"	Male Adult	360mm (14")	204mm (8")	90mm (3 1/2")
B11060	B11088	Right	4*	3 1/4" x 5 1/2" x 2 1/8"	Male Adult	360mm (14")	205mm (8")	88mm (3 1/2")
FHL21	FHEXL21	Left	L21	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - Long Fingers	430mm (17")	202mm (7 7/8")	95mm (3 3/4")
FHL22	FHEXL22	Right	L22	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - Long Fingers	430mm (17")	202mm (7 7/8")	95mm (3 3/4")
B11057	B11089	Left	21*	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - Long Fingers	430mm (17")	204mm (8")	100mm (4")
B11058	B11090	Right	22*	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - Long Fingers	430mm (17")	202mm (7 7/8")	94mm (3 5/8")
B11055	B11055EX	Left	103	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - HD	350mm (13 3/4")	201mm (7 7/8")	89mm (3 1/2")
B11056	B11056EX	Right	104	3 1/4" x 5 1/2" x 2 1/8"	Male Adult - HD	350mm (13 3/4")	205mm (8")	90mm (3 1/2")
FHL1	FHEXL1	Left	L1	3 1/2" x 5 1/2" x 2 1/8"	Male Adult	620mm (24 1/2")	212mm (8 1/4")	100mm (4")
FHL2	FHEXL2	Right	L2	3 1/2" x 5 1/2" x 2 1/8"	Male Adult	620mm (24 1/2")	212mm (8 1/4")	98mm (3 7/8")
B11053	B11083	Left	1	3 1/2" x 5 1/2" x 2 1/8"	Male Adult	350mm (13 3/4")	220mm (8 5/8")	99mm (3 7/8")
B11054	B11084	Right	2	3 1/2" x 5 1/2" x 2 1/8"	Male Adult	350mm (13 3/4")	219mm (8 5/8")	103mm (4")
B21139	B21139EX	Left	200	3 3/4" x 5 1/2" x 2 1/8"	Male Adult	400mm (15 3/4")	229mm (9")	108mm (4 1/4")
B21140	B21140EX	Right	201	3 3/4" x 5 1/2" x 2 1/8"	Male Adult	400mm (15 3/4")	229mm (9")	108mm (4 1/4")

*Partial hand available.

Please see the following pages for our entire glove range in more detail, including specific ordering information for gloves and cosmetic hands.

Cosmetic Gloves (for passive hands)

We manufacture a wide range of cosmetic gloves in both PVC and silicone, designed to suit Steeper passive hands as well as hands from other manufacturers.

Both PVC and silicone gloves are available in Standard or TrueFinish™ shades. TrueFinish™ features micropigmentation for a more lifelike appearance, available in 19 colours (E0 - E18).

Our glove range is detailed over the following pages with reference made to the glove number e.g. 302. To select the correct glove apply a prefix to the number as follows:

- CG** - Standard PVC
 - TG** - TrueFinish™ PVC
 - SG** - Standard Silicone
 - TSG** - TrueFinish™ Silicone
- e.g. TSG302/E4 - TrueFinish™ Silicone Glove 7 3/4" Left, Colour E4

Please refer to opposite page for cosmetic glove colour options.



Cosmetic Glove

Elegance and Elegance Plus Gloves (for myoelectric and functional hands)

The Elegance range is available in both PVC and silicone, and in 19 TrueFinish™ shades and have been designed to work with myoelectric hands, Steeper cable and spring operated hands, as well as myoelectric and functional hands from other manufacturers.

The Elegance Plus silicone glove has been specifically designed to fit the Myo Kinisi. Crafted using multiple layers of advanced silicone material and constructed with an integrated, reinforced inner mesh layer these gloves are highly tear-resistant and are especially suitable for higher activity patients. The addition of an easy-glide coating allows for clothing to be donned or doffed easily and the enhanced detailing on the knuckles, palms, and nail finish provides increased realism.

Two-Tone Finish Gloves

Our two-tone finish gloves are available in Standard (CG) or TrueFinish™ (TG) PVC. Please select the palmar colour (lighter and always allocated to the palm) and then the secondary dorsal colour. The dorsal colour is then applied to the glove to complete the two-tone appearance. Please use the glove ordering chart where this is required and discuss your requirements with your local supplier.

Note: Two tone gloves are only available in dorsal shades E13, E16 and E18

Partial Hand Gloves

Several popular glove sizes are also available with a wider wrist and are recognisable by the identifier PH.

e.g. TG3PH - TrueFinish™ PVC, 3 1/4" Left Wide Wrist

These can be used for partial hand patients who require a cosmetic replacement. The wider wrist allows the residual limb to fit inside the glove and the remaining void can be filled with a soft filler.



For partial hand patients where the residual limb will not fit inside the glove, a zip may be fitted to CG and TG gloves. Please use the glove ordering chart where this is required and discuss your requirements with your local supplier.

Note: Two tone, bilateral and unpainted nail gloves are special orders, please specify these details at the point of order.



Two-Tone Finish PVC Glove

Glove Dimensions

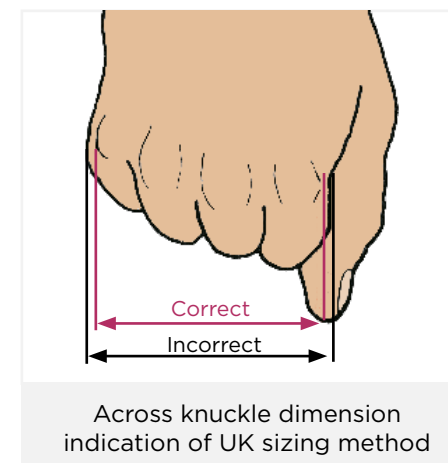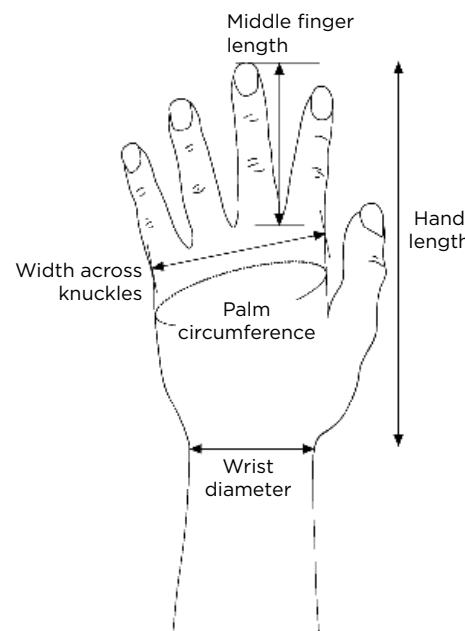

Across knuckle dimension indication of UK sizing method

Note: This is described as 'Hand size' on the ordering information table on p.87.

Cosmetic Glove Colours


To help you select the most appropriate colour of glove, swatches are available and supplied without charge. These are manufactured from production silicone for the most accurate colour matching and will not fade or discolour over time.

E0	E1	E2	E3	E4
E5	E6	E7	E8	E9
E10	E11	E12	E13	E14
E15	E16	E17	E18	Note: Swatch colours are for indication only.

A solid black glove is available using code E20, this is supplied without painted nails.



Children 2-5 Years Glove | 19 & 20
Unisex, 2"

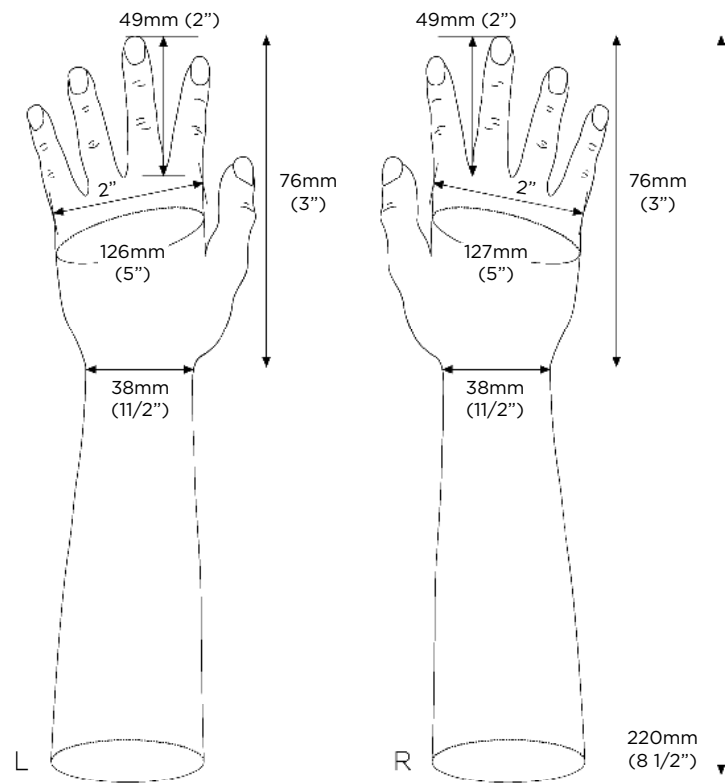

Note: Round Wrist only available with this glove

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11081	B11082
Foam Hand with Extended Wires	B11103	B11104

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG19/E#	TSG20/E#
Standard Silicone Glove	SG19/E#	SG20/E#
TrueFinish™ PVC Glove	TG19/E#	TG20/E#
Standard PVC Glove	CG19/E#	CG20/E#

Child 5-8 Years Glove | 23 & 24
Unisex, 2 1/4"

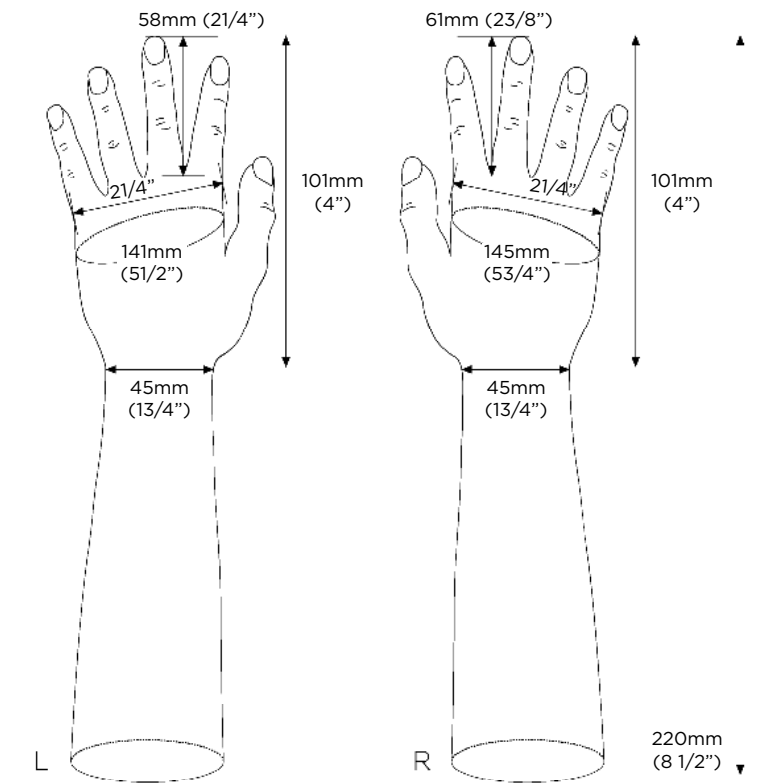

Note: Oval and Round Wrist options available.

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11079	B11080
Foam Hand with Extended Wires	B11105	B11106

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG23/E#	TSG24/E#
Standard Silicone Glove	SG23/E#	SG24/E#
TrueFinish™ PVC Glove	TG23/E#	TG24/E#
Standard PVC Glove	CG23/E#	CG24/E#

Child 6-10 Years Glove | 17 & 18
Unisex, 2³/₈"

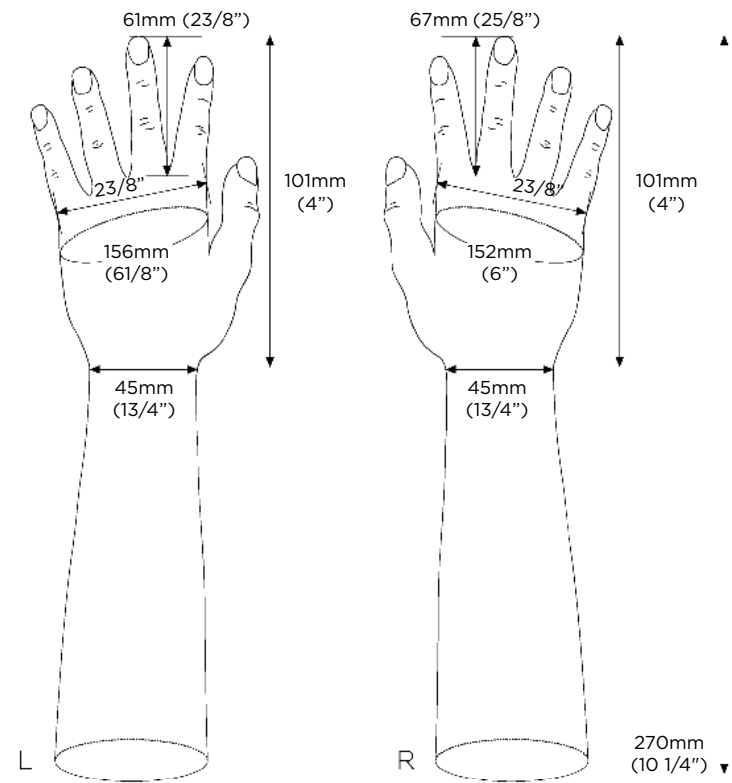

Note: Oval and Round Wrist options available.

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11077	B11078
Foam Hand with Extended Wires	B11101	B11102

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG17/E#	TSG18/E#
Standard Silicone Glove	SG17/E#	SG18/E#
TrueFinish™ PVC Glove	TG17/E#	TG18/E#
Standard PVC Glove	CG17/E#	CG18/E#

Child 8-12 Years Glove | 15 & 16
Unisex, 2¹/₂"

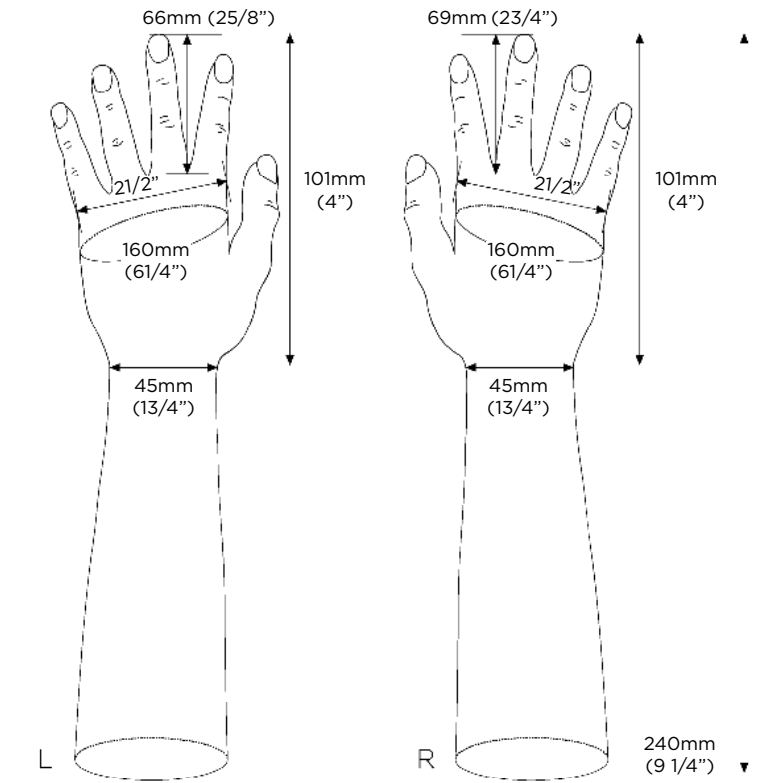

Note: Oval and Round Wrist options available.

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11075	B11076
Foam Hand with Extended Wires	B11099	B11100

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG15/E#	TSG16/E#
Standard Silicone Glove	SG15/E#	SG16/E#
TrueFinish™ PVC Glove	TG15/E#	TG16/E#
Standard PVC Glove	CG15/E#	CG16/E#

**Child 8-12 Years Glove | 11 & 12
Female, 2½"**

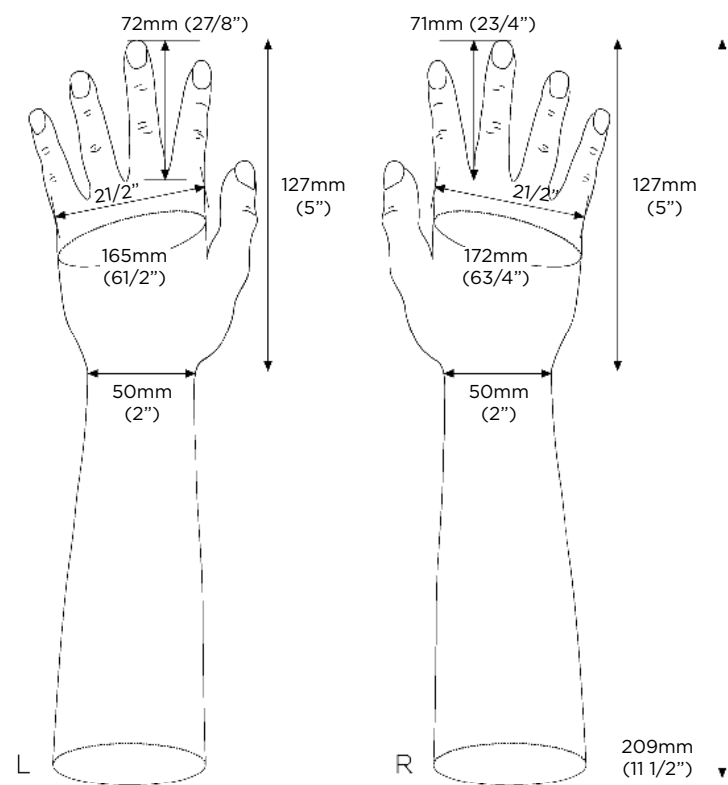

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11073	B11074
Foam Hand with Extended Wires	B11097	B11098

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG11/E#	TSG12/E#
Standard Silicone Glove	SG11/E#	SG12/E#
TrueFinish™ PVC Glove	TG11/E#	TG12/E#
Standard PVC Glove	CG11/E#	CG12/E#

**Youth/Adult Glove | L7 & L8
Female, Long Mould, 2¾"**

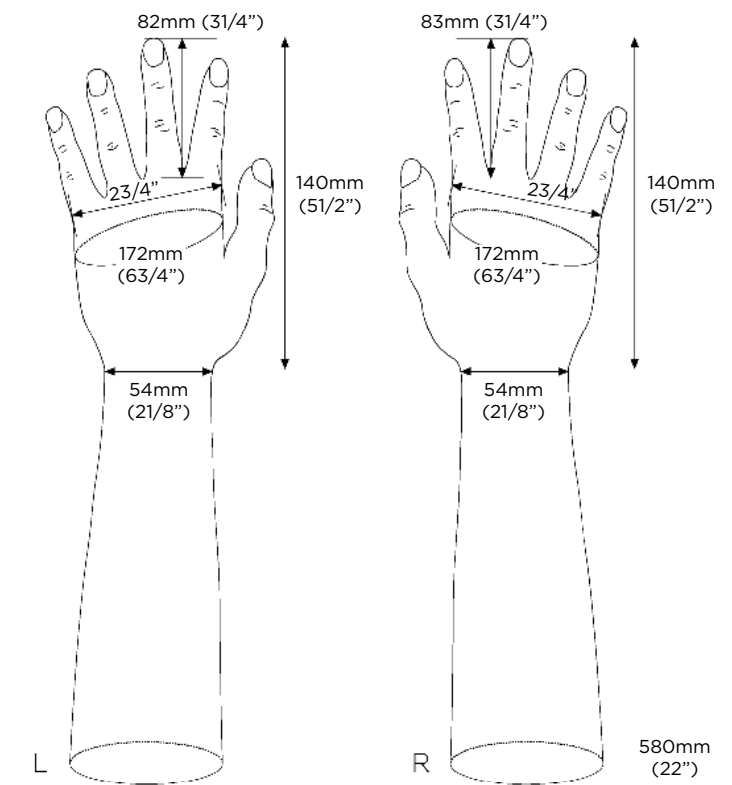

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	FHL7	FHL8
Foam Hand with Extended Wires	FHEXL7	FHEXL8

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSGL7/E#	TSGL8/E#
Standard Silicone Glove	SGL7/E#	SGL8/E#
TrueFinish™ PVC Glove	TGL7/E#	TGL8/E#
Standard PVC Glove	CGL7/E#	CGL8/E#

Youth/Adult Glove | 7 & 8
Female, 2¾"

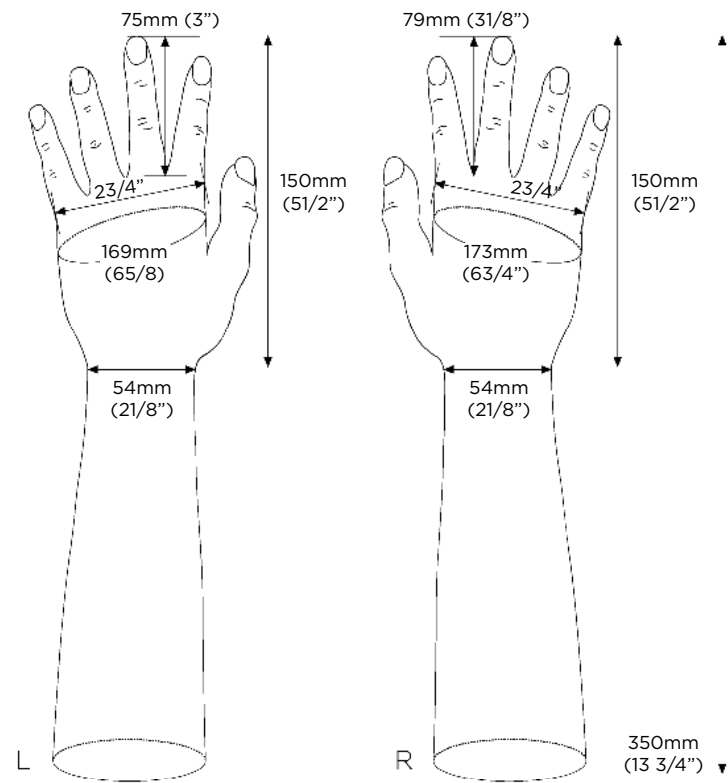

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B11069	B11070
Foam Hand with Extended Wires	B11095	B11096

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.


	Left	Right
TrueFinish™ Silicone Glove	TSG7/E#	TSG8/E#
Standard Silicone Glove	SG7/E#	SG8/E#
TrueFinish™ PVC Glove	TG7/E#	TG8/E#
Standard PVC Glove	CG7/E#	CG8/E#

Partial Hand Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 68 for swatches.

	Left	Right
TrueFinish™ Silicone	TSG7PH/E#	TSG8PH/E#
Standard Silicone	SG7PH/E#	SG8PH/E#
TrueFinish™ PVC	TG7PH/E#	TG8PH/E#
Standard PVC	CG7PH/E#	CG8PH/E#

Youth/Adult High Definition Glove | 107 & 108
Female, 2¾"

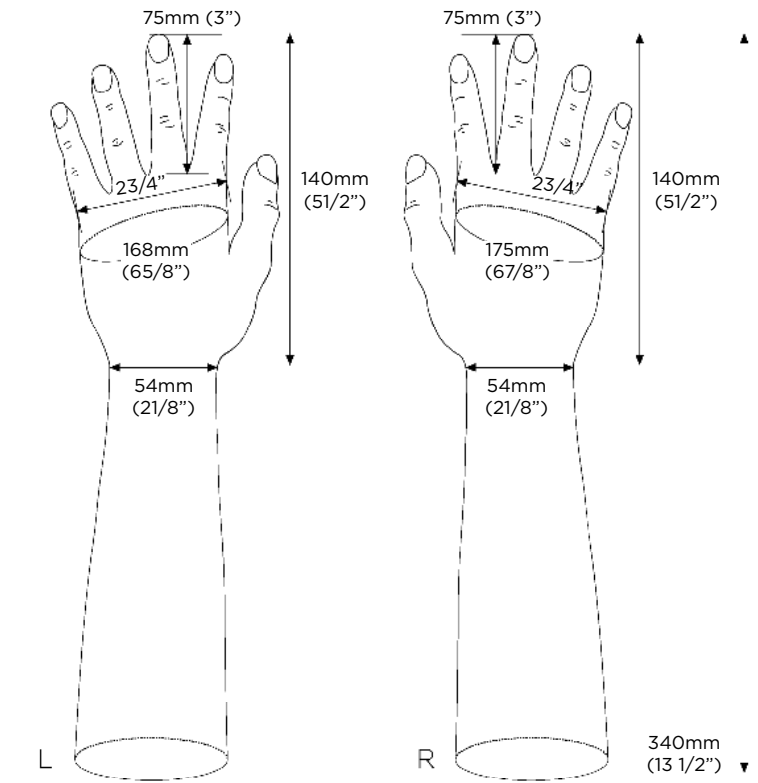

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11067	B11068
Foam Hand with Extended Wires	B11067EX	B11068EX

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG107/E#	TSG108/E#
Standard Silicone Glove	SG107/E#	SG108/E#
TrueFinish™ PVC Glove	TG107/E#	TG108/E#
Standard PVC Glove	CG107/E#	CG108/E#

Youth/Adult High Definition Glove | 90 & 91
Male, 2¾"

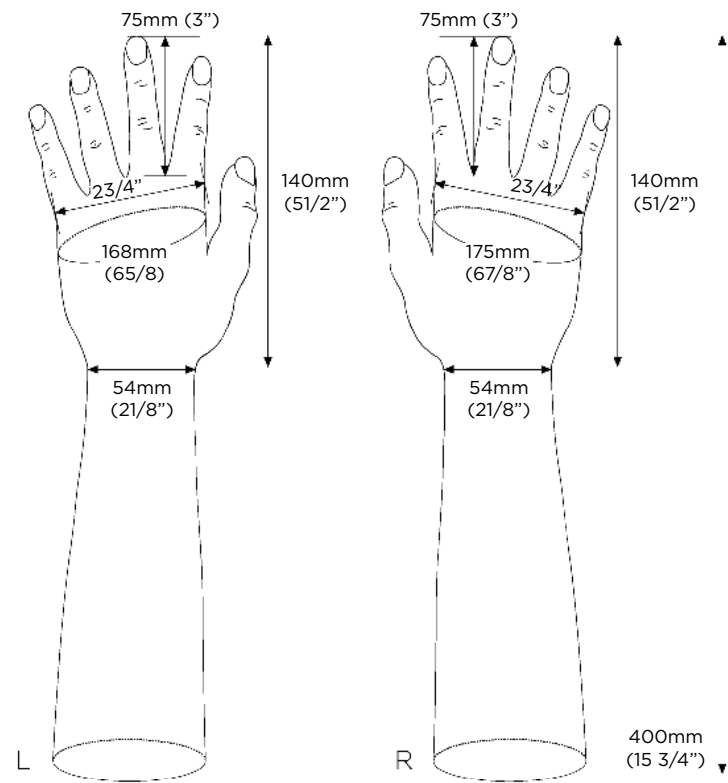

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11067	B11068
Foam Hand with Extended Wires	B11067EX	B11068EX

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG90/E#	TSG91/E#
Standard Silicone Glove	SG90/E#	SG91/E#
TrueFinish™ PVC Glove	TG90/E#	TG91/E#
Standard PVC Glove	CG90/E#	CG91/E#

Adult Glove | L5 & L6
Unisex, Long Mould, 3"

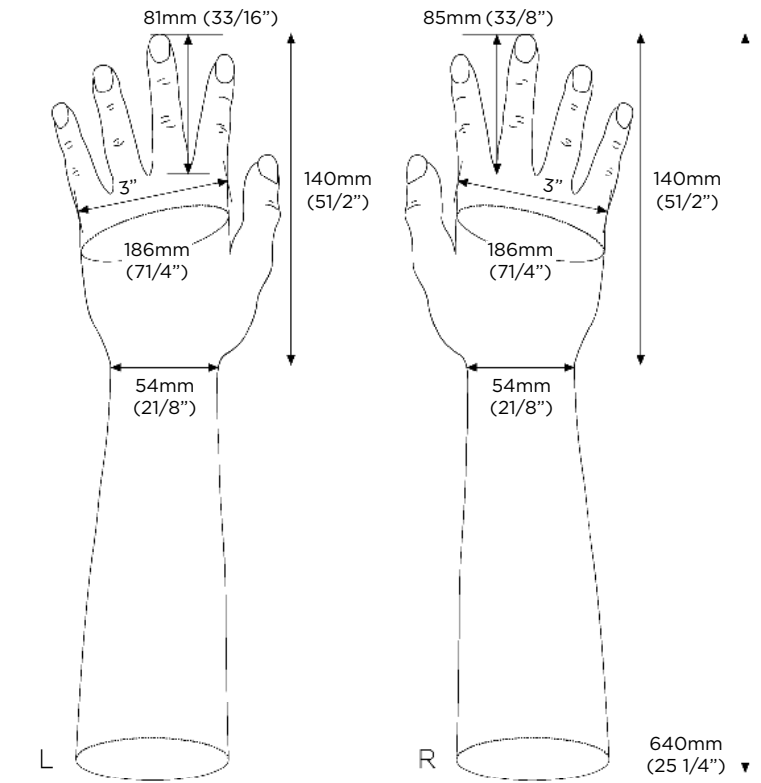

Cosmetic/Passive Hands Ordering Information



	Left	Right
Foam Hand	FHL5	FHL6
Foam Hand with Extended Wires	FHEXL5	FHEXL6

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSGL5/E#	TSGL6/E#
Standard Silicone Glove	SGL5/E#	SGL6/E#
TrueFinish™ PVC Glove	TGL5/E#	TGL6/E#
Standard PVC Glove	CGL5/E#	CGL6/E#

Adult Glove | 5 & 6
Male, 3"

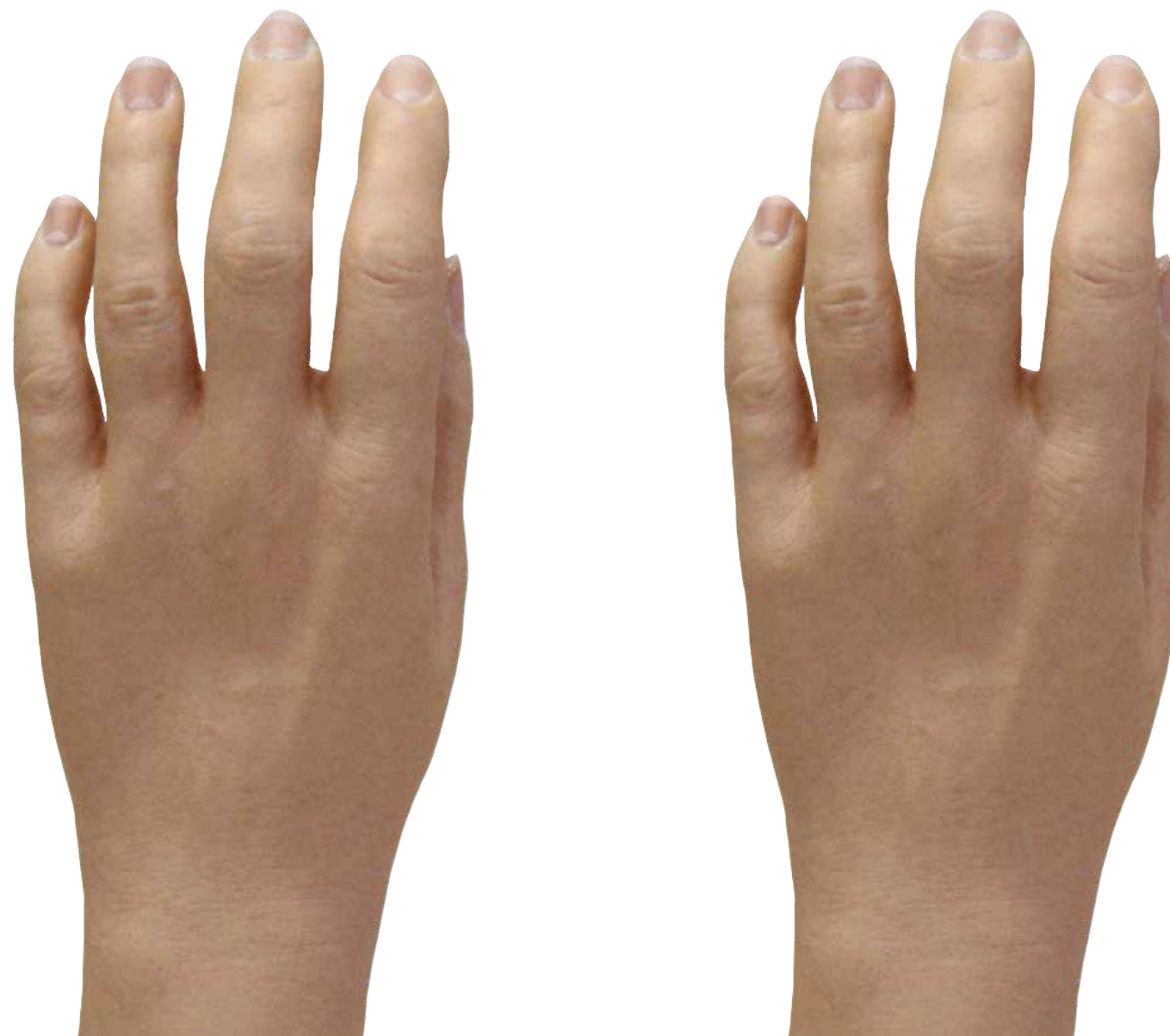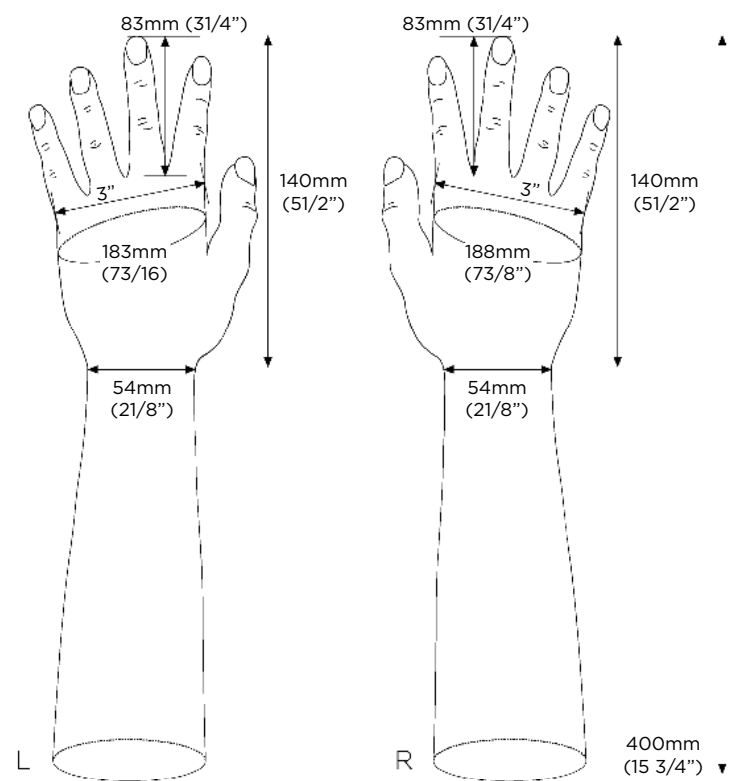

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B11063	B11064
Foam Hand with Extended Wires	B11091	B11092

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.


	Left	Right
TrueFinish™ Silicone Glove	TSG5/E#	TSG6/E#
Standard Silicone Glove	SG5/E#	SG6/E#
TrueFinish™ PVC Glove	TG5/E#	TG6/E#
Standard PVC Glove	CG5/E#	CG6/E#

Partial Hand Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 68 for swatches.

	Left	Right
TrueFinish™ Silicone	TSG5PH/E#	TSG6PH/E#
Standard Silicone	SG5PH/E#	SG6PH/E#
TrueFinish™ PVC	TG5PH/E#	TG6PH/E#
Standard PVC	CG5PH/E#	CG6PH/E#

Adult Glove | 5 & 6
Female, 3"

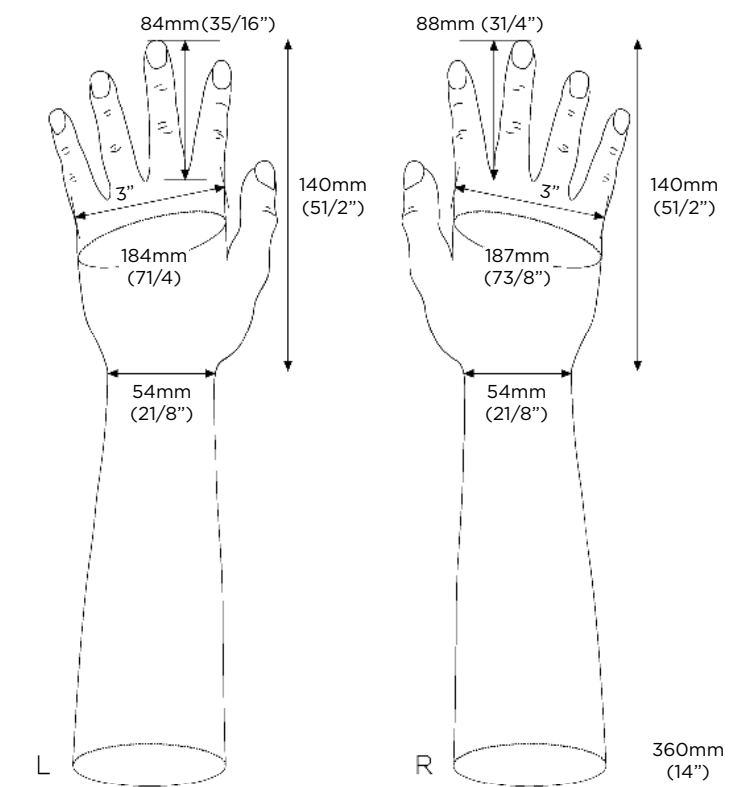

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11065	B11066
Foam Hand with Extended Wires	B11093	B11094

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 88 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG5/E#	TSG6/E#
Standard Silicone Glove	SG5/E#	SG6/E#
TrueFinish™ PVC Glove	TG5/E#	TG6/E#
Standard PVC Glove	CG5/E#	CG6/E#

**Adult High Definition Glove | 105L & 106L
Female, 3"**

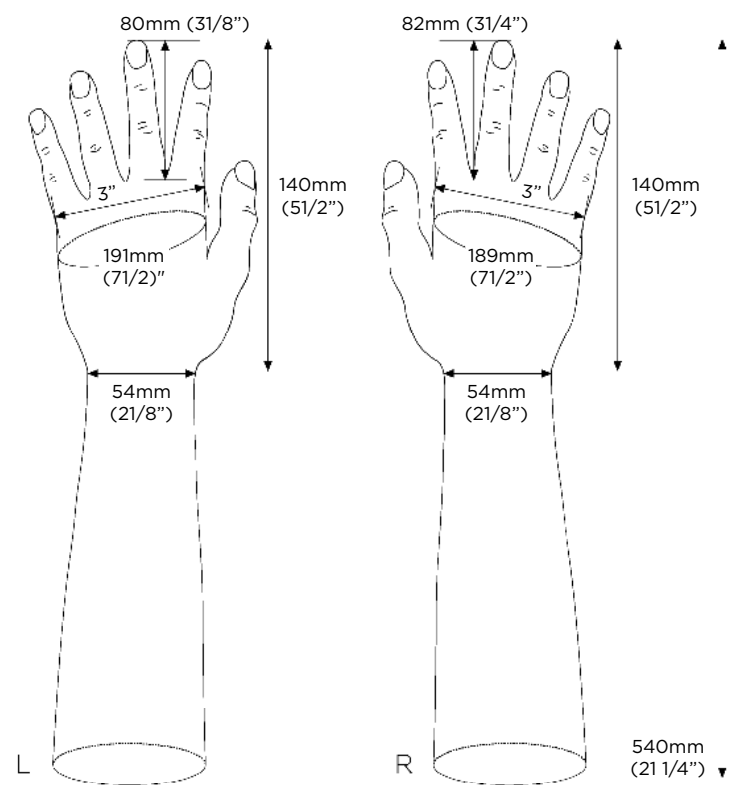

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B11061	B11062
Foam Hand with Extended Wires	B11061EX	B11062EX

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.


	Left	Right
TrueFinish™ Silicone Glove	TSG105L/E#	TSG106L/E#
Standard Silicone Glove	SG105L/E#	SG106L/E#
TrueFinish™ PVC Glove	TG105L/E#	TG106L/E#
Standard PVC Glove	CG105L/E#	CG106L/E#

Partial Hand Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 68 for swatches.

	Left	Right
TrueFinish™ Silicone	TSG105PH/E#	TSG106PH/E#
Standard Silicone	SG105PH/E#	SG106PH/E#
TrueFinish™ PVC	TG105PH/E#	TG106PH/E#
Standard PVC	CG105PH/E#	CG106PH/E#

**Adult Glove | L3 & L4
Male, Long Mould, 3 1/4"**

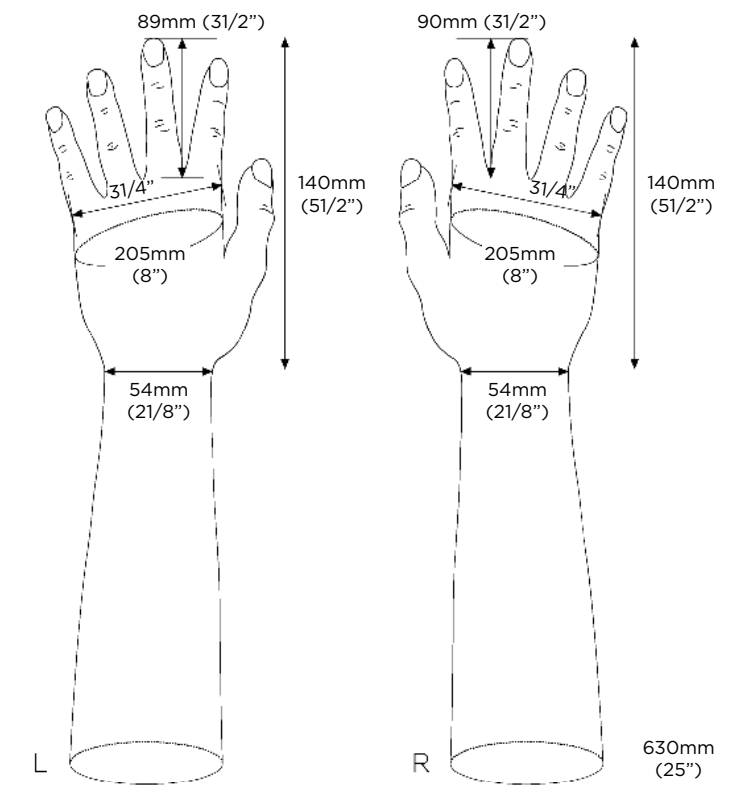

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	FHL3	FHL4
Foam Hand with Extended Wires	FHEXL3	FHEXL4

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSGL3/E#	TSGL4/E#
Standard Silicone Glove	SGL3/E#	SGL4/E#
TrueFinish™ PVC Glove	TGL3/E#	TGL4/E#
Standard PVC Glove	CGL3/E#	CGL4/E#

Adult Glove | 3 & 4
Male, 3 1/4"

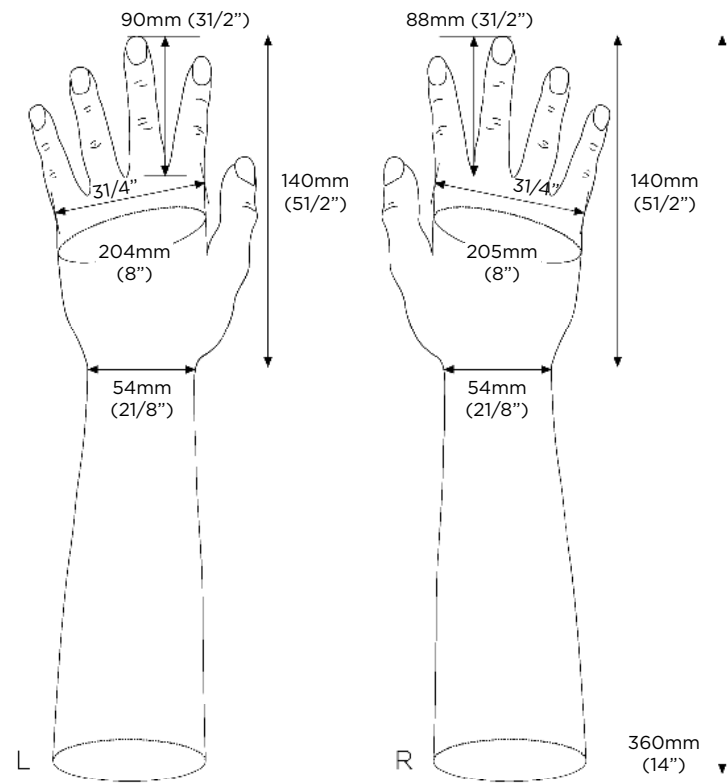

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B11059	B11060
Foam Hand with Extended Wires	B11087	B11088

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.


	Left	Right
TrueFinish™ Silicone Glove	TSG3/E#	TSG4/E#
Standard Silicone Glove	SG3/E#	SG4/E#
TrueFinish™ PVC Glove	TG3/E#	TG4/E#
Standard PVC Glove	CG3/E#	CG4/E#

Partial Hand Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 68 for swatches.

	Left	Right
TrueFinish™ Silicone	TSG3PH/E#	TSG4PH/E#
Standard Silicone	SG3PH/E#	SG4PH/E#
TrueFinish™ PVC	TG3PH/E#	TG4PH/E#
Standard PVC	CG3PH/E#	CG4PH/E#

Adult | L21 & L22
Male, Long Mould & Fingers, 3 1/4"

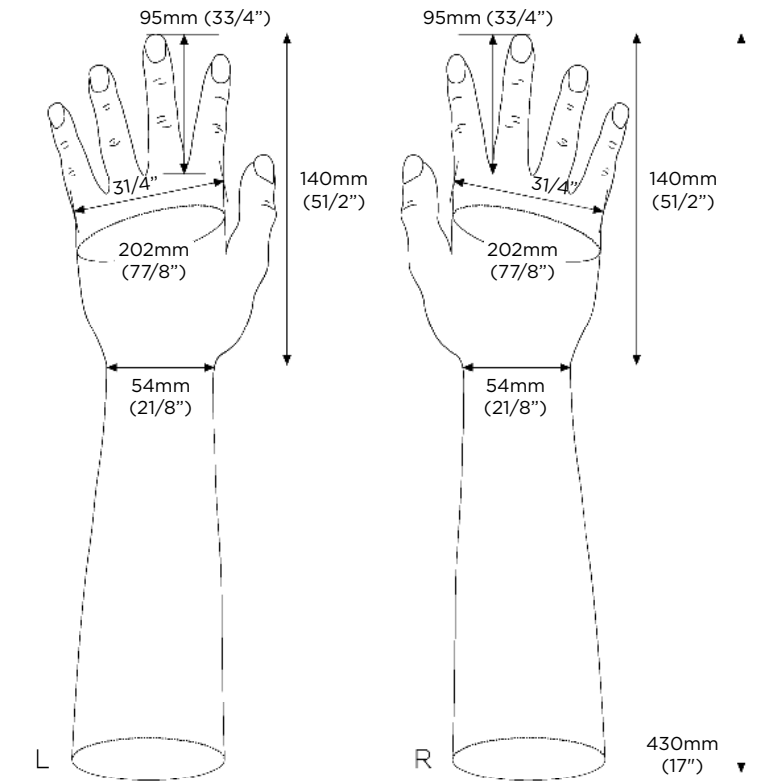

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	FHL21	FHL22
Foam Hand with Extended Wires	FHEXL21	FHEXL22

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSGL21/E#	TSGL22/E#
Standard Silicone Glove	SGL21/E#	SGL22/E#
TrueFinish™ PVC Glove	TGL21/E#	TGL22/E#
Standard PVC Glove	CGL21/E#	CGL22/E#

Adult Glove | 21 & 22
Male, Long Fingers, 3 1/4"

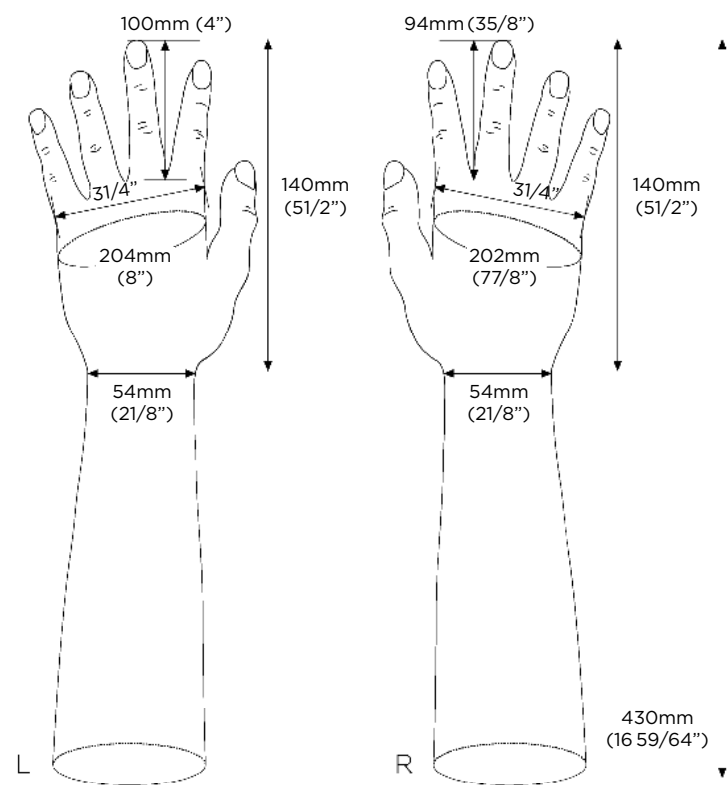

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B11057	B11058
Foam Hand with Extended Wires	B11089	B11090

Cosmetic Glove Ordering Information


Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG21/E#	TSG22/E#
Standard Silicone Glove	SG21/E#	SG22/E#
TrueFinish™ PVC Glove	TG21/E#	TG22/E#
Standard PVC Glove	CG21/E#	CG22/E#

Partial Hand Glove Ordering Information

	Left	Right
TrueFinish™ Silicone	TSG21PH/E#	TSG22PH/E#
Standard Silicone	SG21PH/E#	SG22PH/E#
TrueFinish™ PVC	TG21PH/E#	TG22PH/E#
Standard PVC	CG21PH/E#	CG22PH/E#

Adult High Definition Glove | 103 & 104
Male, 3 1/4"

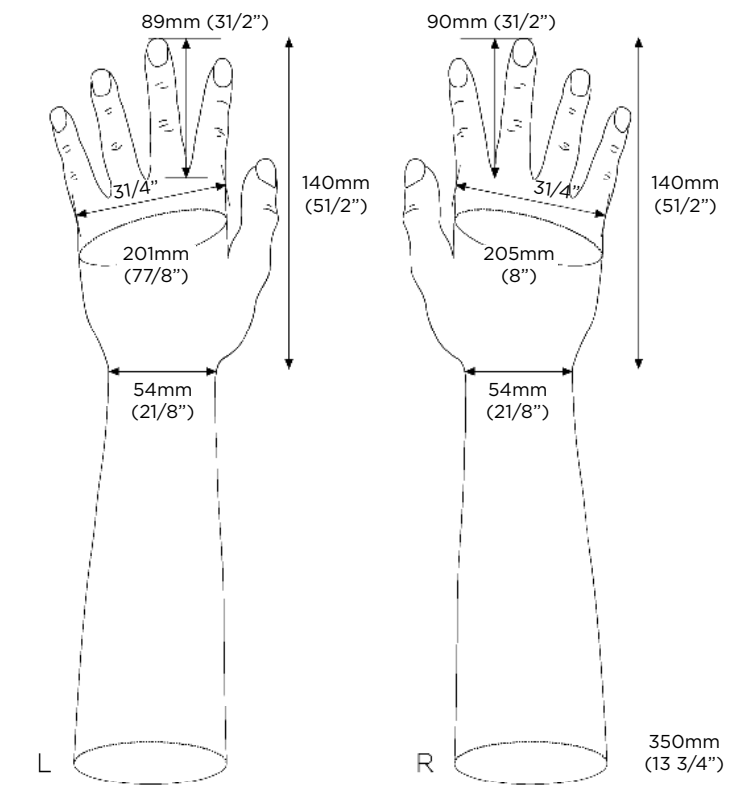

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11055	B11056
Foam Hand with Extended Wires	B11055EX	B11056EX

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG103/E#	TSG104/E#
Standard Silicone Glove	SG103/E#	SG104/E#
TrueFinish™ PVC Glove	TG103/E#	TG104/E#
Standard PVC Glove	CG103/E#	CG104/E#

Adult Glove | L1 & L2
Male, Long Mould, 3 1/2"

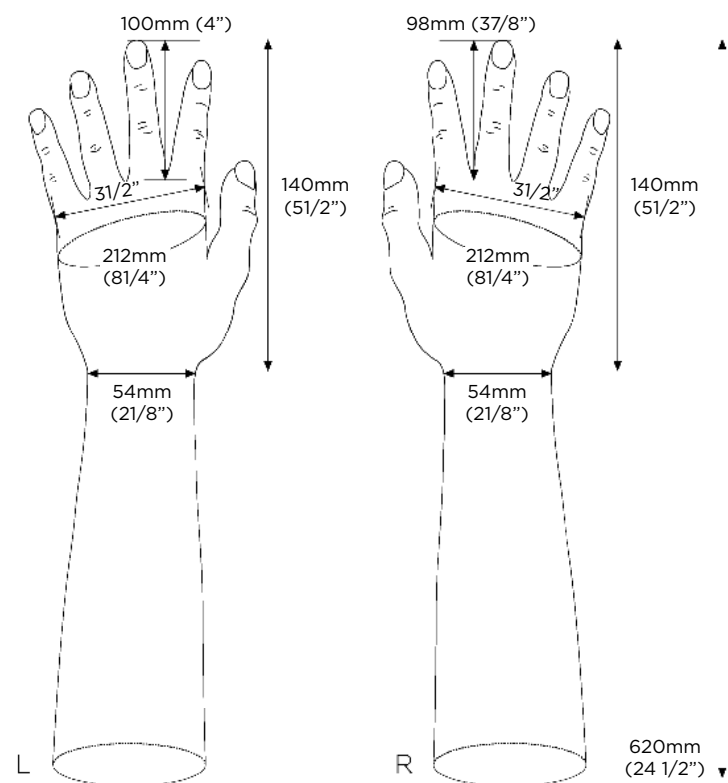

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	FHL1	FHL2
Foam Hand with Extended Wires	FHEXL1	FHEXL2

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSGL1/E#	TSGL2/E#
Standard Silicone Glove	SGL1/E#	SGL2/E#
TrueFinish™ PVC Glove	TGL1/E#	TGL2/E#
Standard PVC Glove	CGL1/E#	CGL2/E#

Adult Glove | 1 & 2
Male, 3 1/2"

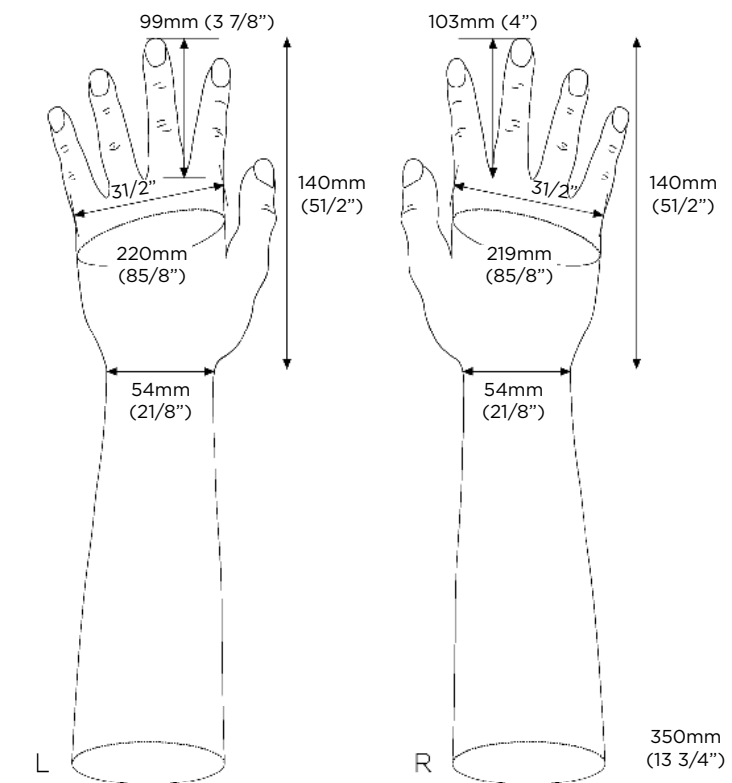

Cosmetic/Passive Hands Ordering Information


	Left	Right
Foam Hand	B11053	B11054
Foam Hand with Extended Wires	B11083	B11084

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG1/E#	TSG2/E#
Standard Silicone Glove	SG1/E#	SG2/E#
TrueFinish™ PVC Glove	TG1/E#	TG2/E#
Standard PVC Glove	CG1/E#	CG2/E#

Adult Glove | 200 & 201
Male, 3¾"

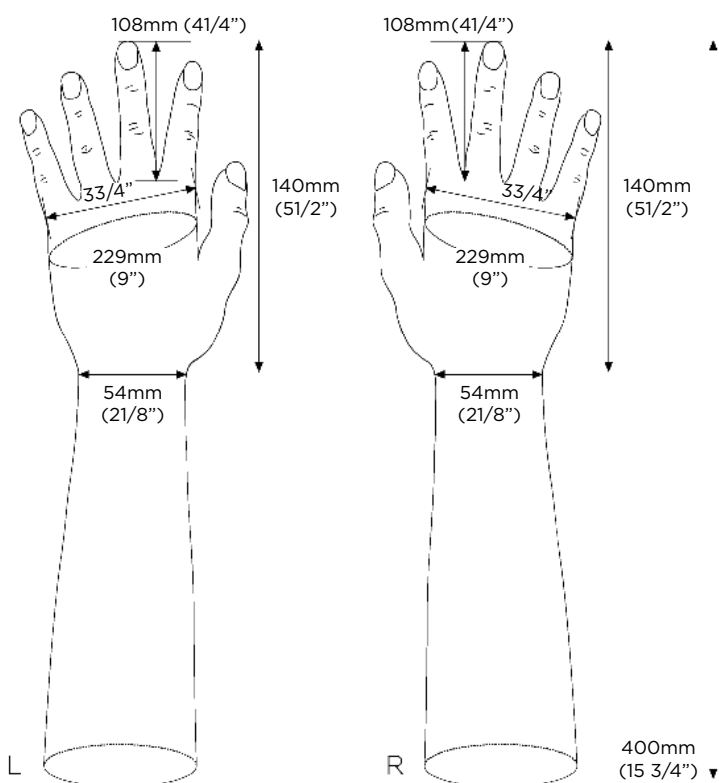

Cosmetic/Passive Hands Ordering Information

	Left	Right
Foam Hand	B21139	B21140
Foam Hand with Extended Wires	B21139EX	B21140EX

Cosmetic Glove Ordering Information

Replace # with the chosen TrueFinish™ or Standard colour EO-E18, see page 89 for swatches.

	Left	Right
TrueFinish™ Silicone Glove	TSG200/E#	TSG201/E#
Standard Silicone Glove	SG200/E#	SG201/E#
TrueFinish™ PVC Glove	TG200/E#	TG201/E#
Standard PVC Glove	CG200/E#	CG201/E#

Cosmetic Glove Order Form

Cosmetic Gloves						
CG / TG / SG / TSG	Glove Number	Colour (EO - E18)	Glove Quantity	Foam Hand Required	With Extended Wires?	Hand Quantity
e.g. TG	L1	E4	3	YES	NO	1

Two Tone (CG and TG only)						
TG/CG	Glove Number	Palmar Colour (EO - E18)	Dorsal Colour (E13, E16, E18)	Glove Quantity	Foam Hand Required	Hand Quantity
e.g. TG	L22	E4	E16	1	NO	0

Zip Fitting *PVC Gloves only

Length Required from Top of Glove to Thumb Tip:

Position of Zip (please tick): Palmar Dorsal
 Zip Length (4-8 Inch): 4" 5" 6" 7" 8"

Non-Standard Finishes (Please tick)	
Uncut	<input type="checkbox"/>
Unpainted Nails	<input type="checkbox"/>
Bilateral	<input type="checkbox"/>

Please note: This information must be provided at the point of order to ensure the correct finishes/non finishes are applied to the glove. For bi-lateral gloves the importance is to ensure unity in the shade chosen by ensuring gloves are manufactured from the same batch of PVC or Silicone.

Customer Details

Name: _____
 Address: _____

 Date: _____
 Email: _____
 P/O No.: _____

Other Remarks/Requirements

