

PIPDriver™ | 2nd Generation

The PIPDriver by Naked Prosthetics is a custom body-driven mechanical finger, designed to restore length and dexterity using the intact PIP joint to provide DIP flexion/extension.

This prosthesis is intended for use on a finger absence at, or just proximal, to the distal interphalangeal (DIP) joint, where there is some joint range of motion and enough residual digit to engage the distal suspension ring at the PIP joint.


Manufactured using medical-grade Nylon 12, the PIPDriver is a durable yet lightweight device, capable of carrying out everyday tasks as it mimics the anatomical motion of the finger.

The force output of the device is dependent on the patient's strength - whatever force the individual is capable of inputting will be safely transmitted by the PIPDriver.

Features and Benefits

- Self-suspended device fabricated to the individual's residual digit size and length
- Cage-like structure protects the amputation site of without making contact with the residuum end
- Silicone rubber tips mimic the grip and resemble the natural fingertip
- The PIPDriver is easy to don and doff, and also simple to clean and care for
- Choice of colour coating
- Full set of silicone shims in three sizes provided so patients can adjust fit in accordance with volume fluctuations
- Minimalist and organic nail options available
- Can be used in combination with other devices in the Naked Prosthetics range

Amputation Level:

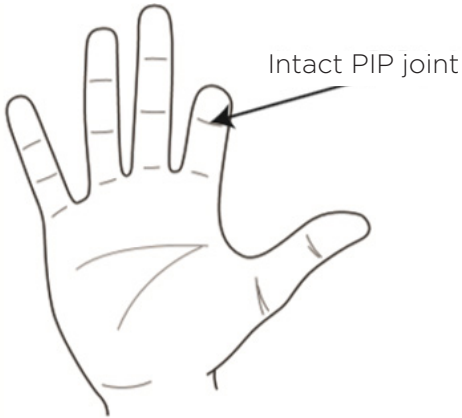

PIPDriver™ Technical Information

Materials	Nylon 12, silicone, titanium
Resistant To	Water, dirt, dust, alcohol, acetone, isopropanol
Heat Resistance	< 175°C (347°F)
Force Ouput	Same as intact finger
Ring Max. Tensile Load	27kg (60lbs)
Warranty	1 year (extended warranty options available)

PIPDriver™ Ordering Information

Product Code	NPIP-01
1-Year Warranty Code	NEW1AY-PIP-01
2-Year Warranty Code	NEW2AY-PIP-01

PIPDriver™ Nail Options



Please note: If ordering a combination of devices for one patient, the MCP and ThumbDrivers only come in minimalist style.

PIPDriver™ Colour Coating Options



*Colour swatches available upon request.

PIPDriver™ Prescription Process

To order a Naked Prosthetic device, you will require sizing rings, colour swatches, and need to fill in the Patient Intake Form. This form will be used to assess if a device(s) will be suitable for the patient to ensure a positive outcome.

Please contact Steeper for sizing rings, swatches and clinical support.

Required for PIPDriver prescription:

- Completed Patient Intake Form (PIF)
- High resolution photos of impaired digit next to a ruler:
 1. Fingers extended, palmar view
 2. Fingers extended, dorsal view
- Range of motion videos:
 1. Full flexion and extension, palmar view
 2. Full flexion and extension, sagittal view
- Photos with sizing rings on
- Device sizing form (on the PIF)
- Colour coating selected
- Nail style selected